

**INTOXILYZER 8000 INSPECTION**

Serial No.: 80-003059 Location: TOXL

- |  | Check When Done |
|--|-----------------|
| A. Is the warm-up time less than 20 minutes?   | <u>✓</u>        |
| B. Is a three-pronged grounded outlet used?  | <u>✓</u>        |
| C. Is the breath tube heated?  | <u>✓</u>        |
| D. Is the diagnostic check complete?   | <u>✓</u>        |
| E. Is the time, date, and year correct (re-set if necessary)?  | <u>✓</u>        |
| F. Print test. (Attach test record.)   | <u>✓</u>        |
| G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)<br>Sim SN <u>DR5144</u> Lot No. <u>—</u> Rep AC <u>0.021</u>  | <u>✓</u>        |
| H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)<br>Sim SN <u>DR3847</u> Lot No. <u>—</u> Rep AC <u>0.294</u>   | <u>✓</u>        |
| I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.<br>(Attach test record.)<br>Sim SN <u>DR3452</u> Lot No. <u>—</u> Rep AC <u>0.10AC + Acetone</u> | <u>✓</u>        |
| J. RFI check. Run CMS mode. Key radio on first room air.<br>(Attach test record.)  | <u>✓</u>        |
| K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.<br>(Attach test record.)  |                 |

Lot No. <u>15410080A1</u>	Cyl No. <u>21</u>	Exp. Date <u>7-1-12</u>
Test 1 <u>0.080</u>	Test 4 <u>0.079</u>	Test 7 <u>0.080</u>
Test 2 <u>0.079</u>	Test 5 <u>0.079</u>	Test 8 <u>0.079</u>
Test 3 <u>0.079</u>	Test 6 <u>0.080</u>	Test 9 <u>0.080</u>
Average <u>0.079</u>		

L. Tank pressure, Level 3 Func D Sub G P: 246 psi Reg 250 psi

M. Remarks/Maintenance record: Annual Inspection

5-18-11  
Date

[Signature]  
Field Inspector's Signature

19 May 2011  
Date

Deb Kashner  
Reviewed by

80-003059  
P. 111 of 119  
JFM



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003059  
Location = TOXL      8164.13.00 06/09  
05/18/2011      10:15

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:15
02 Std. Sol.	0.019	10:16
03 Room Air	0.000	10:17
04 Std. Sol.	0.020	10:18
05 Room Air	0.000	10:18
06 Std. Sol.	0.020	10:19
07 Room Air	0.000	10:19


08 Sim Temp = 34.0°C

Simul Ser No = 0

Std Sol No = 0

County = 08

Oper No. = 555555

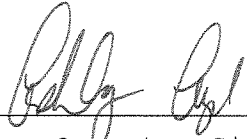

Operator Signature  
ASHLEY AYD

Remarks: *Low AC 0.021*

Form 106-I8000

80-003059  
P 113 of 119  
FM

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003059  
Location = TOXL      8164.13.00 06/09  
05/18/2011      10:20

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:20
02 Std. Sol.	0.292	10:21
03 Room Air	0.000	10:22
04 Std. Sol.	0.295	10:23
05 Room Air	0.000	10:23
06 Std. Sol.	0.294	10:24
07 Room Air	0.000	10:24


08 Sim Temp = 34.0°C

Simul Ser No = 0

Std Sol No = 0

County = 08

Oper No. = 555555

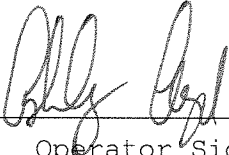

Operator Signature  
ASHLEY AYD

Remarks: *Linearity Check 0.294 AC*

Form 106-I8000

80-003059  
P. 114 of 119  
EM

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-003059  
Location = TOXL                8164.13.00 06/09  
05/18/2011                        13:27

Test	AC	Time
01 Room Air	0.000	13:28
02 *Subject Test	INT*	13:28
03 Room Air	0.000	13:29

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS821456  
Lot No = 15410080A1  
Cyl No = 21  
Expiration Date = 07/01/2012  
County = 08                          Oper No. = 555555



Operator Signature  
ASHLEY AYD

Remarks: *Interferent check 0.10AC +Acetone*

Form 106-I8000

80-003059  
P. 115 of 119  
FM




Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003059  
Location = TOXL      8164.13.00 06/09  
05/18/2011      11:21

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:22
02 Std. Gas	0.080	11:22
03 Room Air	0.000	11:22
04 Std. Gas	0.079	11:23
05 Room Air	0.000	11:23
06 Std. Gas	0.079	11:24
07 Room Air	0.000	11:24

Lot No = 15410080A1  
Cyl No = 21  
Exp Date = 07/01/2012  
County = 08      Oper No. = 555555

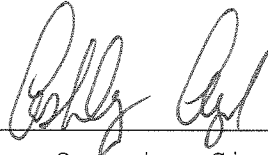

Operator Signature  
ASHLEY AYD

Remarks: Calibration check 0.080 AC

Form 106-I8000

80-003059  
P. 117 of 119  
VAM


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003059  
Location = TOXL      8164.13.00 06/09  
05/18/2011      12:31

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:31
02 Std. Gas	0.079	12:31
03 Room Air	0.000	12:32
04 Std. Gas	0.079	12:32
05 Room Air	0.000	12:33
06 Std. Gas	0.080	12:33
07 Room Air	0.000	12:34

Lot No = 15410080A1  
Cyl No = 21  
Exp Date = 07/01/2012  
County = 08      Oper No. = 555555

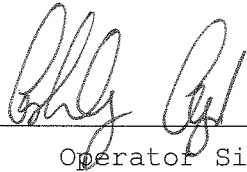

Operator Signature  
ASHLEY AYD

Remarks: Calibration check 0.080AC

Form 106-I8000

80-003059  
P. 118 & 119  
FM




Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003059  
Location = TOXL      8164.13.00 06/09  
05/18/2011      12:35

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:35
02 Std. Gas	0.080	12:36
03 Room Air	0.000	12:36
04 Std. Gas	0.079	12:36
05 Room Air	0.000	12:37
06 Std. Gas	0.080	12:37
07 Room Air	0.000	12:38

Lot No = 15410080A1  
Cyl No = 21  
Exp Date = 07/01/2012  
County = 08      Oper No. = 555555

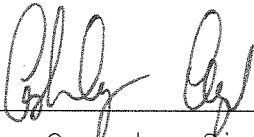  
\_\_\_\_\_  
Operator Signature  
ASHLEY AYD

Remarks: Calibration Check 0.080 AC

Form 106-I8000

80-003059  
P. 119 of 119  
JFM