

INTOXILYZER 8000 INSPECTION

Serial No.: 80-003059

Location: TOXL

- | | Check When Done |
|---|-----------------|
| A. Is the warm-up time less than 20 minutes? | <u>✓</u> |
| B. Is a three-pronged grounded outlet used? | <u>✓</u> |
| C. Is the breath tube heated? | <u>✓</u> |
| D. Is the diagnostic check complete? | <u>✓</u> |
| E. Is the time, date, and year correct (re-set if necessary)? | <u>✓</u> |
| F. Print test. (Attach test record.) | <u>✓</u> |
| G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)
Sim SN <u>DR3455</u> Lot No. <u>10290</u> Rep AC <u>0.020</u> | <u>✓</u> |
| H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN <u>DR3847</u> Lot No. <u>11110</u> Rep AC <u>0.300</u> | <u>✓</u> |
| I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.
(Attach test record.)
Sim SN <u>DR3377</u> Lot No. <u>—</u> Rep AC <u>0.100</u> | <u>✓</u> |
| J. RFI check. Run CMS mode. Key radio on first room air.
(Attach test record.) | <u>✓</u> |
| K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) | |

Lot No. <u>05911080A1</u>	Cyl No. <u>4</u>	Exp. Date <u>2/1/2013</u>
Test 1 <u>0.079</u>	Test 4 <u>0.079</u>	Test 7 <u>0.079</u>
Test 2 <u>0.078</u>	Test 5 <u>0.078</u>	Test 8 <u>0.079</u>
Test 3 <u>0.078</u>	Test 6 <u>0.078</u>	Test 9 <u>0.079</u>
Average <u>0.079</u>		

L. Tank pressure, Level 3 Func D Sub G P: 486 psi Reg 500 psi

M. Remarks/Maintenance record: ANNUAL INSPECTION, REPLACED REGULATOR, REATTACHED MODEM PORT

18 MAY 12
Date

Charles E. Eden
Field Inspector's Signature

21 May 2012
Date

Deb Kashur
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003059
Location = TOXL 8164.13.00 06/09
05/18/2012 14:49

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 05911080A1
Standard Cyl #? 4
Standard Expiration? 02/01/2013
Oper No? 666666

Flow Cal. Date: 04/07/2008
Slope 713
Intercept -692349

IR Calibration Date: 06/04/2008
 3um 9um

	3um	9um
0th Coef(*100):	-15808	-12583
1st Coef(*100):	279401	139580
2nd Coef(*100):	1046	760
H2O adj(mg/l*10k):	478	208

***** Printer Test End *****



Operator Signature
CHARLES EDER

Remarks: PRINT TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003059
Location = TOXL 8164.13.00 06/09
05/18/2012 14:58

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:58
02 Std. Sol.	0.018	14:59
03 Room Air	0.000	15:00
04 Std. Sol.	0.017	15:00
05 Room Air	0.000	15:01
06 Std. Sol.	0.018	15:02
07 Room Air	0.000	15:02

08 Sim Temp = 34.0°C

Simul Ser No = 000000

Std Sol No = 0000

County = 08

Oper No. = 666666



Operator Signature

CHARLES EDER

Remarks:

Low AC
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003059
Location = TOXL 8164.13.00 06/09
05/18/2012 15:04

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:05
02 Std. Sol.	0.290	15:07
03 Room Air	0.000	15:07
04 Std. Sol.	0.291	15:08
05 Room Air	0.000	15:08
06 Std. Sol.	0.292	15:09
07 Room Air	0.000	15:10

08 Sim Temp = 34.0°C

Simul Ser No = 000000
Std Sol No = 0000
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: LINEARITY TEST
 0.300 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003059
Location = TOXL 8164.13.00 06/09
05/18/2012 15:15

Test	AC	Time
01 Room Air	0.000	15:15
02 *Subject Test	INT*	15:16
03 Room Air	0.000	15:17

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = 1234567890QWERT
Dr. Lic. = ND/DIS821456
Simul Ser No = 000000
Std Sol No = 0000
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *INTERFERENT CHECK*
0.100AC + ACETONE

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003059
Location = TOXL 8164.13.00 06/09
05/18/2012 15:18

Test	AC	Time
01 Diagnostic	OK	15:19
02 Room Air	RFI*	15:19
03 Room Air	0.000	15:19

*Invalid Test
 Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = YUIOPASDFGHJKL
Dr. Lic. = ND/DIS821456
Simul Ser No = 000000
Std Sol No = 0000
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks: *RFI CHECK*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003059
Location = TOXL 8164.13.00 06/09
05/18/2012 15:21

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:21
02 Std. Gas	0.079	15:22
03 Room Air	0.000	15:22
04 Std. Gas	0.078	15:23
05 Room Air	0.000	15:23
06 Std. Gas	0.078	15:24
07 Room Air	0.000	15:24

Lot No = 05911080A1
Cyl No = 4
Exp Date = 02/01/2013
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003059
Location = TOXL 8164.13.00 06/09
05/18/2012 15:27

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:27
02 Std. Gas	0.079	15:28
03 Room Air	0.000	15:28
04 Std. Gas	0.078	15:29
05 Room Air	0.000	15:29
06 Std. Gas	0.078	15:29
07 Room Air	0.000	15:30

Lot No = 05911080A1
Cyl No = 4
Exp Date = 02/01/2013
County = 08 Oper No. = 666666


Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003059
Location = TOXL 8164.13.00 06/09
05/18/2012 15:32

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:32
02 Std. Gas	0.079	15:33
03 Room Air	0.000	15:33
04 Std. Gas	0.079	15:33
05 Room Air	0.000	15:34
06 Std. Gas	0.079	15:34
07 Room Air	0.000	15:35

Lot No = 05911080A1
Cyl No = 4
Exp Date = 02/01/2013
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080AC