

INTOXILYZER 8000 INSPECTION

Serial No.: 80-004949

Location: TOXL

- | | Check When Done |
|--|-----------------|
| A. Is the warm-up time less than 20 minutes? | <u>✓</u> |
| B. Is a three-pronged grounded outlet used? | <u>✓</u> |
| C. Is the breath tube heated? | <u>✓</u> |
| D. Is the diagnostic check complete? | <u>✓</u> |
| E. Is the time, date, and year correct (re-set if necessary)? | <u>✓</u> |
| F. Print test. (Attach test record.) | <u>✓</u> |
| G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)
Sim SN <u>DRS144</u> Lot No. <u>—</u> Rep AC <u>0.021</u> | <u>✓</u> |
| H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN <u>DR3847</u> Lot No. <u>—</u> Rep AC <u>0.294</u> | <u>✓</u> |
| I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.
(Attach test record.)
Sim SN <u>DR3452</u> Lot No. <u>—</u> Rep AC <u>0.10AC + Acetone</u> | <u>✓</u> |
| J. RFI check. Run CMS mode. Key radio on first room air.
(Attach test record.) | <u>✓</u> |
| K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) | |

Lot No. <u>15410080A1</u>	Cyl No. <u>21</u>	Exp. Date <u>7-1-12</u>
Test 1 <u>0.078</u>	Test 4 <u>0.079</u>	Test 7 <u>0.079</u>
Test 2 <u>0.078</u>	Test 5 <u>0.079</u>	Test 8 <u>0.079</u>
Test 3 <u>0.078</u>	Test 6 <u>0.079</u>	Test 9 <u>0.079</u>
Average <u>0.079</u>		

L. Tank pressure, Level 3 Func D Sub G P: 189 psi Reg 200 psi

M. Remarks/Maintenance record: Annual Inspection

5-19-11
Date
19 May 2011
Date

[Signature]
Field Inspector's Signature
Deb Keshner
Reviewed by

80-004949
P. 103 of 111
AM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004949
Location = TOXL 8164.13.00 06/09
05/19/2011 09:00

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 15410080A1
Standard Cyl #? 21
Standard Expiration? 07/01/2012
Oper No? 555555

Flow Cal. Date: 09/24/2010
Slope 784
Intercept -862571

IR Calibration Date: 09/24/2010
 3um 9um

0th Coef(*100): -23764 -22121
1st Coef(*100): 267615 132337
2nd Coef(*100): 1069 992
H2O adj(mg/l*10k): 531 480

***** Printer Test End *****

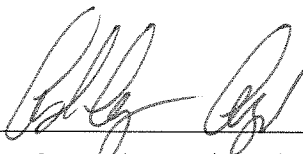

Operator Signature
ASHLEY AYD

Remarks: *Print test*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004949
Location = TOXL 8164.13.00 06/09
05/19/2011 09:15

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:15
02 Std. Sol.	0.019	09:16
03 Room Air	0.000	09:17
04 Std. Sol.	0.019	09:17
05 Room Air	0.000	09:18
06 Std. Sol.	0.019	09:19
07 Room Air	0.000	09:19


08 Sim Temp = 34.0°C

Simul Ser No = 0

Std Sol No = 0

County = 08

Oper No. = 555555

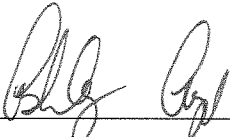

Operator Signature
ASHLEY AYD

Remarks: *Low AC 0.021 AC*

Form 106-I8000

80-004949
P.105 of 111
FM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004949
Location = TOXL 8164.13.00 06/09
05/19/2011 09:19

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:20
02 Std. Sol.	0.291	09:21
03 Room Air	0.000	09:22
04 Std. Sol.	0.292	09:23
05 Room Air	0.000	09:23
06 Std. Sol.	0.292	09:24
07 Room Air	0.000	09:25


08 Sim Temp = 34.0°C

Simul Ser No = 0

Std Sol No = 0

County = 08

Oper No. = 555555

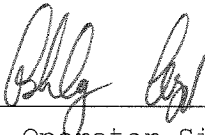

Operator Signature
ASHLEY AYD

Remarks: *Linearity test 0.294AC*

Form 106-I8000

80-004949
P. 106 of 111
VM


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004949
Location = TOXL 8164.13.00 06/09
05/19/2011 09:25

Test	AC	Time
01 Room Air	0.000	09:26
02 *Subject Test	INT*	09:27
03 Room Air	0.000	09:27

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Simul Ser No = 0
Std Sol No = 0
County = 08 Oper No. = 555555



Operator Signature
ASHLEY AYD

Remarks: *Interferent check 0.10AC + Acetone*

Form 106-I8000

80-004949
P. 107 of 111
AM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004949
Location = TOXL 8164.13.00 06/09
05/19/2011 09:28

Test	AC	Time
01 Diagnostic	OK	09:29
02 Room Air	RFI*	09:29
03 Room Air	0.000	09:30

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Simul Ser No = 0
Std Sol No = 0
County = 08 Oper No. = 555555

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
ASHLEY AYD

Remarks: RFI

Form 106-I8000

80-004949
P. 108 of 111
vcm


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004949
Location = TOXL 8164.13.00 06/09
05/19/2011 09:01

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:02
02 Std. Gas	0.078	09:02
03 Room Air	0.000	09:02
04 Std. Gas	0.078	09:03
05 Room Air	0.000	09:03
06 Std. Gas	0.078	09:04
07 Room Air	0.000	09:04

Lot No = 15410080A1
Cyl No = 21
Exp Date = 07/01/2012
County = 08 Oper No. = 555555



Operator Signature
ASHLEY AYD

Remarks: Calibration check 0.080AC

Form 106-I8000

80-004949
p. 109 of 111
JFM


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004949
Location = TOXL 8164.13.00 06/09
05/19/2011 09:04

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:05
02 Std. Gas	0.079	09:05
03 Room Air	0.000	09:06
04 Std. Gas	0.079	09:06
05 Room Air	0.000	09:07
06 Std. Gas	0.079	09:07
07 Room Air	0.000	09:08

Lot No = 15410080A1
Cyl No = 21
Exp Date = 07/01/2012
County = 08 Oper No. = 555555

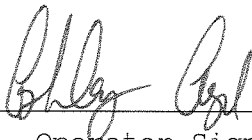

Operator Signature
ASHLEY AYD

Remarks: Calibration checks 0.080AC

Form 106-I8000

80-004949
P. 110 of 111
AM


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004949
Location = TOXL 8164.13.00 06/09
05/19/2011 09:08

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:09
02 Std. Gas	0.079	09:09
03 Room Air	0.000	09:10
04 Std. Gas	0.079	09:10
05 Room Air	0.000	09:11
06 Std. Gas	0.079	09:11
07 Room Air	0.000	09:12

Lot No = 15410080A1
Cyl No = 21
Exp Date = 07/01/2012
County = 08 Oper No. = 555555

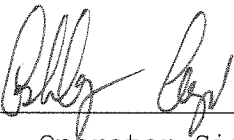

Operator Signature
ASHLEY AYD

Remarks: Calibration check 0.080AC

Form 106-I8000

80-004949
p. 111 of 111
STM