

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-005954

Location: TOXL

- | | Check When Done |
|--|-----------------|
| A. Is the breath tube heated? | <u>✓</u> |
| B. Is the diagnostic check complete? | <u>✓</u> |
| C. Is the time, date, and year correct (re-set if necessary)? | <u>✓</u> |
| D. Print test. (Attach test record.) | <u>✓</u> |
| E. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)
Sim SN <u>DR337T</u> Lot No. <u>13270</u> Rep AC <u>0.020</u> | <u>✓</u> |
| F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN <u>DR5190</u> Lot No. <u>12105</u> Rep AC <u>0.400</u> | <u>✓</u> |
| G. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.
(Attach test record.)
Sim SN <u>DR5135</u> Lot No. <u>IS#2</u> Rep AC <u>0.100AC+0.05% ACETONE</u> | <u>✓</u> |
| H. RFI check. Run CMS mode. Key radio on first room air.
(Attach test record.) | <u>✓</u> |
| I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) | |

Lot No. <u>16512080A2</u>	Cyl No. <u>019</u>	Exp. Date <u>08/01/2014</u>
Test 1 <u>0.080</u>	Test 4 <u>0.080</u>	Test 7 <u>0.081</u>
Test 2 <u>0.081</u>	Test 5 <u>0.081</u>	Test 8 <u>0.081</u>
Test 3 <u>0.080</u>	Test 6 <u>0.080</u>	Test 9 <u>0.080</u>
Average <u>0.080</u>		

J. Tank pressure, Level 3 Func D Sub G P: 825 psi Reg 800 psi

K. Remarks/Maintenance record: ANNUAL INSPECTION

19 MAY 14
Date

19 May 14
Date

Charles E. Eds
Field Inspector's Signature

Deb Kashner
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005954
Location = TOXL 8164.13.00 06/09
05/19/2014 13:22

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&*()_+?

abcdefghijklmnopqrstuvwxy1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 16512080A2
Standard Cyl #? 19
Standard Expiration? 08/01/2014
Oper No? 666666

Flow Cal. Date: 05/09/2014
Slope 647
Intercept -668435

IR Calibration Date: 05/09/2014

	3um	9um
0th Coef(*100):	-23295	-17360
1st Coef(*100):	283612	136126
2nd Coef(*100):	1929	1289
H2O adj(mg/l*10k):	481	487

***** Printer Test End *****



Operator Signature
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005954
Location = TOXL 8164.13.00 06/09
05/19/2014 13:25

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:26
02 Std. Sol.	0.018	13:26
03 Room Air	0.000	13:27
04 Std. Sol.	0.018	13:28
05 Room Air	0.000	13:28
06 Std. Sol.	0.018	13:29
07 Room Air	0.000	13:29

08 Sim Temp = 34.0°C

Simul Ser No = DR3377

Std Sol No = 13270

County = 08

Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Low AC

0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005954
Location = TOXL 8164.13.00 06/09
05/19/2014 13:31

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:31
02 Std. Sol.	0.398	13:32
03 Room Air	0.000	13:33
04 Std. Sol.	0.397	13:34
05 Room Air	0.000	13:34
06 Std. Sol.	0.397	13:35
07 Room Air	0.000	13:35

08 Sim Temp = 34.0°C

Simul Ser No = DR5190
Std Sol No = 12105
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

HIGH AC
0.400 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005954
Location = TOXL 8164.13.00 06/09
05/19/2014 13:41

Test	AC	Time
01 Room Air	0.000	13:42
02 *Subject Test	INT*	13:43
03 Room Air	0.000	13:43

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 16512080A2
Cyl No = 19
Expiration Date = 08/01/2014
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *INTERFERENT CHECK*
0.100AC + 0.05% ACETONE

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005954
Location = TOXL 8164.13.00 06/09
05/19/2014 13:48

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:48
02 Std. Gas	0.080	13:49
03 Room Air	0.000	13:49
04 Std. Gas	0.081	13:49
05 Room Air	0.000	13:50
06 Std. Gas	0.080	13:50
07 Room Air	0.000	13:51

Lot No = 16512080A2
Cyl No = 19
Exp Date = 08/01/2014
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *CALIBRATION CHECK*
0.080 AC

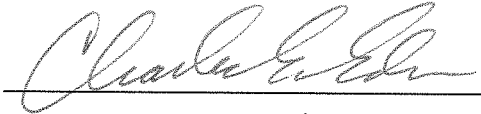
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005954
Location = TOXL 8164.13.00 06/09
05/19/2014 13:52

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:52
02 Std. Gas	0.080	13:52
03 Room Air	0.000	13:53
04 Std. Gas	0.081	13:53
05 Room Air	0.000	13:54
06 Std. Gas	0.080	13:54
07 Room Air	0.000	13:55

Lot No = 16512080A2
Cyl No = 19
Exp Date = 08/01/2014
County = 08 Oper No. = 666666


Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005954
Location = TOXL 8164.13.00 06/09
05/19/2014 13:55

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:56
02 Std. Gas	0.081	13:56
03 Room Air	0.000	13:57
04 Std. Gas	0.081	13:57
05 Room Air	0.000	13:58
06 Std. Gas	0.080	13:58
07 Room Air	0.000	13:59

Lot No = 16512080A2
Cyl No = 19
Exp Date = 08/01/2014
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *CALIBRATION CHECK*
0.080AC