

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-004205

Location: TOXL

- | | Check When Done |
|---|-----------------|
| A. Is the breath tube heated? | <u>✓</u> |
| B. Is the diagnostic check complete? | <u>✓</u> |
| C. Is the time, date, and year correct (re-set if necessary)? | <u>✓</u> |
| D. Print test. (Attach test record.) | <u>✓</u> |
| E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.) Sim SN <u>MP3066</u> Lot No. <u>201507F</u> Rep AC <u>0.020</u> | <u>✓</u> |
| F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.) Sim SN <u>MP3048</u> Lot No. <u>201603A</u> Rep AC <u>0.300</u> | <u>✓</u> |
| G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode. (Attach test record.) Sim SN <u>DR5135</u> Lot No. <u>ICS4</u> Solution <u>0.10 AC + 0.05% ACETONE</u> | <u>✓</u> |
| H. RFI check. Run CMS mode. Key radio during CMS test. (Attach test record.) | <u>✓</u> |
| I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode. (Attach test record.) <u>0.080</u> AC | |

Lot No. 1331408043 Cyl No. 034 Exp. Date 06/05/2016

Test 1 0.079 Test 4 0.078 Test 7 0.078

Test 2 0.078 Test 5 0.078 Test 8 0.078

Test 3 0.078 Test 6 0.079 Test 9 0.078

Average 0.078

J. Tank pressure, Level 3 Func D Sub G P: 949 psi Reg ~950 psi

K. Remarks/Maintenance record: ANNUAL INSPECTION, REPLACED EXTERNAL BATTERY,

BREATH HOSE & SIMULATOR RETURN O-RINGS REPLACED (DEB.S.) BREATH SCREEN CLEANED (DEB.S.)

5/18/2016
Date

Charles E. Ed
Field Inspector's Signature

19 May 16 ✓
Date

Deb Shanover
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004205
Location = TOXL 8164.13.00 06/09
05/18/2016 13:29

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-|=|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-|=|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%&*()_+?


Current Instrument Setup
Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer (Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 13314080A3
Standard Cyl #? 34
Standard Expiration? 06/05/2016
Oper No? 666666

Flow Cal. Date: 06/04/2009
Slope 704
Intercept -610062

IR Calibration Date: 04/17/2015

| | 3um | 9um |
|--------------------|--------|--------|
| 0th Coef(*100): | -26411 | -27902 |
| 1st Coef(*100): | 266995 | 129434 |
| 2nd Coef(*100): | 2919 | 1480 |
| H2O adj(mg/l*10k): | 363 | 481 |

***** Printer Test End *****

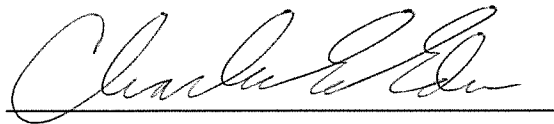

Operator Signature
CHARLES EDER

Remarks: PRINT TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004205
Location = TOXL 8164.13.00 06/09
05/18/2016 13:31

WET CAL CHECK

| Test | AC | Time |
|--------------|-------|-------|
| 01 Room Air | 0.000 | 13:31 |
| 02 Std. Sol. | 0.018 | 13:32 |
| 03 Room Air | 0.000 | 13:33 |
| 04 Std. Sol. | 0.018 | 13:33 |
| 05 Room Air | 0.000 | 13:34 |
| 06 Std. Sol. | 0.018 | 13:35 |
| 07 Room Air | 0.000 | 13:35 |

08 Sim Temp = 34.0°C

Simul Ser No = MP3066
Std Sol No = 201507F
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *Low AC*
 0.020 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004205
Location = TOXL 8164.13.00 06/09
05/18/2016 13:48

WET CAL CHECK

| Test | AC | Time |
|--------------|-------|-------|
| 01 Room Air | 0.000 | 13:48 |
| 02 Std. Sol. | 0.298 | 13:49 |
| 03 Room Air | 0.000 | 13:50 |
| 04 Std. Sol. | 0.300 | 13:50 |
| 05 Room Air | 0.000 | 13:51 |
| 06 Std. Sol. | 0.300 | 13:52 |
| 07 Room Air | 0.000 | 13:52 |

08 Sim Temp = 34.0°C

Simul Ser No = MP3048
Std Sol No = 201603A
County = 08 Oper No. = 666666

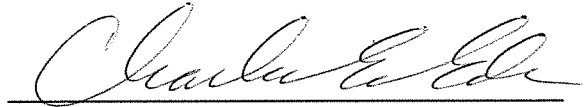

Operator Signature
CHARLES EDER

Remarks: HIGH AC
0.300 AC


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004205
Location = TOXL 8164.13.00 06/09
05/18/2016 13:54

| Test | AC | Time |
|------------------|-------|-------|
| 01 Room Air | 0.000 | 13:55 |
| 02 *Subject Test | INT* | 13:56 |
| 03 Room Air | 0.000 | 13:56 |

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 13314080A3
Cyl No = 34
Expiration Date = 06/05/2016
County = 08 Oper No. = 666666

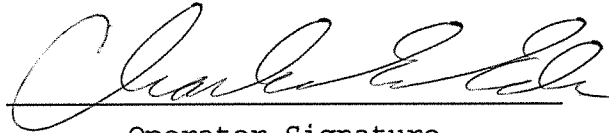

Operator Signature
CHARLES EDER

Remarks:

*Interferent CHECK
0.10 AC + 0.05% ACETONE*

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004205
Location = TOXL 8164.13.00 06/09
05/18/2016 13:59

| Test | AC | Time |
|-------------------|-------|-------|
| 01 Diagnostic | OK | 14:00 |
| 02 Room Air | 0.000 | 14:00 |
| 03 Subject Test 1 | 0.000 | 14:00 |
| 04 Room Air | 0.000 | 14:03 |
| 05 Std. Gas | 0.078 | 14:04 |
| 06 Room Air | 0.000 | 14:05 |
| 07 *Subject Test | RFI* | 14:05 |
| 08 Room Air | 0.000 | 14:06 |

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 13314080A3
Cyl No = 34
Expiration Date = 06/05/2016
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks: *RFI CHECK*


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004205
Location = TOXL 8164.13.00 06/09
05/18/2016 14:14

DRY CAL CHECK

| Test | AC | Time |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 14:15 |
| 02 Std. Gas | 0.079 | 14:15 |
| 03 Room Air | 0.000 | 14:16 |
| 04 Std. Gas | 0.078 | 14:16 |
| 05 Room Air | 0.000 | 14:16 |
| 06 Std. Gas | 0.078 | 14:17 |
| 07 Room Air | 0.000 | 14:17 |

Lot No = 13314080A3
Cyl No = 34
Exp Date = 06/05/2016
County = 08 Oper No. = 666666

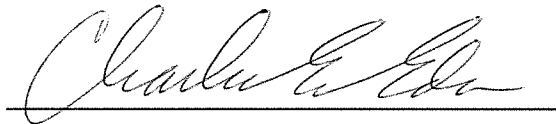

Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004205
Location = TOXL 8164.13.00 06/09
05/18/2016 14:18

DRY CAL CHECK

| Test | AC | Time |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 14:18 |
| 02 Std. Gas | 0.078 | 14:19 |
| 03 Room Air | 0.000 | 14:19 |
| 04 Std. Gas | 0.078 | 14:20 |
| 05 Room Air | 0.000 | 14:20 |
| 06 Std. Gas | 0.079 | 14:20 |
| 07 Room Air | 0.000 | 14:21 |

Lot No = 13314080A3
Cyl No = 34
Exp Date = 06/05/2016
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004205
Location = TOXL 8164.13.00 06/09
05/18/2016 14:21

DRY CAL CHECK

| Test | AC | Time |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 14:22 |
| 02 Std. Gas | 0.078 | 14:22 |
| 03 Room Air | 0.000 | 14:23 |
| 04 Std. Gas | 0.078 | 14:23 |
| 05 Room Air | 0.000 | 14:24 |
| 06 Std. Gas | 0.078 | 14:24 |
| 07 Room Air | 0.000 | 14:25 |

Lot No = 13314080A3
Cyl No = 34
Exp Date = 06/05/2016
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC