

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-005943

Location: TOXL

- Check When Done
- A. Is the breath tube heated? ✓
 - B. Is the diagnostic check complete? ✓
 - C. Is the time, date, and year correct (re-set if necessary)? ✓
 - D. Print test. (Attach test record.) ✓
 - E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN MP3066 Lot No. 201507F Rep AC 0.070 ✓
 - F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN MP3057 Lot No. 16102 Rep AC 0.400 ✓
 - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN DR5135 Lot No. ICS4 Solution 0.10 AC + 0.05% ACETONE ✓
 - H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) ✓
 - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) 0.080 AC

Lot No. 13314080A3 Cyl No. 034 Exp. Date 06/05/2016

Test 1 <u>0.080</u>	Test 4 <u>0.080</u>	Test 7 <u>0.079</u>
Test 2 <u>0.081</u>	Test 5 <u>0.079</u>	Test 8 <u>0.080</u>
Test 3 <u>0.079</u>	Test 6 <u>0.080</u>	Test 9 <u>0.080</u>

Average 0.080

J. Tank pressure, Level 3 Func D Sub G P: 987 psi Reg ~1000 psi

K. Remarks/Maintenance record: ANNUAL INSPECTION, SIMULATOR RETURN O-RING REPLACED (DEB S.), BREATH SCREEN CLEANED (DEB S.), BREATH HOSE REPLACED DUE TO SMALL PIECE OF MULTIPIECE BROKEN OFF AT ENTRANCE, GAS REGULATOR REPLACED DUE TO READING LOW

5/17/2016
Date

Charles E. Edle
Field Inspector's Signature

Deb Shanaver
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005943
Location = TOXL 8164.13.00 06/09
05/17/2016 13:11

***** Printer Test *****

abcdefghijklmnopqrstuvxyz1234567890-|=|
ABCDEFGHIJKLMN0PQRSTUvwXYZ!@#%&*()_+?

abcdefghijklmnopqrstuvxyz1234567890-|=|
ABCDEFGHIJKLMN0PQRSTUvwXYZ!@#%&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer (Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 13314080A3
Standard Cyl #? 34
Standard Expiration? 06/05/2016
Oper No? 666666

Flow Cal. Date: 06/20/2013
Slope 686
Intercept -735246

IR Calibration Date: 05/05/2015
3um 9um

0th Coef(*100): -24459 -34413
1st Coef(*100): 288841 143291
2nd Coef(*100): 2315 1134
H2O adj(mg/l*10k): 355 403

***** Printer Test End *****

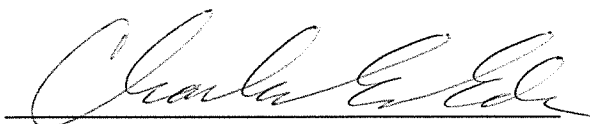

Operator Signature
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005943
Location = TOXL 8164.13.00 06/09
05/17/2016 13:25

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:25
02 Std. Sol.	0.018	13:26
03 Room Air	0.000	13:27
04 Std. Sol.	0.018	13:27
05 Room Air	0.000	13:28
06 Std. Sol.	0.017	13:29
07 Room Air	0.000	13:29

08 Sim Temp = 34.0°C

Simul Ser No = MP3066
Std Sol No = 201507F
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *Low AC*
0.020 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005943
Location = TOXL 8164.13.00 06/09
05/17/2016 13:41

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:42
02 Std. Sol.	0.398	13:42
03 Room Air	0.000	13:43
04 Std. Sol.	0.397	13:44
05 Room Air	0.000	13:44
06 Std. Sol.	0.397	13:45
07 Room Air	0.000	13:46

08 Sim Temp = 34.0°C

Simul Ser No = MP3057
Std Sol No = 16102
County = 08 Oper No. = 666666

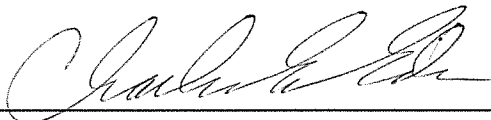

Operator Signature
CHARLES EDER

Remarks:

HIGH AC
0.400 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005943
Location = TOXL 8164.13.00 06/09
05/17/2016 14:18

Test	AC	Time
01 Room Air	0.000	14:19
02 *Subject Test	INT*	14:20
03 Room Air	0.000	14:20

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 13314080A3
Cyl No = 34
Expiration Date = 06/05/2016
County = 08 Oper No. = 666666

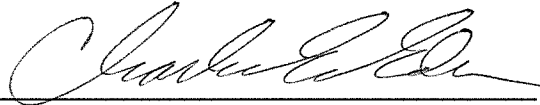

Operator Signature
CHARLES EDER

Remarks:

Interferent Check
0.10 AC + 0.05% ACETONE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005943
Location = TOXL 8164.13.00 06/09
05/17/2016 14:05

Test	AC	Time
01 Diagnostic	OK	14:05
02 Room Air	0.000	14:06
03 *Subject Test	RFI*	14:06
04 Room Air	0.000	14:07

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 13314080A3
Cyl No = 34
Expiration Date = 06/05/2016
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks:

RFI CHECK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005943
Location = TOXL 8164.13.00 06/09
05/17/2016 14:11

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:11
02 Std. Gas	0.080	14:12
03 Room Air	0.000	14:12
04 Std. Gas	0.081	14:12
05 Room Air	0.000	14:13
06 Std. Gas	0.079	14:13
07 Room Air	0.000	14:14

Lot No = 13314080A3
Cyl No = 34
Exp Date = 06/05/2016
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005943
Location = TOXL 8164.13.00 06/09
05/17/2016 14:15

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:15
02 Std. Gas	0.080	14:16
03 Room Air	0.000	14:16
04 Std. Gas	0.079	14:16
05 Room Air	0.000	14:17
06 Std. Gas	0.080	14:17
07 Room Air	0.000	14:18

Lot No = 13314080A3
Cyl No = 34
Exp Date = 06/05/2016
County = 08 Oper No. = 666666

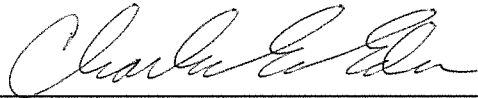

Operator Signature
CHARLES EDER

Remarks:

CALIBRATION CHECK
0.080 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005943
Location = TOXL 8164.13.00 06/09
05/17/2016 14:22

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:22
02 Std. Gas	0.079	14:23
03 Room Air	0.000	14:23
04 Std. Gas	0.080	14:23
05 Room Air	0.000	14:24
06 Std. Gas	0.080	14:24
07 Room Air	0.000	14:25

Lot No = 13314080A3

Cyl No = 34

Exp Date = 06/05/2016

County = 08

Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

CALIBRATION CHECK
0.080 AC

Form 106-I8000