



### INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

OFFICE OF ATTORNEY GENERAL  
CRIME LABORATORY DIVISION  
Toxicology Section/Breath Alcohol Program  
SFN 59281 (5-2011)

Serial Number <i>80-005946</i>	Instrument Location <i>Watford City, Police Dept.</i>
Reason for Install/Repair <input type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input checked="" type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. RFI Test (Use CMS Mode; Key Radio During First Room Air).
- 7. Repair and/or Maintenance Performed (if Any): *None Needed*
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>[Signature]</i>	Date <i>05-20-2014</i> ✓
Reviewed By (Crime Laboratory Use Only) <i>Deb Kasher</i>	Date <i>22 May 14</i>

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-005946  
Location = WCPD                    8164.13.00 06/09  
05/20/2014                            19:22

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890-=  
ABCDEFGHIJKLMNopQRSTUVWXYZ!@#\$\$%^&\*()\_+?

abcdefghijklmnopqrstuvwxy1234567890-=  
ABCDEFGHIJKLMNopQRSTUVWXYZ!@#\$\$%^&\*()\_+?

Current Instrument Setup

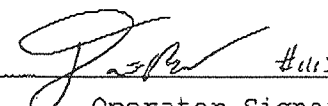
Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?          1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            16512080A2  
Standard Cyl #?            64  
Standard Expiration?       08/01/2014  
Oper No?                    120701

Flow Cal. Date:            11/01/2013  
Slope                        628  
Intercept                   -526243

IR Calibration Date:       04/01/2014  
                                    3um                      9um

-----  
0th Coef(\*100):            -43472                      -36988  
1st Coef(\*100):            280525                      139494  
2nd Coef(\*100):            2586                        750  
H2O adj(mg/l\*10k):        846                         658

\*\*\*\*\* Printer Test End \*\*\*\*\*

  
-----  
Operator Signature  
DANIEL BLOOD

Remarks: *Print Test*

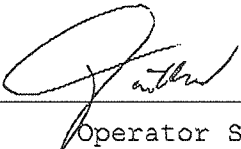
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005946  
Location = WCPD      8164.13.00 06/09  
05/20/2014      19:27

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	19:28
02 Std. Gas	0.077	19:28
03 Room Air	0.000	19:29
04 Std. Gas	0.076	19:29
05 Room Air	0.000	19:30
06 Std. Gas	0.076	19:30
07 Room Air	0.000	19:30

Lot No = 16512080A2  
Cyl No = 64  
Exp Date = 08/01/2014  
County = 27      Oper No. = 120701

 #1113  
Operator Signature  
DANIEL BLOOD

Remarks: ACA Test

Intoxilyzer Test Record and Checklist  
 NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
 North Dakota Model 8000      SN 80-005946  
 Location = WCPD      8164.13.00 06/09  
 05/20/2014      19:31

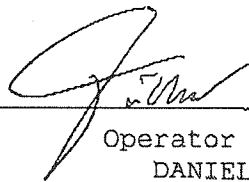
Test	AC	Time
01 Diagnostic	OK	19:32
02 Room Air	RFI*	19:32
03 Room Air	0.000	19:33

\*Invalid Test  
 Inhibited - RFI

---

Sub Name = TEST, TEST TEST  
 Sub DOB = 05/19/2014  
 Sub Sex = Unknown      Weight = 100  
 Test = DUI      Cit = NA  
 Dr. Lic. = ND/NA  
 Lot No = 16512080A2  
 Cyl No = 64  
 Expiration Date = 08/01/2014  
 County = 27      Oper No. = 120701

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

  
 Operator Signature  
 DANIEL BLOOD

Remarks: RFI Test (Cms mode)