

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-004203

Location: ^{DK} Heart of Mt America Conv Ctr

- Check When Done
- A. Is the breath tube heated? ✓
 - B. Is the diagnostic check complete? ✓
 - C. Is the time, date, and year correct (re-set if necessary)? ✓
 - D. Print test. (Attach test record.) ✓
 - E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN DR5143 Lot No. 201501B Rep AC 0.020 ✓
 - F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN DR3455 Lot No. 14270 Rep AC 0.300 ✓
 - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN DR5132 Lot No. ICS 2 Solution 0.10 AC + 0.05% acetone ✓
 - H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) ✓
 - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) 0.080 AC

Lot No. <u>14214080A6</u>	Cyl No. <u>56</u>	Exp. Date <u>07/05/16</u>
Test 1 <u>0.084</u>	Test 4 <u>0.083</u>	Test 7 <u>0.083</u>
Test 2 <u>0.083</u>	Test 5 <u>0.083</u>	Test 8 <u>0.083</u>
Test 3 <u>0.082</u>	Test 6 <u>0.083</u>	Test 9 <u>0.084</u>
Average <u>0.083</u>		

J. Tank pressure, Level 3 Func D Sub G P: 1012 psi Reg 1000 psi

K. Remarks/Maintenance record: annual inspection, replaced o-rings on
breath hose & simulator return

18 May 15
Date

20 MAY 15 ✓
Date

Deb Kashur
Field Inspector's Signature

[Signature]
Reviewed by

Intoxilyzer Test Record and Checklist
 NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
 North Dakota Model 8000 SN 80-004203
 Location = RUGB 8164.13.00 06/09
 05/18/2015 17:36

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=
 ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=
 ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
 Start Test Sequence: DABACABA
 Display Prelim Rslt? Yes
 Display Third Digit? Yes
 Inhib Printer(Y/N)? No
 Display Volume? No
 Disable On Memfull? Yes
 # of Print Copies? 1
 Select Std (D/W/I)? Dry
 Standard Value? 0.080
 Standard Lot #? 14214080A6
 Standard Cyl #? 56
 Standard Expiration? 07/05/2016
 Oper No? 777777

Flow Cal. Date: 01/28/2010
 Slope 719
 Intercept -711903

IR Calibration Date: 01/28/2010
 3um 9um


 0th Coef(*100): -17469 -28710
 1st Coef(*100): 264216 135022
 2nd Coef(*100): 2778 1081
 H2O adj(mg/l*10k): 430 605

***** Printer Test End *****



Operator Signature
 DEB KASHUR

Remarks:



Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004203
Location = RUGB 8164.13.00 06/09
05/18/2015 17:44

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	17:45
02 Std. Sol.	0.016	17:46
03 Room Air	0.000	17:46
04 Std. Sol.	0.017	17:47
05 Room Air	0.000	17:48
06 Std. Sol.	0.017	17:48
07 Room Air	0.000	17:49

08 Sim Temp = 34.0°C

Simul Ser No = DR5143

Std Sol No = 201501B

County = 35

Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Low AC 0.020AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004203
Location = RUGB 8164.13.00 06/09
05/18/2015 18:02

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	18:03
02 Std. Sol.	0.288	18:04
03 Room Air	0.000	18:04
04 Std. Sol.	0.289	18:05
05 Room Air	0.000	18:05
06 Std. Sol.	0.289	18:06
07 Room Air	0.000	18:07

08 Sim Temp = 34.0°C

Simul Ser No = DR3455

Std Sol No = 14270

County = 35

Oper No. = 777777

Deb Keshur

Operator Signature
DEB KASHUR

Remarks:

High AC ^{DK} ~~0.6~~ 0.300 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004203
Location = RUGB 8164.13.00 06/09
05/18/2015 18:08

Test	AC	Time
01 Room Air	0.000	18:09
02 *Subject Test	INT*	18:09
03 Room Air	0.000	18:10

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 14214080A6
Cyl No = 56
Expiration Date = 07/05/2016
County = 35 Oper No. = 777777



Operator Signature
DEB KASHUR

Remarks:

Interferent check 0.10 AC + 0.05^g acetone

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004203
Location = RUGB 8164.13.00 06/09
05/18/2015 18:10

Test	AC	Time
01 Diagnostic	OK	18:11
02 Room Air	RFI*	18:11
03 Room Air	0.000	18:11

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 14214080A6
Cyl No = 56
Expiration Date = 07/05/2016
County = 35 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

RFI check

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004203
Location = RUGB 8164.13.00 06/09
05/18/2015 17:37

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	17:37
02 Std. Gas	0.084	17:37
03 Room Air	0.000	17:38
04 Std. Gas	0.083	17:38
05 Room Air	0.000	17:39
06 Std. Gas	0.082	17:39
07 Room Air	0.000	17:40

Lot No = 14214080A6
Cyl No = 56
Exp Date = 07/05/2016
County = 35 Oper No. = 777777

Deb Kasher
Operator Signature
DEB KASHUR

Remarks:

Calibration check. 0.080 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004203
Location = RUGB 8164.13.00 06/09
05/18/2015 17:40

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	17:41
02 Std. Gas	0.083	17:41
03 Room Air	0.000	17:42
04 Std. Gas	0.083	17:42
05 Room Air	0.000	17:43
06 Std. Gas	0.083	17:43
07 Room Air	0.000	17:44

Lot No = 14214080A6

Cyl No = 56

Exp Date = 07/05/2016

County = 35

Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004203
Location = RUGB 8164.13.00 06/09
05/18/2015 18:12

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	18:12
02 Std. Gas	0.083	18:13
03 Room Air	0.000	18:13
04 Std. Gas	0.083	18:13
05 Room Air	0.000	18:14
06 Std. Gas	0.084	18:14
07 Room Air	0.000	18:15

Lot No = 14214080A6
Cyl No = 56
Exp Date = 07/05/2016
County = 35 Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080AC

Form 106-I8000