

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-005941

Location: BIA - Ft. Totten

- Check When Done
- A. Is the breath tube heated? ✓
 - B. Is the diagnostic check complete? ✓
 - C. Is the time, date, and year correct (re-set if necessary)? ✓
 - D. Print test. (Attach test record.) ✓
 - E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN DR5143 Lot No. 201561B Rep AC 0.020 ✓
 - F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN DR3455 Lot No. 14270 Rep AC 0.300 ✓
 - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN DR5132 Lot No. 1CS2 Solution 0.10 AC + 0.05% acetone ✓
 - H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) ✓

I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) 0.080 AC

Lot No. OK 14214080 A16 Cyl No. 71 Exp. Date 07/05/16

Test 1 <u>0.082</u>	Test 4 <u>0.081</u>	Test 7 <u>0.082</u>
Test 2 <u>0.082</u>	Test 5 <u>0.081</u>	Test 8 <u>0.081</u>
Test 3 <u>0.082</u>	Test 6 <u>0.082</u>	Test 9 <u>0.081</u>

Average 0.082

J. Tank pressure, Level 3 Func D Sub G P: 1018 psi Reg 1000 psi

K. Remarks/Maintenance record: Annual inspection

18 May 15
Date

20 MAY 15 ✓
Date

Deb Kashner
Field Inspector's Signature

Charles E. Edwards
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005941
Location = FTOT 8164.13.00 06/09
05/18/2015 14:27

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-=_|
ABCDEFGHIJKLMNopQRSTUVWXYZ!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxy1234567890-=_|
ABCDEFGHIJKLMNopQRSTUVWXYZ!@#\$\$%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 14214080A6
Standard Cyl #? 71
Standard Expiration? 07/05/2016
Oper No? 777777

Flow Cal. Date: 06/18/2013
Slope 695
Intercept -675557

IR Calibration Date: 06/18/2013
 3um 9um

0th Coef(*100): -20306 -25137
1st Coef(*100): 291687 138772
2nd Coef(*100): 2450 1058
H2O adj(mg/l*10k): 562 452

***** Printer Test End *****



Operator Signature
DEB KASHUR

Remarks:

Print test

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005941
Location = FTOT 8164.13.00 06/09
05/18/2015 14:47

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:48
02 Std. Sol.	0.017	14:49
03 Room Air	0.000	14:49
04 Std. Sol.	0.018	14:50
05 Room Air	0.000	14:50
06 Std. Sol.	0.017	14:51
07 Room Air	0.000	14:52

08 Sim Temp = 34.0°C

Simul Ser No = DR5143

Std Sol No = 201501B

County = 62

Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Low AC 0.020AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005941
Location = FTOT 8164.13.00 06/09
05/18/2015 14:53

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:54
02 Std. Sol.	0.305	14:55
03 Room Air	0.000	14:55
04 Std. Sol.	0.303	14:56
05 Room Air	0.000	14:56
06 Std. Sol.	0.303	14:57
07 Room Air	0.000	14:58

08 Sim Temp = 34.0°C

Simul Ser No = DR3455

Std Sol No = 14270

County = 62

Oper No. = 777777

Deb Keshur

Operator Signature
DEB KASHUR

Remarks:

High AC 0.300AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005941
Location = FTOT 8164.13.00 06/09
05/18/2015 14:44

Test	AC	Time
01 Room Air	0.000	14:45
02 *Subject Test	INT*	14:45
03 Room Air	0.000	14:46

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 14214080A6
Cyl No = 71
Expiration Date = 07/05/2016
County = 62 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

*Interferent check 0.10 AC
+ 0.05% acetone*

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005941
Location = FTOT 8164.13.00 06/09
05/18/2015 14:39

Test	AC	Time
01 Diagnostic	OK	14:41
02 Room Air	RFI*	14:41
03 Room Air	0.000	14:42

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 14214080A6
Cyl No = 71
Expiration Date = 07/05/2016
County = 62 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Keshur

Operator Signature
DEB KASHUR

Remarks:

RFI check

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005941
Location = FTOT 8164.13.00 06/09
05/18/2015 14:28

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:29
02 Std. Gas	0.082	14:29
03 Room Air	0.000	14:30
04 Std. Gas	0.082	14:30
05 Room Air	0.000	14:30
06 Std. Gas	0.082	14:31
07 Room Air	0.000	14:31

Lot No = 14214080A6
Cyl No = 71
Exp Date = 07/05/2016
County = 62 Oper No. = 777777

Deb Kasher
Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005941
Location = FTOT 8164.13.00 06/09
05/18/2015 14:31

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:32
02 Std. Gas	0.081	14:32
03 Room Air	0.000	14:33
04 Std. Gas	0.081	14:33
05 Room Air	0.000	14:34
06 Std. Gas	0.082	14:34
07 Room Air	0.000	14:35

Lot No = 14214080A6

Cyl No = 71

Exp Date = 07/05/2016

County = 62

Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005941
Location = FTOT 8164.13.00 06/09
05/18/2015 14:35

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:36
02 Std. Gas	0.082	14:36
03 Room Air	0.000	14:37
04 Std. Gas	0.081	14:37
05 Room Air	0.000	14:38
06 Std. Gas	0.081	14:38
07 Room Air	0.000	14:38

Lot No = 14214080A6

Cyl No = 71

Exp Date = 07/05/2016

County = 62

Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000