



**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**  
 OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION  
 Toxicology Section/Breath Alcohol Program  
 SFN 59281 (03-2014)

Serial Number <i>90-006499</i>	Instrument Location <i>STURMAN VEC - CORRECTIONS</i>
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): *n/a*
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>[Signature]</i>	Date <i>05-20-16</i> ✓
Reviewed By (Crime Laboratory Use Only) <i>Deb Shanaver</i>	Date <i>24 May 16</i>

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006499  
Location = JAME                8164.13.00 06/09  
05/20/2016                      19:29

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^\*()\_+?

Current Instrument Setup

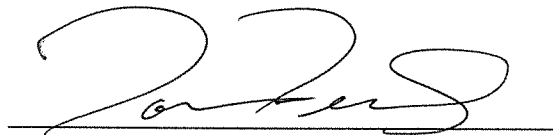
Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?         1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            19615080A5  
Standard Cyl #?            34  
Standard Expiration?       09/05/2017  
Oper No?                    039706

Flow Cal. Date:            08/18/2015  
Slope                      678  
Intercept                  -685002

IR Calibration Date:        08/18/2015

	3um	9um
0th Coef(*100):	-22123	-16014
1st Coef(*100):	264933	138505
2nd Coef(*100):	3281	1621
H2O adj(mg/l*10k):	704	456

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
THOMAS HERZIG

Remarks:

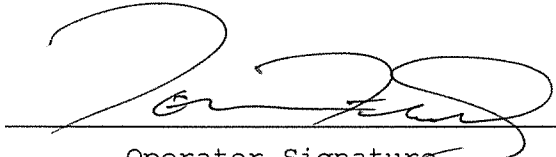
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006499  
Location = JAME                 8164.13.00 06/09  
05/20/2016                        19:29

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	19:29
02 Std. Gas	0.079	19:30
03 Room Air	0.000	19:30
04 Std. Gas	0.079	19:31
05 Room Air	0.000	19:31
06 Std. Gas	0.080	19:32
07 Room Air	0.000	19:32

Lot No = 19615080A5  
Cyl No = 34  
Exp Date = 09/05/2017  
County = 47                        Oper No. = 039706

  
Operator Signature  
THOMAS HERZIG

Remarks:

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

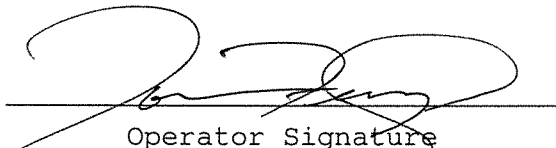
CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-006499  
Location = JAME                    8164.13.00 06/09  
05/20/2016                            19:33

DRY CAL CHECK

Test	AC	Time
01 Room Air	RFI*	19:33
02 Room Air	0.000	19:34

\*Invalid Test  
Inhibited - RFI

Lot No = 19615080A5  
Cyl No = 34  
Exp Date = 09/05/2017  
County = 47                            Oper No. = 039706

  
Operator Signature  
THOMAS HERZIG

Remarks: