

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-003061

Location: TOXL

- Check When Done
- A. Is the breath tube heated? ✓
  - B. Is the diagnostic check complete? ✓
  - C. Is the time, date, and year correct (re-set if necessary)? ✓
  - D. Print test. (Attach test record.) ✓
  - E. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)  
Sim SN DR3377 Lot No. 13270 Rep AC 0.020 ✓
  - F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)  
Sim SN DR5190 Lot No. 12105 Rep AC 0.400 AC ✓
  - G. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.  
(Attach test record.)  
Sim SN DR5135 Lot No. ICS#2 Rep AC 0.100 AC + 0.05% ACETONE ✓
  - H. RFI check. Run CMS mode. Key radio on first room air.  
(Attach test record.) ✓
  - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.  
(Attach test record.)

Lot No. 16512080A2 Cyl No. 019 Exp. Date 08/01/2014

Test 1 <u>0.081</u>	Test 4 <u>0.081</u>	Test 7 <u>0.080</u>
Test 2 <u>0.081</u>	Test 5 <u>0.080</u>	Test 8 <u>0.080</u>
Test 3 <u>0.080</u>	Test 6 <u>0.081</u>	Test 9 <u>0.080</u>

Average 0.080

J. Tank pressure, Level 3 Func D Sub G P: 756 psi Reg 750 psi

K. Remarks/Maintenance record: REPLACED SIMULATOR RETURN O-RING,

ANNUAL INSPECTION

22 MAY 14  
Date

22 May 14  
Date

Charles E. Egan  
Field Inspector's Signature

Deb Kershner  
Reviewed by

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003061  
Location = TOXL      8164.13.00 06/09  
05/22/2014      10:35

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890-=  
ABCDEFGHIJKLMN0PQRSTUVWXYZ!@#%&^\*()\_+?

abcdefghijklmnopqrstuvwxy1234567890-=  
ABCDEFGHIJKLMN0PQRSTUVWXYZ!@#%&^\*()\_+?

Current Instrument Setup


Data Entry Mode:      Enabled  
Start Test Sequence:      DABACABA  
Display Prelim Rslt?      Yes  
Display Third Digit?      Yes  
Inhib Printer(Y/N)?      No  
Display Volume?      No  
Disable On Memfull?      Yes  
# of Print Copies?      1  
Select Std (D/W/I)?      Dry  
Standard Value?      0.080  
Standard Lot #?      16512080A2  
Standard Cyl #?      19  
Standard Expiration?      08/01/2014  
Oper No?      666666

Flow Cal. Date:      03/04/2011  
Slope      700  
Intercept      -623726

IR Calibration Date:      04/26/2011

	3um	9um
0th Coef(*100):	-10139	-4793
1st Coef(*100):	265984	136608
2nd Coef(*100):	2678	1277
H2O adj(mg/l*10k):	524	433

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
CHARLES EDER

Remarks:      PRINT TEST

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003061  
Location = TOXL      8164.13.00 06/09  
05/22/2014      10:38

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:39
02 Std. Sol.	0.019	10:40
03 Room Air	0.000	10:40
04 Std. Sol.	0.020	10:41
05 Room Air	0.000	10:42
06 Std. Sol.	0.019	10:42
07 Room Air	0.000	10:43


08 Sim Temp = 34.0°C

Simul Ser No = DR3377

Std Sol No = 13270

County = 08

Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

Low AC  
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003061  
Location = TOXL      8164.13.00 06/09  
05/22/2014      10:46

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:46
02 Std. Sol.	0.404	10:47
03 Room Air	0.000	10:48
04 Std. Sol.	0.405	10:49
05 Room Air	0.000	10:49
06 Std. Sol.	0.405	10:50
07 Room Air	0.000	10:50


08 Sim Temp = 34.0°C

Simul Ser No = DR5190

Std Sol No = 12105

County = 08

Oper No. = 666666

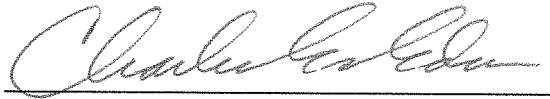

Operator Signature  
CHARLES EDER

Remarks:

HIGH AC  
0.400 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003061  
Location = TOXL      8164.13.00 06/09  
05/22/2014      10:52

Test	AC	Time
01 Room Air	0.000	10:53
02 *Subject Test	INT*	10:54
03 Room Air	0.000	10:55

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = INTERFERENT  
Dr. Lic. = ND/DIS821456  
Lot No = 16512080A2  
Cyl No = 19  
Expiration Date = 08/01/2014  
County = 08      Oper No. = 666666

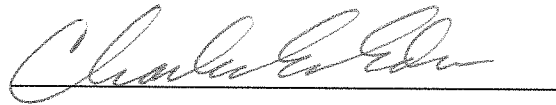

Operator Signature  
CHARLES EDER

Remarks: *INTERFERENT CHECK*  
*0.100 AC + 0.05% ACETONE*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-003061  
Location = TOXL                    8164.13.00 06/09  
05/22/2014                            10:56

Test	AC	Time
01 Diagnostic	OK	10:57
02 Room Air	RFI*	10:57
03 Room Air	0.000	10:58

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = RFI CHECK  
Dr. Lic. = ND/DIS821456  
Lot No = 16512080A2  
Cyl No = 19  
Expiration Date = 08/01/2014  
County = 08                            Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
CHARLES EDER

Remarks: *RFI CHECK*


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003061  
Location = TOXL      8164.13.00 06/09  
05/22/2014      10:59

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:00
02 Std. Gas	0.081	11:00
03 Room Air	0.000	11:00
04 Std. Gas	0.081	11:01
05 Room Air	0.000	11:01
06 Std. Gas	0.080	11:02
07 Room Air	0.000	11:02

Lot No = 16512080A2  
Cyl No = 19  
Exp Date = 08/01/2014  
County = 08      Oper No. = 666666

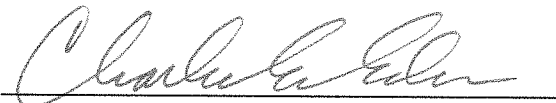

Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003061  
Location = TOXL      8164.13.00 06/09  
05/22/2014      11:04

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:05
02 Std. Gas	0.081	11:05
03 Room Air	0.000	11:06
04 Std. Gas	0.080	11:06
05 Room Air	0.000	11:07
06 Std. Gas	0.081	11:07
07 Room Air	0.000	11:08

Lot No = 16512080A2  
Cyl No = 19  
Exp Date = 08/01/2014  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003061  
Location = TOXL      8164.13.00 06/09  
05/22/2014      11:08

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:09
02 Std. Gas	0.080	11:09
03 Room Air	0.000	11:10
04 Std. Gas	0.080	11:10
05 Room Air	0.000	11:11
06 Std. Gas	0.080	11:11
07 Room Air	0.000	11:12

Lot No = 16512080A2  
Cyl No = 19  
Exp Date = 08/01/2014  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-I8000