

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-004189

Location: Bottineau Co SO-BOTT

- Check When Done
- A. Is the breath tube heated? ✓
  - B. Is the diagnostic check complete? ✓
  - C. Is the time, date, and year correct (re-set if necessary)? ✓
  - D. Print test. (Attach test record.) ✓
  - E. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)  
Sim SN DR5115 Lot No. 12111 Rep AC 0.020 ✓
  - F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)  
Sim SN DR5143 Lot No. 13080 Rep AC 0.300 ✓
  - G. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.  
(Attach test record.)  
Sim SN DR3375 Lot No. 1CS#2 Rep AC 0.10 AC + 0.05% acetone ✓
  - H. RFI check. Run CMS mode. Key radio on first room air.  
(Attach test record.) ✓
  - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.  
(Attach test record.)

Lot No. 16512080A2 Cyl No. 6 Exp. Date 08/01/2014

Test 1 <u>0.079</u>	Test 4 <u>0.079</u>	Test 7 <u>0.079</u>
Test 2 <u>0.079</u>	Test 5 <u>0.078</u>	Test 8 <u>0.079</u>
Test 3 <u>0.080</u>	Test 6 <u>0.078</u>	Test 9 <u>0.079</u>

Average 0.079

J. Tank pressure, Level 3 Func D Sub G P: 773 psi Reg 780 psi

K. Remarks/Maintenance record: Annual inspection, replaced simulator return & breath hose o-rings

20 May 2014  
Date

22 MAY 2014 ✓  
Date

Deb Kashur  
Field Inspector's Signature

Charles E. Elin  
Reviewed by

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004189  
Location = BOTT                 8164.13.00 06/09  
05/20/2014                        13:00

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNPNOPQRSTUVWXYZ!@#%&^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNPNOPQRSTUVWXYZ!@#%&^&\*()\_+?

Current Instrument Setup

Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?          1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            16512080A2  
Standard Cyl #?            6  
Standard Expiration?       08/01/2014  
Oper No?                    777777

Flow Cal. Date:            02/02/2011  
Slope                      628  
Intercept                  -452608

IR Calibration Date:       02/02/2011  
                                    3um                      9um

-----  
0th Coef(\*100):        -22075                      -26801  
1st Coef(\*100):        257433                      132480  
2nd Coef(\*100):        2132                        1061  
H2O adj(mg/l\*10k):    437                         430

\*\*\*\*\* Printer Test End \*\*\*\*\*

*Deb Kasher*

Operator Signature  
DEB KASHUR

Remarks:

*Print test*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004189  
Location = BOTT      8164.13.00 06/09  
05/20/2014      13:01

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:02
02 Std. Sol.	0.016	13:02
03 Room Air	0.000	13:03
04 Std. Sol.	0.016	13:04
05 Room Air	0.000	13:04
06 Std. Sol.	0.017	13:05
07 Room Air	0.000	13:05

08 Sim Temp = 34.0°C

Simul Ser No = DR5115

Std Sol No = 12111

County = 05

Oper No. = 777777

Deb Kasher

Operator Signature  
DEB KASHUR

Remarks:

Low AC 0.020AE

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004189  
Location = BOTT                    8164.13.00 06/09  
05/20/2014                            13:06

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:07
02 Std. Sol.	0.292	13:08
03 Room Air	0.000	13:08
04 Std. Sol.	0.296	13:09
05 Room Air	0.000	13:10
06 Std. Sol.	0.298	13:10
07 Room Air	0.000	13:11

08 Sim Temp = 34.0°C

Simul Ser No = DR5143  
Std Sol No = 13080  
County = 05                            Oper No. = 777777

  
\_\_\_\_\_  
Operator Signature  
DEB KASHUR

Remarks:

*High AC    0.300 AC*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004189  
Location = BOTT                    8164.13.00 06/09  
05/20/2014                            13:14

Test	AC	Time
01 Room Air	0.000	13:15
02 *Subject Test	INT*	13:15
03 Room Air	0.000	13:16

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 16512080A2  
Cyl No = 06  
Expiration Date = 08/01/2014  
County = 05                            Oper No. = 777777

*Deb Kasher*

Operator Signature  
DEB KASHUR

Remarks:

*Interferent check 0.100AC + 0.05% acetone*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-004189  
Location = BOTT                    8164.13.00 06/09  
05/20/2014                            13:23

Test	AC	Time
01 Diagnostic	OK	13:24
02 Room Air	RFI*	13:24
03 Room Air	0.000	13:24

\*Invalid Test  
  Inhibited - RFI

---

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 16512080A2  
Cyl No = 06  
Expiration Date = 08/01/2014  
County = 05                            Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
DEB KASHUR

Remarks:

RFI check

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004189  
Location = BOTT      8164.13.00 06/09  
05/20/2014      12:49

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:50
02 Std. Gas	0.079	12:50
03 Room Air	0.000	12:51
04 Std. Gas	0.079	12:51
05 Room Air	0.000	12:52
06 Std. Gas	0.080	12:52
07 Room Air	0.000	12:53

Lot No = 16512080A2  
Cyl No = 6  
Exp Date = 08/01/2014  
County = 05      Oper No. = 777777

Deb Kasher  
Operator Signature  
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004189  
Location = BOTT                    8164.13.00 06/09  
05/20/2014                            12:53

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:54
02 Std. Gas	0.079	12:54
03 Room Air	0.000	12:54
04 Std. Gas	0.078	12:55
05 Room Air	0.000	12:55
06 Std. Gas	0.078	12:56
07 Room Air	0.000	12:56

Lot No = 16512080A2  
Cyl No = 6  
Exp Date = 08/01/2014  
County = 05                            Oper No. = 777777

Deb Kasher  
Operator Signature  
DEB KASHUR

Remarks:

Calibration check 0.080AE



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004189  
Location = BOTT      8164.13.00 06/09  
05/20/2014      12:56

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:57
02 Std. Gas	0.079	12:57
03 Room Air	0.000	12:58
04 Std. Gas	0.079	12:58
05 Room Air	0.000	12:59
06 Std. Gas	0.079	12:59
07 Room Air	0.000	13:00

Lot No = 16512080A2  
Cyl No = 6  
Exp Date = 08/01/2014  
County = 05      Oper No. = 777777

Deb Kashur  
Operator Signature  
DEB KASHUR

Remarks:

*Calibration check 0.080 AC*