

INTOXILYZER® 8000 INSPECTION

Serial No.: 811-004940

Location: BURK - Burke Co SO

- Check When Done
- A. Is the breath tube heated? ✓
- B. Is the diagnostic check complete? ✓
- C. Is the time, date, and year correct (re-set if necessary)? ✓
- D. Print test. (Attach test record.) ✓
- E. Low AC. Use  $< 0.03$  AC in ACA mode. (Attach test record.)  
Sim SN DR5115 Lot No. 0-DK Rep AC 0.020  
12111 ✓
- F. High AC. Use  $\geq 0.25$  AC in ACA mode. (Attach test record.)  
Sim SN DR5143 Lot No. 13080 Rep AC 0.300 ✓
- G. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.  
(Attach test record.) ✓  
Sim SN DR3375 Lot No. 1CS#2 Rep AC 0.10 Ae + 0.05% acetone
- H. RFI check. Run CMS mode. Key radio on first room air.  
(Attach test record.) ✓
- I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.  
(Attach test record.)

Lot No. 16512080A2 Cyl No. 57 Exp. Date 08/01/2014

Test 1 0.078 Test 4 0.078 Test 7 0.078

Test 2 0.078 Test 5 0.078 Test 8 0.078

Test 3 0.078 Test 6 0.078 Test 9 0.078

Average 0.078

J. Tank pressure, Level 3 Func D Sub G - P: 630 psi Reg 630 psi

K. Remarks/Maintenance record: Annual inspection, replaced breath  
nose O-ring.

20 May 14  
Date

22 MAY 14 ✓  
Date

Deb Keshner  
Field Inspector's Signature

Charles E. Egan  
Reviewed by

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-004940  
Location = BURK                    8164.13.00 06/09  
05/20/2014                            16:19

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNopQRSTUVWXYZ!@#\$\$%^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNopQRSTUVWXYZ!@#\$\$%^&\*()\_+?

Current Instrument Setup

Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?          1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            16512080A2  
Standard Cyl #?            57  
Standard Expiration?        08/01/2014  
Oper No?                    777777

Flow Cal. Date:            01/18/2010  
Slope                        720  
Intercept                   -746294

IR Calibration Date:        06/22/2010  
                                  3um                            9um

-----  
0th Coef(\*100):            -10417                        -10800  
1st Coef(\*100):            259947                        132173  
2nd Coef(\*100):            2872                            1619  
H2O adj(mg/l\*10k):        604                            506

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
DEB KASHUR

Remarks:



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004940  
Location = BURK      8164.13.00 06/09  
05/20/2014      16:33

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:34
02 Std. Sol.	0.021	16:35
03 Room Air	0.000	16:35
04 Std. Sol.	0.021	16:36
05 Room Air	0.000	16:37
06 Std. Sol.	0.021	16:37
07 Room Air	0.000	16:38

08 Sim Temp = 34.0°C

Simul Ser No = DR5115  
Std Sol No = 12111  
County = 07      Oper No. = 777777

Deb Kasher  
Operator Signature  
DEB KASHUR

Remarks:

LOW AC      0.020 AC

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004940  
Location = BURK      8164.13.00 06/09  
05/20/2014      16:39

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:39
02 Std. Sol.	0.305	16:40
03 Room Air	0.000	16:41
04 Std. Sol.	0.306	16:42
05 Room Air	0.000	16:42
06 Std. Sol.	0.307	16:43
07 Room Air	0.000	16:43

08 Sim Temp = 34.0°C

Simul Ser No = DR5143

Std Sol No = 13080

County = 07

Oper No. = 777777

Deb Kashur

Operator Signature

DEB KASHUR

Remarks:

High AC 0.300AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004940  
Location = BURK                    8164.13.00 06/09  
05/20/2014                            16:19

Test	AC	Time
01 Room Air	0.000	16:20
02 *Subject Test	INT*	16:20
03 Room Air	0.000	16:21

\*Invalid Test  
Interferent Detected

---

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 16512080A2  
Cyl No = 57  
Expiration Date = 08/01/2014  
County = 07                            Oper No. = 777777

*Deb Kasher*

Operator Signature  
DEB KASHUR

Remarks:

*Interferent check 0.100AC + 0.05% acetone*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-004940  
Location = BURK                    8164.13.00 06/09  
05/20/2014                            16:21

Test	AC	Time
01 Diagnostic	OK	16:22
02 Room Air	RFI*	16:22
03 Room Air	0.000	16:23

\*Invalid Test  
Inhibited - RFI

---

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 16512080A2  
Cyl No = 57  
Expiration Date = 08/01/2014  
County = 07                            Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Kasher

Operator Signature  
DEB KASHUR

Remarks:

*RFI check*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004940  
Location = BURK      8164.13.00 06/09  
05/20/2014      16:23

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:24
02 Std. Gas	0.078	16:24
03 Room Air	0.000	16:24
04 Std. Gas	0.078	16:25
05 Room Air	0.000	16:25
06 Std. Gas	0.078	16:26
07 Room Air	0.000	16:26

Lot No = 16512080A2  
Cyl No = 57  
Exp Date = 08/01/2014  
County = 07      Oper No. = 777777

Deb Kasher  
Operator Signature  
DEB KASHUR

Remarks:

*Calibration check 0.080 AC*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004940  
Location = BURK      8164.13.00 06/09  
05/20/2014      16:26

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:27
02 Std. Gas	0.078	16:27
03 Room Air	0.000	16:28
04 Std. Gas	0.078	16:28
05 Room Air	0.000	16:29
06 Std. Gas	0.078	16:29
07 Room Air	0.000	16:30

Lot No = 16512080A2  
Cyl No = 57  
Exp Date = 08/01/2014  
County = 07      Oper No. = 777777

Deb Kashur  
Operator Signature  
DEB KASHUR

Remarks:

calibration check 0.080 AC



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004940  
Location = BURK                 8164.13.00 06/09  
05/20/2014                        16:30

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:30
02 Std. Gas	0.078	16:31
03 Room Air	0.000	16:31
04 Std. Gas	0.078	16:32
05 Room Air	0.000	16:32
06 Std. Gas	0.078	16:32
07 Room Air	0.000	16:33

Lot No = 16512080A2  
Cyl No = 57  
Exp Date = 08/01/2014  
County = 07                            Oper No. = 777777

Deb Kasher  
Operator Signature  
DEB KASHUR

Remarks:

*Calibration check 0.080 AC*