

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-004948

Location: Moundtrail Co SO - STAN

- Check When Done
- A. Is the breath tube heated? ✓
  - B. Is the diagnostic check complete? ✓
  - C. Is the time, date, and year correct (re-set if necessary)? ✓
  - D. Print test. (Attach test record.) ✓
  - E. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)  
Sim SN DR5115 Lot No. 12111 Rep AC 0.020 ✓
  - F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)  
Sim SN DR5143 Lot No. 13086 Rep AC 0.306 ✓
  - G. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.  
(Attach test record.)  
Sim SN DR3375 Lot No. 165#2 Rep AC 0.10 AC + 0.05% acetone ✓
  - H. RFI check. Run CMS mode. Key radio on first room air.  
(Attach test record.) ✓
  - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.  
(Attach test record.)

Lot No. 16512080A2 Cyl No. 70 Exp. Date 08/01/2014  
Test 1 0.079 Test 4 0.080 Test 7 0.079  
Test 2 0.079 Test 5 0.079 Test 8 0.079  
Test 3 0.079 Test 6 0.079 Test 9 0.078  
Average 0.079

J. Tank pressure, Level 3 Func D Sub G P: 901 psi Reg 900 psi

K. Remarks/Maintenance record: Annual inspection, replaced simulator  
return o-ring.

21 May 2014  
Date

22 May 2014 ✓  
Date

Deb Kashur  
Field Inspector's Signature

Charles E. Edm  
Reviewed by

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-004948  
Location = STAN                    8164.13.00 06/09  
05/21/2014                            12:44

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNopQRSTUVWXYZ!@#%&^\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNopQRSTUVWXYZ!@#%&^\*()\_+?

Current Instrument Setup

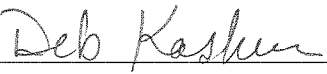
~~Data Entry Mode:~~            Enabled  
Start Test Sequence:            DABACABA  
Display Prelim Rslt?            Yes  
Display Third Digit?            Yes  
Inhib Printer(Y/N)?            No  
Display Volume?                No  
Disable On Memfull?            Yes  
# of Print Copies?              1  
Select Std (D/W/I)?            Dry  
Standard Value?                0.080  
Standard Lot #?                16512080A2  
Standard Cyl #?                70  
Standard Expiration?            08/01/2014  
Oper No?                        777777

Flow Cal. Date:                07/14/2010  
Slope                            778  
Intercept                       -795769

IR Calibration Date:            07/14/2010  
                                  3um                            9um

-----  
0th Coef(\*100):            -31061                    -21991  
1st Coef(\*100):            265097                   133596  
2nd Coef(\*100):            802                       1052  
H2O adj(mg/l\*10k):        675                       417

\*\*\*\*\* Printer Test End \*\*\*\*\*

  
\_\_\_\_\_  
Operator Signature  
DEB KASHUR

Remarks:

*Print test*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004948  
Location = STAN      8164.13.00 06/09  
05/21/2014      12:58

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:58
02 Std. Sol.	0.019	12:59
03 Room Air	0.000	12:59
04 Std. Sol.	0.019	13:00
05 Room Air	0.000	13:01
06 Std. Sol.	0.019	13:01
07 Room Air	0.000	13:02

08 Sim Temp = 34.0°C

Simul Ser No = DR5115

Std Sol No = 12111

County = 31

Oper No. = 777777

Deb Kashur

Operator Signature

DEB KASHUR

Remarks:

Low AC 0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004948  
Location = STAN      8164.13.00 06/09  
05/21/2014      13:03

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:04
02 Std. Sol.	0.302	13:05
03 Room Air	0.000	13:05
04 Std. Sol.	0.303	13:06
05 Room Air	0.000	13:06
06 Std. Sol.	0.305	13:07
07 Room Air	0.000	13:08

08 Sim Temp = 34.0°C

Simul Ser No = DR5143  
Std Sol No = 13080  
County = 31      Oper No. = 777777

Deb Kasher  
Operator Signature  
DEB KASHUR

Remarks:

High AC 0.300 AC

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004948  
Location = STAN      8164.13.00 06/09  
05/21/2014      13:08

Test	AC	Time
01 Room Air	0.000	13:09
02 *Subject Test	INT*	13:09
03 Room Air	0.000	13:10

\*Invalid Test  
Interferent Detected

---

Sub Name = ~~DISCOVER, THE SPIRIT~~  
Sub DOB = 02/01/1992  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 16512080A2  
Cyl No = 70  
Expiration Date = 08/01/2014  
County = 31      Oper No. = 777777

Deb Kashur

Operator Signature  
DEB KASHUR

Remarks:

*Interferent check 0.100AC + 0.05 acetone*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-004948  
Location = STAN                    8164.13.00 06/09  
05/21/2014                            13:10


Test	AC	Time
01 Diagnostic	OK	13:11
02 Room Air	RFI*	13:11
03 Room Air	0.000	13:12

\*Invalid Test  
Inhibited - RFI

---

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 16512080A2  
Cyl No = 70  
Expiration Date = 08/01/2014  
County = 31                            Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
DEB KASHUR

Remarks:

RFI check

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004948  
Location = STAN      8164.13.00 06/09  
05/21/2014      12:45

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:45
02 Std. Gas	0.079	12:46
03 Room Air	0.000	12:46
04 Std. Gas	0.079	12:47
05 Room Air	0.000	12:47
06 Std. Gas	0.079	12:47
07 Room Air	0.000	12:48

Lot No = 16512080A2  
Cyl No = 70  
Exp Date = 08/01/2014  
County = 31      Oper No. = 777777

Deb Kasher  
Operator Signature  
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004948  
Location = STAN      8164.13.00 06/09  
05/21/2014      12:48

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:49
02 Std. Gas	0.080	12:49
03 Room Air	0.000	12:50
04 Std. Gas	0.079	12:50
05 Room Air	0.000	12:50
06 Std. Gas	0.079	12:51
07 Room Air	0.000	12:51

Lot No = 16512080A2  
Cyl No = 70  
Exp Date = 08/01/2014  
County = 31      Oper No. = 77777

Deb Kasher  
Operator Signature  
DEB KASHUR

Remarks:

Calibration check 0.080AC



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004948  
Location = STAN      8164.13.00 06/09  
05/21/2014      12:52

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:52
02 Std. Gas	0.079	12:52
03 Room Air	0.000	12:53
04 Std. Gas	0.079	12:53
05 Room Air	0.000	12:54
06 Std. Gas	0.078	12:54
07 Room Air	0.000	12:55

Lot No = 16512080A2  
Cyl No = 70  
Exp Date = 08/01/2014  
County = 31      Oper No. = 777777

*Deb Kashur*

Operator Signature  
DEB KASHUR

Remarks:

*Calibration check 0.080AC*

Form 106-I8000