

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-004954 Location: ROLL (Rolle Co)

- | | Check When Done |
|--|-----------------|
| A. Is the breath tube heated? | <u>✓</u> |
| B. Is the diagnostic check complete? | <u>✓</u> |
| C. Is the time, date, and year correct (re-set if necessary)? | <u>✓</u> |
| D. Print test. (Attach test record.) | <u>✓</u> |
| E. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)
Sim SN <u>DR5115</u> Lot No. <u>1211</u> Rep AC <u>0.020</u> | <u>✓</u> |
| F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN <u>DR5143</u> Lot No. <u>13080</u> Rep AC <u>0.300</u> | <u>✓</u> |
| G. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.
(Attach test record.)
Sim SN <u>DR3375</u> Lot No. <u>JCS#2</u> Rep AC <u>0.10AE + 0.05% acetone</u> | <u>✓</u> |
| H. RFI check. Run CMS mode. Key radio on first room air.
(Attach test record.) | <u>✓</u> |
| I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) | |

Lot No. 165/2080A2 Cyl No. 50 Exp. Date 08/01/2014

Test 1 <u>0.081</u>	Test 4 <u>0.082</u>	Test 7 <u>0.081</u>
Test 2 <u>0.081</u>	Test 5 <u>0.082</u>	Test 8 <u>0.082</u>
Test 3 <u>0.081</u>	Test 6 <u>0.082</u>	Test 9 <u>0.082</u>

Average 0.082

J. Tank pressure, Level 3 Func D Sub G P: 313 psi Reg 300 psi

K. Remarks/Maintenance record: annual inspection, replaced simulator return & breath hose o-rings

20 May 2014
Date
22 MAY 2014 ✓
Date

Deb Kashur
Field Inspector's Signature
Charles Elder
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004954
Location = ROLL 8164.13.00 06/09
05/20/2014 11:26

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:26
02 Std. Sol.	0.018	11:27
03 Room Air	0.000	11:27
04 Std. Sol.	0.019	11:28
05 Room Air	0.000	11:29
06 Std. Sol.	0.020	11:29
07 Room Air	0.000	11:30

08 Sim Temp = 34.0°C

Simul Ser No = DR5115

Std Sol No = 12111

County = 40

Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Low AC 0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004954
Location = ROLL 8164.13.00 06/09
05/20/2014 11:32

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:33
02 Std. Sol.	0.288	11:33
03 Room Air	0.000	11:34
04 Std. Sol.	0.292	11:35
05 Room Air	0.000	11:35
06 Std. Sol.	0.294	11:36
07 Room Air	0.000	11:36

08 Sim Temp = 34.0°C

Simul Ser No = DR5143

Std Sol No = 13080

County = 40

Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

High AC 0.300 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004954
Location = ROLL 8164.13.00 06/09
05/20/2014 11:15

Test	AC	Time
01 Room Air	0.000	11:15
02 *Subject Test	INT*	11:16
03 Room Air	0.000	11:16

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 16512080A2
Cyl No = 050
Expiration Date = 08/01/2014
County = 40 Oper No. = 777777



Operator Signature
DEB KASHUR

Remarks:

Interferent check 0.100AC + 0.05% acetone

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004954
Location = ROLL 8164.13.00 06/09
05/20/2014 11:19

Test	AC	Time
01 Diagnostic	OK	11:20
02 Room Air	RFI*	11:20
03 Room Air	0.000	11:20

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 16512080A2
Cyl No = 050
Expiration Date = 08/01/2014
County = 40 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

RFI check

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004954
Location = ROLL 8164.13.00 06/09
05/20/2014 11:00

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:01
02 Std. Gas	0.081	11:01
03 Room Air	0.000	11:02
04 Std. Gas	0.081	11:02
05 Room Air	0.000	11:03
06 Std. Gas	0.081	11:03
07 Room Air	0.000	11:03

Lot No = 16512080A2
Cyl No = 050
Exp Date = 08/01/2014
County = 40 Oper No. = 777777

Deb Kasher
Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004954
Location = ROLL 8164.13.00 06/09
05/20/2014 11:04

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:05
02 Std. Gas	0.082	11:05
03 Room Air	0.000	11:06
04 Std. Gas	0.082	11:06
05 Room Air	0.000	11:06
06 Std. Gas	0.082	11:07
07 Room Air	0.000	11:07

Lot No = 16512080A2

Cyl No = 050

Exp Date = 08/01/2014

County = 40

Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004954
Location = ROLL 8164.13.00 06/09
05/20/2014 11:08

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:08
02 Std. Gas	0.081	11:09
03 Room Air	0.000	11:09
04 Std. Gas	0.082	11:10
05 Room Air	0.000	11:10
06 Std. Gas	0.082	11:11
07 Room Air	0.000	11:11

Lot No = 16512080A2
Cyl No = 050
Exp Date = 08/01/2014
County = 40 Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080AC

Form 106-I8000