



ETHANOL BREATH STANDARD CYLINDER REPORT

OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
Toxicology Section/Breath Alcohol Program
SFN 59282 (5-2011)

Chemical Test Operator Name (Print) <i>Steven R Fischer</i>		
Location <i>New Town Police Dept.</i>	Intoxilyzer® Serial Number <i>80-004953</i>	
Gas Lot Number <i>709352</i>	Gas Cylinder Number <i>36</i>	Gas Expiration Date <i>12-23-2011</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 - A. 0. 078 AC
 - B. 0. 079 AC
 - C. 0. 078 AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Breath Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature <i>Steven R Fischer</i>	Date <i>5-23-11</i>
Reviewed By (Crime Laboratory Use Only) <i>Deb Kashner</i>	Date <i>31 May 2011</i>

80-004953
P. 59 of 60
SFM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004953
Location = NEWT 8164.13.00 06/09
05/23/2011 20:12

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	20:13
02 Std. Gas	0.078	20:13
03 Room Air	0.000	20:14
04 Std. Gas	0.079	20:14
05 Room Air	0.000	20:15
06 Std. Gas	0.078	20:15
07 Room Air	0.000	20:16

Lot No = 709352
Cyl No = 36
Exp Date = 12/23/2011
County = 31 Oper No. = 049904


Operator Signature
STEVEN FISCHER

Remarks:

Form 106-I8000

80-004953
P. 20 of 60
JFM