

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-002666

Location: Harvey PD (HARV)

- | | Check When Done |
|--|-----------------|
| A. Is the breath tube heated? | <u>✓</u> |
| B. Is the diagnostic check complete? | <u>✓</u> |
| C. Is the time, date, and year correct (re-set if necessary)? | <u>✓</u> |
| D. Print test. (Attach test record.) | <u>✓</u> |
| E. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)
Sim SN <u>DR5115</u> Lot No. <u>12111</u> Rep AC <u>0.020</u> | <u>✓</u> |
| F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN <u>DR5143</u> Lot No. <u>13080</u> Rep AC <u>0.300</u> | <u>✓</u> |
| G. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.
(Attach test record.)
Sim SN <u>DR3375</u> Lot No. <u>ICS2</u> Rep AC <u>0.10 Ac + 0.05% acetone</u> | <u>✓</u> |
| H. RFI check. Run CMS mode. Key radio on first room air.
(Attach test record.) | <u>✓</u> |
| I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) | |

Lot No. <u>16512080 A2</u>	Cyl No. <u>32</u>	Exp. Date <u>08/01/2014</u>
Test 1 <u>0.079</u>	Test 4 <u>0.079</u>	Test 7 <u>0.080</u>
Test 2 <u>0.079</u>	Test 5 <u>0.080</u>	Test 8 <u>0.080</u>
Test 3 <u>0.080</u>	Test 6 <u>0.080</u>	Test 9 <u>0.080</u>
Average <u>0.080</u>		

J. Tank pressure, Level 3 Func D Sub G P: 968 psi Reg 970 psi

K. Remarks/Maintenance record: Annual inspection, recalibrated gas regulator.

05 May 2014
Date
23 MAY 14 ✓
Date

Deb Kashur
Field Inspector's Signature
Charles E. Edr
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002666
Location = HARV 8164.13.00 06/09
05/05/2014 11:02

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 16512080A2
Standard Cyl #? 32
Standard Expiration? 08/01/2014
Oper No? 777777

Flow Cal. Date: 12/31/2012
Slope 657
Intercept -418473

IR Calibration Date: 12/31/2012
 3um 9um

0th Coef(*100): -31967 -16114
1st Coef(*100): 279454 136158
2nd Coef(*100): 1837 1190
H2O adj(mg/l*10k): 527 293

***** Printer Test End *****

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Print Test

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002666
Location = HARV 8164.13.00 06/09
05/05/2014 11:23

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:23
02 Std. Sol.	0.019	11:24
03 Room Air	0.000	11:25
04 Std. Sol.	0.019	11:25
05 Room Air	0.000	11:26
06 Std. Sol.	0.020	11:27
07 Room Air	0.000	11:27

08 Sim Temp = 34.0°C

Simul Ser No = DR5115

Std Sol No = 12111

County = 52

Oper No. = 777777

Debs Kashur

Operator Signature
DEB KASHUR

Remarks:

low AC 0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002666
Location = HARV 8164.13.00 06/09
05/05/2014 11:28

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:29
02 Std. Sol.	0.300	11:29
03 Room Air	0.000	11:30
04 Std. Sol.	0.302	11:31
05 Room Air	0.000	11:31
06 Std. Sol.	0.302	11:32
07 Room Air	0.000	11:33

08 Sim Temp = 34.0°C

Simul Ser No = DR5143
Std Sol No = 13080
County = 52 Oper No. = 777777



Operator Signature
DEB KASHUR

Remarks:

High AC 0.300 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002666
Location = HARV 8164.13.00 06/09
05/05/2014 11:36

Test	AC	Time
01 Room Air	0.000	11:37
02 *Subject Test	INT*	11:37
03 Room Air	0.000	11:38

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 16512080A2
Cyl No = 32
Expiration Date = 08/01/2014
County = 52 Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Interferent check 0.10AC + 0.05% acetone

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002666
Location = HARV 8164.13.00 06/09
05/05/2014 11:07

Test	AC	Time
01 Room Air	RFI*	11:09
02 Room Air	0.000	11:09

*Invalid Test
 Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 16512080A2
Cyl No = 32
Expiration Date = 08/01/2014
County = 52 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

RFI check

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002666
Location = HARV 8164.13.00 06/09
05/05/2014 11:03

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:03
02 Std. Gas	0.079	11:04
03 Room Air	0.000	11:04
04 Std. Gas	0.079	11:05
05 Room Air	0.000	11:05
06 Std. Gas	0.080	11:05
07 Room Air	0.000	11:06

Lot No = 16512080A2
Cyl No = 32
Exp Date = 08/01/2014
County = 52 Oper No. = 777777

Deb Kasher
Operator Signature
DEB KASHUR

Remarks:
Calibration check 0.080

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002666
Location = HARV 8164.13.00 06/09
05/05/2014 11:14

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:15
02 Std. Gas	0.079	11:15
03 Room Air	0.000	11:15
04 Std. Gas	0.080	11:16
05 Room Air	0.000	11:16
06 Std. Gas	0.080	11:17
07 Room Air	0.000	11:17

Lot No = 16512080A2
Cyl No = 32
Exp Date = 08/01/2014
County = 52 Oper No. = 777777

Deb Kasher
Operator Signature
DEB KASHUR

Remarks:
Calibration check 0.080 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002666
Location = HARV 8164.13.00 06/09
05/05/2014 11:18

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:19
02 Std. Gas	0.080	11:19
03 Room Air	0.000	11:20
04 Std. Gas	0.080	11:20
05 Room Air	0.000	11:20
06 Std. Gas	0.080	11:21
07 Room Air	0.000	11:21

Lot No = 16512080A2
Cyl No = 32
Exp Date = 08/01/2014
County = 52 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000

06 May 2014

While inspecting Intoxilyzer 80-002666 during the Annual Inspection, I mistakenly ran the RFI (Radio Frequency Interferent) test in ABA mode (one breath test) instead of CMS mode (two breath tests), like the Intoxilyzer 8000 Inspection form suggests. Running the RFI test in either mode proves that the RFI is functioning properly; therefore, this annual inspection is complete and acceptable.



Deb Kashur
Forensic Scientist II
