

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-003065

Location: TOXL

- Check When Done
- A. Is the breath tube heated? ✓
 - B. Is the diagnostic check complete? ✓
 - C. Is the time, date, and year correct (re-set if necessary)? ✓
 - D. Print test. (Attach test record.) ✓
 - E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN DR3374 Lot No. 201501B Rep AC 0.020 ✓
 - F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN DR3841 Lot No. 201603A Rep AC 0.300 ✓
 - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN DR5144 Lot No. IC54 Solution 0.10AC + 0.05% acetone ✓
 - H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) ✓
 - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) 0.080 AC

Lot No. 19615080A5 Cyl No. 26 Exp. Date 09/05/2017

Test 1 <u>0.080</u>	Test 4 <u>0.081</u>	Test 7 <u>0.081</u>
Test 2 <u>0.080</u>	Test 5 <u>0.080</u>	Test 8 <u>0.080</u>
Test 3 <u>0.080</u>	Test 6 <u>0.080</u>	Test 9 <u>0.081</u>

Average 0.080

J. Tank pressure, Level 3 Func D Sub G P: 635 psi Reg 630 psi

K. Remarks/Maintenance record: Annual inspection, replaced o-rings on breath hose/simulator return, CE9 replaced external battery, cleaned
20 May 16 Deb Shanaver breath hose screen
Date Field Inspector's Signature
23 MAY 16 Charles E. E...
Date Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003065
Location = TOXL 8164.13.00 06/09
05/20/2016 14:51

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:52
02 Std. Sol.	0.019	14:52
03 Room Air	0.000	14:53
04 Std. Sol.	0.020	14:54
05 Room Air	0.000	14:54
06 Std. Sol.	0.020	14:55
07 Room Air	0.000	14:55

08 Sim Temp = 34.0°C

Simul Ser No = DR3374
Std Sol No = 210501B
County = 08 Oper No. = 777777



Operator Signature
DEB SHANAVER

Remarks:

low alcohol 0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003065
Location = TOXL 8164.13.00 06/09
05/20/2016 14:57

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:57
02 Std. Sol.	0.292	14:58
03 Room Air	0.000	14:59
04 Std. Sol.	0.293	14:59
05 Room Air	0.000	15:00
06 Std. Sol.	0.293	15:01
07 Room Air	0.000	15:01

08 Sim Temp = 34.0°C

Simul Ser No = DR3841
Std Sol No = 201603A
County = 08 Oper No. = 777777

Deb Shanaver
Operator Signature
DEB SHANAVER

Remarks:

High alcohol 0.300AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003065
Location = TOXL 8164.13.00 06/09
05/20/2016 15:11

Test	AC	Time
01 Room Air	0.000	15:11
02 *Subject Test	INT*	15:12
03 Room Air	0.000	15:13

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 19615080A5
Cyl No = 26
Expiration Date = 09/05/2017
County = 08 Oper No. = 777777

Deb Shanaver
Operator Signature
DEB SHANAVER

Remarks:

Interferent detected 0.10AC + 0.05% acetone

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003065
Location = TOXL 8164.13.00 06/09
05/20/2016 14:03

Test	AC	Time
01 Diagnostic	OK	14:04
02 Room Air	RFI*	14:04
03 Room Air	0.000	14:05

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 19615080A5
Cyl No = 26
Expiration Date = 09/05/2017
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

RFI test

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003065
Location = TOXL 8164.13.00 06/09
05/20/2016 13:41

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:41
02 Std. Gas	0.080	13:41
03 Room Air	0.000	13:42
04 Std. Gas	0.080	13:42
05 Room Air	0.000	13:43
06 Std. Gas	0.080	13:43
07 Room Air	0.000	13:44

Lot No = 19615080A5
Cyl No = 26
Exp Date = 09/05/2017
County = 08 Oper No. = 777777


Operator Signature
DEB SHANAVER

Remarks:

Calibration check 0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003065
Location = TOXL 8164.13.00 06/09
05/20/2016 13:45

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:46
02 Std. Gas	0.081	13:46
03 Room Air	0.000	13:46
04 Std. Gas	0.080	13:47
05 Room Air	0.000	13:47
06 Std. Gas	0.080	13:48
07 Room Air	0.000	13:48

Lot No = 19615080A5
Cyl No = 26
Exp Date = 09/05/2017
County = 08 Oper No. = 777777

Deb Shanaver
Operator Signature
DEB SHANAVER

Remarks:
Calibration check 0.080 AC


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003065
Location = TOXL 8164.13.00 06/09
05/20/2016 13:49

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:50
02 Std. Gas	0.081	13:50
03 Room Air	0.000	13:51
04 Std. Gas	0.080	13:51
05 Room Air	0.000	13:51
06 Std. Gas	0.081	13:52
07 Room Air	0.000	13:52

Lot No = 19615080A5
Cyl No = 26
Exp Date = 09/05/2017
County = 08 Oper No. = 777777


Operator Signature
DEB SHANAVER

Remarks:

Calibration check 0.080AC