

INTOXILYZER® 8000 INSPECTION

DS

Serial No.: 80-004200

Location: 8000 TOXL

- Check When Done
- A. Is the breath tube heated? ✓
 - B. Is the diagnostic check complete? ✓
 - C. Is the time, date, and year correct (re-set if necessary)? ✓
 - D. Print test. (Attach test record.) ✓
 - E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN DR3374 Lot No. 201501B Rep AC 0.020 ✓
 - F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN DR3841 Lot No. 201603A Rep AC 0.300 ✓
 - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN DR5144 Lot No. ICS 4 Solution 0.10AC + 0.05% acetone ✓
 - H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) ✓
 - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) 0.080 AC

Lot No. 29614080A2 Cyl No. 12 Exp. Date 12/5/2016

Test 1 <u>0.082</u>	Test 4 <u>0.081</u>	Test 7 <u>0.081</u>
Test 2 <u>0.081</u>	Test 5 <u>0.082</u>	Test 8 <u>0.081</u>
Test 3 <u>0.082</u>	Test 6 <u>0.081</u>	Test 9 <u>0.082</u>

Average 0.081

J. Tank pressure, Level 3 Func D Sub G P: 887 psi Reg 900 psi

K. Remarks/Maintenance record: Annual inspection, replaced breath hose/simulator return o-rings, cleaned breath hose screen

23 May 16
Date

23 MAY 16
Date

Deb Shanover
Field Inspector's Signature

Charles E. [Signature]
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004200
Location = TOXL 8164.13.00 06/09
05/23/2016 09:28

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=-|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=-|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 29614080A2
Standard Cyl #? 12
Standard Expiration? 12/05/2016
Oper No? 777777

Flow Cal. Date: 12/10/2012
Slope 637
Intercept -382487

IR Calibration Date: 12/10/2012
 3um 9um


0th Coef(*100): -14942 -22738
1st Coef(*100): 256196 136921
2nd Coef(*100): 3535 1396
H2O adj(mg/l*10k): 492 515

***** Printer Test End *****



Operator Signature
DEB SHANAVER

Remarks:

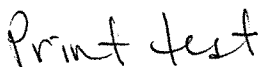

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004200
Location = TOXL 8164.13.00 06/09
05/23/2016 10:41

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:41
02 Std. Sol.	0.018	10:42
03 Room Air	0.000	10:43
04 Std. Sol.	0.019	10:44
05 Room Air	0.000	10:44
06 Std. Sol.	0.018	10:45
07 Room Air	0.000	10:45

08 Sim Temp = 34.0°C

Simul Ser No = DR3374
Std Sol No = 201501B
County = 08 Oper No. = 777777

Deb Shanaver
Operator Signature
DEB SHANAVER

Remarks:

low alcohol 0.020 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004200
Location = TOXL 8164.13.00 06/09
05/23/2016 10:04

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:05
02 Std. Sol.	0.296	10:06
03 Room Air	0.000	10:06
04 Std. Sol.	0.297	10:07
05 Room Air	0.000	10:07
06 Std. Sol.	0.298	10:08
07 Room Air	0.000	10:09

08 Sim Temp = 34.0°C

Simul Ser No = DR3841
Std Sol No = 201603A
County = 08 Oper No. = 777777

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

High alcohol 0.300AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004200
Location = TOXL 8164.13.00 06/09
05/23/2016 11:24

Test	AC	Time
01 Room Air	0.000	11:25
02 *Subject Test	INT*	11:25
03 Room Air	0.000	11:26

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 29614080A2
Cyl No = 12
Expiration Date = 12/05/2016
County = 08 Oper No. = 777777


Operator Signature
DEB SHANAVER

Remarks:

Interferent check 0.10AC + 0.05% acetone

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004200
Location = TOXL 8164.13.00 06/09
05/23/2016 11:27

Test	AC	Time
01 Diagnostic	OK	11:28
02 Room Air	RFI*	11:28
03 Room Air	0.000	11:29

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 29614080A2
Cyl No = 12
Expiration Date = 12/05/2016
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
DEB SHANAVER

Remarks:

RFI test

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004200
Location = TOXL 8164.13.00 06/09
05/23/2016 10:49

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:49
02 Std. Gas	0.082	10:49
03 Room Air	0.000	10:50
04 Std. Gas	0.081	10:50
05 Room Air	0.000	10:51
06 Std. Gas	0.082	10:51
07 Room Air	0.000	10:52

Lot No = 29614080A2
Cyl No = 12
Exp Date = 12/05/2016
County = 08 Oper No. = 777777

Deb Shanaver
Operator Signature
DEB SHANAVER

Remarks:

Calibration check 0.080AC


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004200
Location = TOXL 8164.13.00 06/09
05/23/2016 11:10

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:11
02 Std. Gas	0.081	11:11
03 Room Air	0.000	11:12
04 Std. Gas	0.082	11:12
05 Room Air	0.000	11:12
06 Std. Gas	0.081	11:13
07 Room Air	0.000	11:13

Lot No = 29614080A2
Cyl No = 12
Exp Date = 12/05/2016
County = 08 Oper No. = 777777



Operator Signature
DEB SHANAVER

Remarks:

Calibration check 0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004200
Location = TOXL 8164.13.00 06/09
05/23/2016 11:14

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:15
02 Std. Gas	0.081	11:15
03 Room Air	0.000	11:16
04 Std. Gas	0.081	11:16
05 Room Air	0.000	11:16
06 Std. Gas	0.082	11:17
07 Room Air	0.000	11:17

Lot No = 29614080A2
Cyl No = 12
Exp Date = 12/05/2016
County = 08 Oper No. = 777777

Deb Shanaver
Operator Signature
DEB SHANAVER

Remarks:

Calibration check 0.080 AC