

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-004949

Location: STANCOE T (MERCER COUNTY S.O.)

- Check When Done
- A. Is the breath tube heated? ✓
 - B. Is the diagnostic check complete? ✓
 - C. Is the time, date, and year correct (re-set if necessary)? ✓
 - D. Print test. (Attach test record.) ✓
 - E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN DR5191 Lot No. 15160 Rep AC 0.020 ✓
 - F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN DR3847 Lot No. 16060 Rep AC 0.300 ✓
 - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN DR5190 Lot No. ICS4 Solution 0.10AC + 0.05% ACETONE ✓
 - H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) ✓
 - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) 0.080 AC

Lot No. 13314080A2 Cyl No. 059 Exp. Date 06/05/2016

Test 1 <u>0.078</u>	Test 4 <u>0.078</u>	Test 7 <u>0.079</u>
Test 2 <u>0.078</u>	Test 5 <u>0.078</u>	Test 8 <u>0.078</u>
Test 3 <u>0.078</u>	Test 6 <u>0.078</u>	Test 9 <u>0.078</u>

Average 0.078

J. Tank pressure, Level 3 Func D Sub G P: 888 psi Reg ~900 psi

K. Remarks/Maintenance record: ADD ANNUAL INSPECTION

5/24/2016
Date

Charles E. Ed
Field Inspector's Signature

24 May 16
Date

Deb Shanaver
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004949
Location = STAT 8164.13.00 06/09
05/24/2016 14:34

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 13314080A2
Standard Cyl #? 59
Standard Expiration? 06/05/2016
Oper No? 666666

Flow Cal. Date: 09/24/2010
Slope 784
Intercept -862571

IR Calibration Date: 04/08/2015
 3um 9um

0th Coef(*100): -19414 -19489
1st Coef(*100): 269741 132608
2nd Coef(*100): 2662 1599
H2O adj(mg/l*10k): 288 363

***** Printer Test End *****

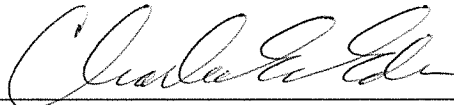

Operator Signature
CHARLES EDER

Remarks: PRINT TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004949
Location = STAT 8164.13.00 06/09
05/24/2016 14:44

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:44
02 Std. Sol.	0.018	14:45
03 Room Air	0.000	14:46
04 Std. Sol.	0.019	14:46
05 Room Air	0.000	14:47
06 Std. Sol.	0.019	14:48
07 Room Air	0.000	14:48

08 Sim Temp = 34.0°C

Simul Ser No = DR5191
Std Sol No = 15160
County = 29 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *Low AC*
 0.020 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004949
Location = STAT 8164.13.00 06/09
05/24/2016 14:49

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:50
02 Std. Sol.	0.299	14:51
03 Room Air	0.000	14:51
04 Std. Sol.	0.302	14:52
05 Room Air	0.000	14:53
06 Std. Sol.	0.303	14:53
07 Room Air	0.000	14:54

08 Sim Temp = 34.0°C

Simul Ser No = DR3847
Std Sol No = 16060
County = 29 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: HIGH AC
0.300 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004949
Location = STAT 8164.13.00 06/09
05/24/2016 14:40

Test	AC	Time
01 Room Air	0.000	14:41
02 *Subject Test	INT*	14:42
03 Room Air	0.000	14:42

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 13314080A2
Cyl No = 59
Expiration Date = 06/05/2016
County = 29 Oper No. = 666666

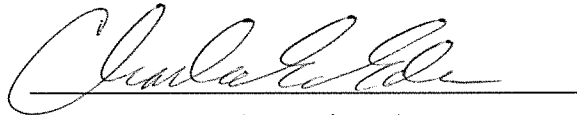

Operator Signature
CHARLES EDER

Remarks:

Interferent Check
0.10Ac + 0.05% ACETONE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004949
Location = STAT 8164.13.00 06/09
05/24/2016 14:38

Test	AC	Time
01 Diagnostic	OK	14:39
02 Room Air	RFI*	14:39
03 Room Air	0.000	14:40

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 13314080A2
Cyl No = 59
Expiration Date = 06/05/2016
County = 29 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks: RFI CHECK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004949
Location = STAT 8164.13.00 06/09
05/24/2016 14:59

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:59
02 Std. Gas	0.078	15:00
03 Room Air	0.000	15:00
04 Std. Gas	0.078	15:00
05 Room Air	0.000	15:01
06 Std. Gas	0.078	15:01
07 Room Air	0.000	15:02

Lot No = 13314080A2

Cyl No = 59

Exp Date = 06/05/2016

County = 29

Oper No. = 666666



Operator Signature

CHARLES EDER

Remarks:

CALIBRATION CHECK

0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004949
Location = STAT 8164.13.00 06/09
05/24/2016 15:02

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:03
02 Std. Gas	0.078	15:03
03 Room Air	0.000	15:03
04 Std. Gas	0.078	15:04
05 Room Air	0.000	15:04
06 Std. Gas	0.078	15:05
07 Room Air	0.000	15:05

Lot No = 13314080A2
Cyl No = 59
Exp Date = 06/05/2016
County = 29 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004949
Location = STAT 8164.13.00 06/09
05/24/2016 15:05

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:06
02 Std. Gas	0.079	15:06
03 Room Air	0.000	15:07
04 Std. Gas	0.078	15:07
05 Room Air	0.000	15:08
06 Std. Gas	0.078	15:08
07 Room Air	0.000	15:09

Lot No = 13314080A2
Cyl No = 59
Exp Date = 06/05/2016
County = 29 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000