

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-005361

Location: ToxL

- Check When Done
- A. Is the breath tube heated? ✓
 - B. Is the diagnostic check complete? ✓
 - C. Is the time, date, and year correct (re-set if necessary)? ✓
 - D. Print test. (Attach test record.) ✓
 - E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN MP3066 Lot No. 201507F Rep AC 0.02^a ✓
 - F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN MP3048 Lot No. 201603A Rep AC 0.300 ✓
 - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN DR5135 Lot No. ICS4 Solution 0.10AC + 0.05% ACETONE ✓
 - H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) ✓

I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) 0.080 AC

Lot No. <u>13314080A3</u>	Cyl No. <u>034</u>	Exp. Date <u>06/05/2016</u>
Test 1 <u>0.080</u>	Test 4 <u>0.080</u>	Test 7 <u>0.080</u>
Test 2 <u>0.081</u>	Test 5 <u>0.080</u>	Test 8 <u>0.081</u>
Test 3 <u>0.080</u>	Test 6 <u>0.081</u>	Test 9 <u>0.081</u>
Average <u>0.080</u>		

J. Tank pressure, Level 3 Func D Sub G P: 731 psi Reg ~750 psi

K. Remarks/Maintenance record: ANNUAL INSPECTION

5/24/2016
Date

Charles E. Egan
Field Inspector's Signature

24 May 16
Date

Deb Shanover
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005361
Location = TOXL 8164.13.00 06/09
05/24/2016 10:43

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:43
02 Std. Sol.	0.019	10:44
03 Room Air	0.000	10:45
04 Std. Sol.	0.020	10:45
05 Room Air	0.000	10:46
06 Std. Sol.	0.019	10:47
07 Room Air	0.000	10:47


08 Sim Temp = 34.0°C

Simul Ser No = MP3066

Std Sol No = 201507F

County = 08

Oper No. = 666666

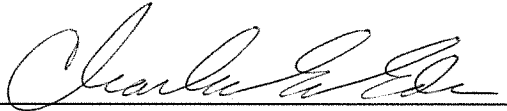

Operator Signature
CHARLES EDER

Remarks:

Low AC
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005361
Location = TOXL 8164.13.00 06/09
05/24/2016 10:53

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:53
02 Std. Sol.	0.294	10:54
03 Room Air	0.000	10:55
04 Std. Sol.	0.297	10:55
05 Room Air	0.000	10:56
06 Std. Sol.	0.296	10:57
07 Room Air	0.000	10:57

08 Sim Temp = 34.0°C

Simul Ser No = MP3048
Std Sol No = 201603A
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: HIGH AC
 0.300 AC


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005361
Location = TOXL 8164.13.00 06/09
05/24/2016 11:08

Test	AC	Time
01 Room Air	0.000	11:09
02 *Subject Test	INT*	11:09
03 Room Air	0.000	11:10

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 13314080A3
Cyl No = 34
Expiration Date = 06/05/2016
County = 08 Oper No. = 666666

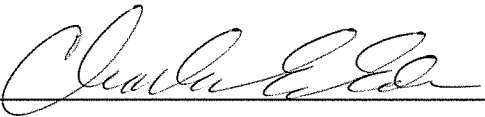
Operator Signature
CHARLES EDER

Remarks: *Interferent Check*
0.10 AC + 0.05% ACETONE

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005361
Location = TOXL 8164.13.00 06/09
05/24/2016 11:11

Test	AC	Time
01 Diagnostic	OK	11:12
02 Room Air	RFI*	11:12
03 Room Air	0.000	11:12

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 13314080A3
Cyl No = 34
Expiration Date = 06/05/2016
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks:

RFI CHECK


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005361
Location = TOXL 8164.13.00 06/09
05/24/2016 11:14

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:15
02 Std. Gas	0.080	11:15
03 Room Air	0.000	11:16
04 Std. Gas	0.081	11:16
05 Room Air	0.000	11:17
06 Std. Gas	0.080	11:17
07 Room Air	0.000	11:18

Lot No = 13314080A3
Cyl No = 34
Exp Date = 06/05/2016
County = 08 Oper No. = 666666

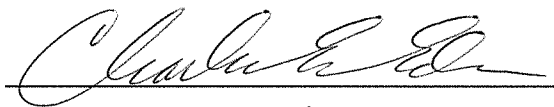

Operator Signature
CHARLES EDER

Remarks:

CALIBRATION CHECK
0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005361
Location = TOXL 8164.13.00 06/09
05/24/2016 11:24

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:25
02 Std. Gas	0.080	11:25
03 Room Air	0.000	11:25
04 Std. Gas	0.080	11:26
05 Room Air	0.000	11:26
06 Std. Gas	0.081	11:27
07 Room Air	0.000	11:27

Lot No = 13314080A3
Cyl No = 34
Exp Date = 06/05/2016
County = 08 Oper No. = 666666

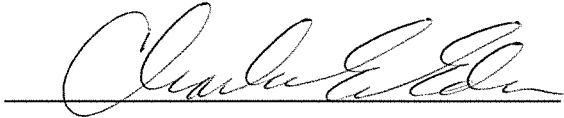

Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK

0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005361
Location = TOXL 8164.13.00 06/09
05/24/2016 11:27

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:28
02 Std. Gas	0.080	11:28
03 Room Air	0.000	11:29
04 Std. Gas	0.081	11:29
05 Room Air	0.000	11:30
06 Std. Gas	0.081	11:30
07 Room Air	0.000	11:31

Lot No = 13314080A3
Cyl No = 34
Exp Date = 06/05/2016
County = 08 Oper No. = 666666

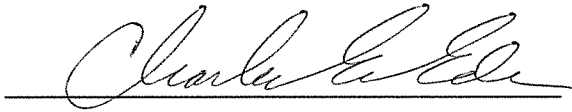

Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC
Form 106-I8000