

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-003060

Location: TOXL

- Check When Done
- A. Is the breath tube heated? ✓
 - B. Is the diagnostic check complete? ✓
 - C. Is the time, date, and year correct (re-set if necessary)? ✓
 - D. Print test. (Attach test record.) ✓
 - E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN DR3374 Lot No. 201501B Rep AC 0.020 ✓
 - F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN DR3841 Lot No. 201603A Rep AC 0.300 ✓
 - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN DR5144 Lot No. ICS4 Solution 0.10 AC + 0.05% acetone ✓
 - H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) ✓
 - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) 0.080 AC

Lot No. 14214080A6 Cyl No. 51 Exp. Date 07/05/2016

Test 1 <u>0.079</u>	Test 4 <u>0.080</u>	Test 7 <u>0.080</u>
Test 2 <u>0.079</u>	Test 5 <u>0.079</u>	Test 8 <u>0.080</u>
Test 3 <u>0.079</u>	Test 6 <u>0.080</u>	Test 9 <u>0.079</u>

Average 0.079

J. Tank pressure, Level 3 Func D Sub G P: 398 psi Reg 400 psi

K. Remarks/Maintenance record: Annual inspection, no sign of a voltage fail.

24 May 2016
Date

25 May 2016 ✓
Date

Deb Shanaver
Field Inspector's Signature

[Signature]
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003060
Location = TOXL 8164.13.00 06/09
05/24/2016 14:00

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNQPQRSTUVWXYZ!@#%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNQPQRSTUVWXYZ!@#%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 14214080A6
Standard Cyl #? 51
Standard Expiration? 07/05/2016
Oper No? 777777

Flow Cal. Date: 04/08/2008
Slope 702
Intercept -647610

IR Calibration Date: 06/04/2008
 3um 9um

0th Coef(*100): -5190 -7737
1st Coef(*100): 270580 138779
2nd Coef(*100): 2499 956
H2O adj(mg/l*10k): 377 188

***** Printer Test End *****



Operator Signature
DEB SHANAVER

Remarks:

Print test

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003060
Location = TOXL 8164.13.00 06/09
05/24/2016 14:44

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:45
02 Std. Sol.	0.020	14:46
03 Room Air	0.000	14:46
04 Std. Sol.	0.020	14:47
05 Room Air	0.000	14:47
06 Std. Sol.	0.020	14:48
07 Room Air	0.000	14:49

08 Sim Temp = 34.0°C

DR 3374
Simul Ser No = ~~DR334~~ DS
Std Sol No = 201501B
County = 08 Oper No. = 777777



Operator Signature
DEB SHANAVER

Remarks:

low alcohol 0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003060
Location = TOXL 8164.13.00 06/09
05/24/2016 14:52

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:53
02 Std. Sol.	0.296	14:55
03 Room Air	0.000	14:56
04 Std. Sol.	0.296	14:56
05 Room Air	0.000	14:57
06 Std. Sol.	0.297	14:58
07 Room Air	0.000	14:58

08 Sim Temp = 34.0°C

Simul Ser No = DR3841
Std Sol No = 201603A
County = 08 Oper No. = 777777

Deb Shanaver
Operator Signature
DEB SHANAVER

Remarks:

High alcohol 0.300AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003060
Location = TOXL 8164.13.00 06/09
05/24/2016 15:02

Test	AC	Time
01 Room Air	0.000	15:03
02 *Subject Test	INT*	15:03
03 Room Air	0.000	15:04

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 14214080A6
Cyl No = 51
Expiration Date = 07/05/2016
County = 08 Oper No. = 777777

Deb Shanaver
Operator Signature
DEB SHANAVER

Remarks:

Interferent detected 0.10AC + 0.05% acetone

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003060
Location = TOXL 8164.13.00 06/09
05/24/2016 15:05

Test	AC	Time
01 Diagnostic	OK	15:06
02 Room Air	RFI*	15:06
03 Room Air	0.000	15:06

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 14214080A6
Cyl No = 51
Expiration Date = 07/05/2016
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

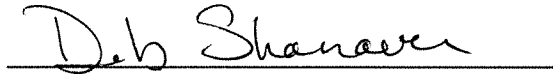

Operator Signature
DEB SHANAVER

Remarks:

RFI test

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003060
Location = TOXL 8164.13.00 06/09
05/24/2016 14:07

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:08
02 Std. Gas	0.079	14:08
03 Room Air	0.000	14:09
04 Std. Gas	0.079	14:09
05 Room Air	0.000	14:10
06 Std. Gas	0.079	14:10
07 Room Air	0.000	14:11

Lot No = 14214080A6
Cyl No = 51
Exp Date = 07/05/2016
County = 08 Oper No. = 777777

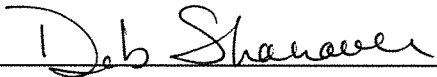
Operator Signature
DEB SHANAVER

Remarks:

Calibration check 0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003060
Location = TOXL 8164.13.00 06/09
05/24/2016 14:18

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:18
02 Std. Gas	0.080	14:18
03 Room Air	0.000	14:19
04 Std. Gas	0.079	14:19
05 Room Air	0.000	14:20
06 Std. Gas	0.080	14:20
07 Room Air	0.000	14:21

Lot No = 14214080A6
Cyl No = 51
Exp Date = 07/05/2016
County = 08 Oper No. = 777777

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

Calibration check 0.080AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003060
Location = TOXL 8164.13.00 06/09
05/24/2016 14:26

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:27
02 Std. Gas	0.080	14:27
03 Room Air	0.000	14:28
04 Std. Gas	0.080	14:28
05 Room Air	0.000	14:29
06 Std. Gas	0.079	14:29
07 Room Air	0.000	14:29

Lot No = 14214080A6
Cyl No = 51
Exp Date = 07/05/2016
County = 08 Oper No. = 777777



Operator Signature
DEB SHANAVER

Remarks:

Calibration check 0.080 AC

Form 106-I8000