

INTOXILYZER® 8000 INSPECTION

Serial No.: 00-004956

Location: WASH (McLEAN County S.O.)

- Check When Done
- A. Is the breath tube heated? ✓
  - B. Is the diagnostic check complete? ✓
  - C. Is the time, date, and year correct (re-set if necessary)? ✓
  - D. Print test. (Attach test record.) ✓
  - E. Low AC. Use  $\leq 0.03$  AC in ACA mode. (Attach test record.)  
Sim SN DR5191 Lot No. 15160 Rep AC 0.020 ✓
  - F. High AC. Use  $\geq 0.25$  AC in ACA mode. (Attach test record.)  
Sim SN DR3847 Lot No. 16060 Rep AC 0.300 ✓
  - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.  
(Attach test record.)  
Sim SN DR5190 Lot No. ICS4 Solution 0.10AC + 0.05% ACETONE ✓
  - H. RFI check. Run CMS mode. Key radio during CMS test.  
(Attach test record.) ✓
  - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.  
(Attach test record.) 0.080 AC

Lot No. 13314080A3 Cyl No. 030 Exp. Date 06/05/2016

|                     |                     |                     |
|---------------------|---------------------|---------------------|
| Test 1 <u>0.079</u> | Test 4 <u>0.079</u> | Test 7 <u>0.080</u> |
| Test 2 <u>0.079</u> | Test 5 <u>0.079</u> | Test 8 <u>0.079</u> |
| Test 3 <u>0.079</u> | Test 6 <u>0.079</u> | Test 9 <u>0.079</u> |

Average 0.079

J. Tank pressure, Level 3 Func D Sub G P: 871 psi Reg ~900 psi

K. Remarks/Maintenance record: ANNUAL INSPECTION

5/24/2016  
Date

Charles E. Eln  
Field Inspector's Signature

25 May 16 ✓  
Date

Deb Shanaver  
Reviewed by



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004956  
Location = WASH      8164.13.00 06/09  
05/24/2016      16:05

WET CAL CHECK

| Test         | AC    | Time  |
|--------------|-------|-------|
| 01 Room Air  | 0.000 | 16:05 |
| 02 Std. Sol. | 0.019 | 16:06 |
| 03 Room Air  | 0.000 | 16:07 |
| 04 Std. Sol. | 0.019 | 16:07 |
| 05 Room Air  | 0.000 | 16:08 |
| 06 Std. Sol. | 0.019 | 16:09 |
| 07 Room Air  | 0.000 | 16:09 |

08 Sim Temp = 34.0°C

Simul Ser No = DR5191

Std Sol No = 15160

County = 28

Oper No. = 666666



Operator Signature

CHARLES EDER

Remarks:

*Low AC*

*0.020 AC*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004956  
Location = WASH      8164.13.00 06/09  
05/24/2016      16:10

WET CAL CHECK

| Test         | AC    | Time  |
|--------------|-------|-------|
| 01 Room Air  | 0.000 | 16:10 |
| 02 Std. Sol. | 0.294 | 16:11 |
| 03 Room Air  | 0.000 | 16:12 |
| 04 Std. Sol. | 0.295 | 16:13 |
| 05 Room Air  | 0.000 | 16:13 |
| 06 Std. Sol. | 0.295 | 16:14 |
| 07 Room Air  | 0.000 | 16:14 |


08 Sim Temp = 34.0°C

Simul Ser No = DR3847

Std Sol No = 16060

County = 28

Oper No. = 666666

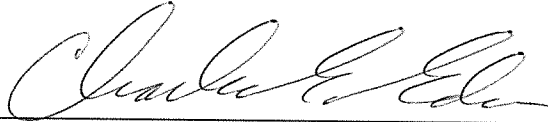

Operator Signature

CHARLES EDER

Remarks:

HIGH AC

0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004956  
Location = WASH                8164.13.00 06/09  
05/24/2016                        16:00

| Test             | AC    | Time  |
|------------------|-------|-------|
| 01 Room Air      | 0.000 | 16:01 |
| 02 *Subject Test | INT*  | 16:02 |
| 03 Room Air      | 0.000 | 16:03 |

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                Weight = 150  
Test = DUI                        Cit = INTERFERENT  
Dr. Lic. = ND/DIS821456  
Lot No = 13314080A3  
Cyl No = 30  
Expiration Date = 06/05/2016  
County = 28                        Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

*Interferent Check*  
*0.10 AC + 0.05% ACETONE*

Form 106-I8000



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004956  
Location = WASH      8164.13.00 06/09  
05/24/2016      16:29

DRY CAL CHECK

| Test        | AC    | Time  |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 16:30 |
| 02 Std. Gas | 0.079 | 16:30 |
| 03 Room Air | 0.000 | 16:30 |
| 04 Std. Gas | 0.079 | 16:31 |
| 05 Room Air | 0.000 | 16:31 |
| 06 Std. Gas | 0.079 | 16:32 |
| 07 Room Air | 0.000 | 16:32 |

Lot No = 13314080A3

Cyl No = 30

Exp Date = 06/05/2016

County = 28

Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

CALIBRATION CHECK

0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004956  
Location = WASH      8164.13.00 06/09  
05/24/2016      16:32

DRY CAL CHECK

| Test        | AC    | Time  |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 16:33 |
| 02 Std. Gas | 0.079 | 16:33 |
| 03 Room Air | 0.000 | 16:34 |
| 04 Std. Gas | 0.079 | 16:34 |
| 05 Room Air | 0.000 | 16:35 |
| 06 Std. Gas | 0.079 | 16:35 |
| 07 Room Air | 0.000 | 16:35 |

Lot No = 13314080A3

Cyl No = 30

Exp Date = 06/05/2016

County = 28

Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

CALIBRATION CHECK  
0.080 AC

Form 106-I8000



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004956  
Location = WASH      8164.13.00 06/09  
05/24/2016      16:36

DRY CAL CHECK

| Test        | AC    | Time  |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 16:36 |
| 02 Std. Gas | 0.080 | 16:36 |
| 03 Room Air | 0.000 | 16:37 |
| 04 Std. Gas | 0.079 | 16:37 |
| 05 Room Air | 0.000 | 16:38 |
| 06 Std. Gas | 0.079 | 16:38 |
| 07 Room Air | 0.000 | 16:39 |

Lot No = 13314080A3  
Cyl No = 30  
Exp Date = 06/05/2016  
County = 28      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

CALIBRATION CHECK  
0.080 AC

Form 106-I8000