

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-004203

Location: RUGB  
35

(HEART OF AMERICA CORRECTIONS)

- Check When Done
- A. Is the breath tube heated? ✓
  - B. Is the diagnostic check complete? ✓
  - C. Is the time, date, and year correct (re-set if necessary)? ✓
  - D. Print test. (Attach test record.) ✓
  - E. Low AC. Use  $\leq 0.03$  AC in ACA mode. (Attach test record.)  
Sim SN DR5191 Lot No. 15160 Rep AC 0.020 ✓
  - F. High AC. Use  $\geq 0.25$  AC in ACA mode. (Attach test record.)  
Sim SN DR3847 Lot No. 16060 Rep AC 0.300 ✓
  - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.  
(Attach test record.)  
Sim SN DR5190 Lot No. JCS4 Solution 0.10AC + 0.05% ✓  
ACETONE
  - H. RFI check. Run CMS mode. Key radio during CMS test.  
(Attach test record.) ✓
  - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.  
(Attach test record.) 0.080 AC

Lot No. 14214080A6 Cyl No. 56 Exp. Date 07/05/2016

Test 1 <u>0.082</u>	Test 4 <u>0.081</u>	Test 7 <u>0.082</u>
Test 2 <u>0.081</u>	Test 5 <u>0.080</u>	Test 8 <u>0.081</u>
Test 3 <u>0.081</u>	Test 6 <u>0.080</u>	Test 9 <u>0.081</u>

Average 0.081

J. Tank pressure, Level 3 Func D Sub G P: 994 psi Reg ~1000 psi

K. Remarks/Maintenance record: ANNUAL INSPECTION, REPLACED BREATH

HOSE/SIMULATOR RETURN O-RINGS

5/25/2016  
Date

26 May 16 ✓  
Date

Charles E. [Signature]  
Field Inspector's Signature

Db Shanaver  
Reviewed by

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-004203  
Location = RUGB                    8164.13.00 06/09  
05/25/2016                            15:47

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&\*()\_+?

Current Instrument Setup


Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?         1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            14214080A6  
Standard Cyl #?            56  
Standard Expiration?       07/05/2016  
Oper No?                    666666

Flow Cal. Date:            01/28/2010  
Slope                      719  
Intercept                  -711903

IR Calibration Date:        01/28/2010  
                                        3um                            9um

-----  
0th Coef(\*100):            -17469                    -28710  
1st Coef(\*100):            264216                    135022  
2nd Coef(\*100):            2778                      1081  
H2O adj(mg/l\*10k):        430                        605

\*\*\*\*\* Printer Test End \*\*\*\*\*

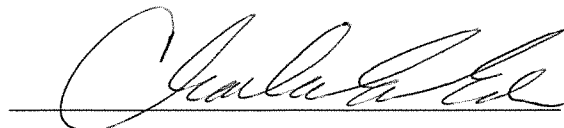

Operator Signature  
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004203  
Location = RUGB      8164.13.00 06/09  
05/25/2016      16:22

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:23
02 Std. Sol.	0.017	16:24
03 Room Air	0.000	16:24
04 Std. Sol.	0.017	16:25
05 Room Air	0.000	16:25
06 Std. Sol.	0.018	16:26
07 Room Air	0.000	16:26

08 Sim Temp = 34.0°C

Simul Ser No = DR5191  
Std Sol No = 15160  
County = 35      Oper No. = 666666

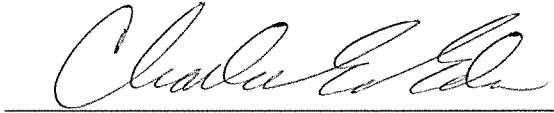

Operator Signature  
CHARLES EDER

Remarks:

*Low AC*  
*0.020 AC*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004203  
Location = RUGB      8164.13.00 06/09  
05/25/2016      16:16

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:17
02 Std. Sol.	0.289	16:18
03 Room Air	0.000	16:18
04 Std. Sol.	0.290	16:19
05 Room Air	0.000	16:20
06 Std. Sol.	0.291	16:20
07 Room Air	0.000	16:21


08 Sim Temp = 34.0°C

Simul Ser No = DR3847

Std Sol No = 16060

County = 35

Oper No. = 666666

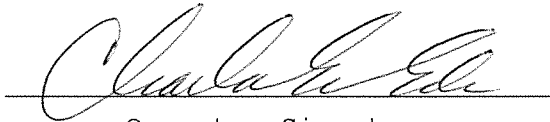

Operator Signature  
CHARLES EDER

Remarks:

HIGH AC  
0.300 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004203  
Location = RUGB                8164.13.00 06/09  
05/25/2016                        15:52

Test	AC	Time
01 Room Air	0.000	15:53
02 *Subject Test	INT*	15:54
03 Room Air	0.000	15:55

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = INTERFERENT  
Dr. Lic. = ND/DIS821456  
Lot No = 14214080A6  
Cyl No = 56  
Expiration Date = 07/05/2016  
County = 35                            Oper No. = 666666

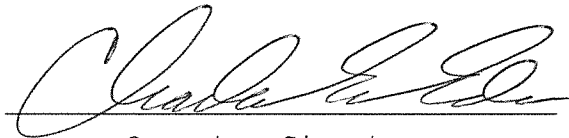

Operator Signature  
CHARLES EDER

Remarks: *Interferent Check*  
*0.10AC + 0.05% ACETONE*  
Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004203  
Location = RUGB                    8164.13.00 06/09  
05/25/2016                            15:56

Test	AC	Time
01 Diagnostic	OK	15:56
02 Room Air	0.000	15:57
03 *Subject Test	RFI*	15:57
04 Room Air	0.000	15:58

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = RFI CHECK  
Dr. Lic. = ND/DIS821456  
Lot No = 14214080A6  
Cyl No = 56  
Expiration Date = 07/05/2016  
County = 35                            Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
CHARLES EDER

Remarks: *RFI CHECK*


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004203  
Location = RUGB      8164.13.00 06/09  
05/25/2016      15:59

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:59
02 Std. Gas	0.082	15:59
03 Room Air	0.000	16:00
04 Std. Gas	0.081	16:00
05 Room Air	0.000	16:01
06 Std. Gas	0.081	16:01
07 Room Air	0.000	16:02

Lot No = 14214080A6  
Cyl No = 56  
Exp Date = 07/05/2016  
County = 35      Oper No. = 666666

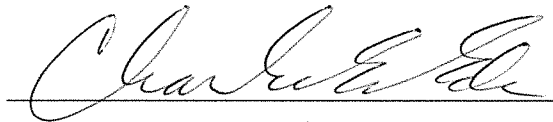

Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004203  
Location = RUGB      8164.13.00 06/09  
05/25/2016      16:02

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:03
02 Std. Gas	0.081	16:03
03 Room Air	0.000	16:04
04 Std. Gas	0.080	16:04
05 Room Air	0.000	16:04
06 Std. Gas	0.080	16:05
07 Room Air	0.000	16:05

Lot No = 14214080A6  
Cyl No = 56  
Exp Date = 07/05/2016  
County = 35      Oper No. = 666666

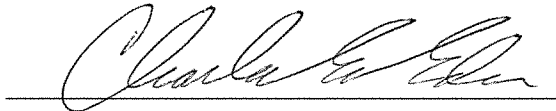

Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004203  
Location = RUGB      8164.13.00 06/09  
05/25/2016      16:28

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:29
02 Std. Gas	0.082	16:29
03 Room Air	0.000	16:30
04 Std. Gas	0.081	16:30
05 Room Air	0.000	16:30
06 Std. Gas	0.081	16:31
07 Room Air	0.000	16:31

Lot No = 14214080A6  
Cyl No = 56  
Exp Date = 07/05/2016  
County = 35      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-I8000