



**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**  
 OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION  
 Toxicology Section/Breath Alcohol Program  
 SFN 59281 (03-2014)

Serial Number <i>80-004942</i>	Instrument Location <i>BURKE Co. S.O. (BURK)</i>
Reason for Install/Repair <input type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input checked="" type="checkbox"/> Other (Specify) <u><i>INSTALL DURING ANNUAL INSPECTION</i></u>	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- NA* 2. Telephone Line Connected to Intoxilyzer® 8000. *SHARED W/FAX*
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P). ✓
  - B. ACA Test (Level 1, Function C). ✓
  - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test). ✓
- 7. Repair and/or Maintenance Performed (if Any): *INSTALL DURING ANNUAL INSPECTION*
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Charles E. Edr</i>	Date <i>5/27/15 ✓</i>
Reviewed By (Crime Laboratory Use Only) <i>Deb Kashner</i>	Date <i>29 May 15</i>

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-004942  
Location = BURK                    8164.13.00 06/09  
05/27/2015                            11:44

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNPNOPQRSTUVWXYZ!@#%&^\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNPNOPQRSTUVWXYZ!@#%&^\*()\_+?

Current Instrument Setup

Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?         1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            13314080A2  
Standard Cyl #?            58  
Standard Expiration?       06/05/2016  
Oper No?                    666666

Flow Cal. Date:            08/21/2012  
Slope                      638  
Intercept                  -487461

IR Calibration Date:       05/12/2015  
                                    3um                      9um

-----  
0th Coef(\*100):            -11858                    -18295  
1st Coef(\*100):            255387                   131150  
2nd Coef(\*100):            3253                     1510  
H2O adj(mg/l\*10k):        493                      483

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
CHARLES EDER

Remarks:

PRINT TEST


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004942  
Location = BURK      8164.13.00 06/09  
05/27/2015      11:45

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:46
02 Std. Gas	0.080	11:46
03 Room Air	0.000	11:47
04 Std. Gas	0.079	11:47
05 Room Air	0.000	11:48
06 Std. Gas	0.079	11:48
07 Room Air	0.000	11:49

Lot No = 13314080A2  
Cyl No = 58  
Exp Date = 06/05/2016  
County = 07      Oper No. = 666666

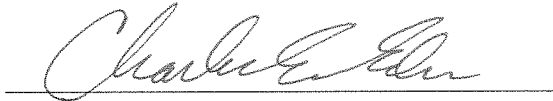

Operator Signature  
CHARLES EDER

Remarks:

ACA TEST  
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-004942  
Location = BURK                    8164.13.00 06/09  
05/27/2015                            11:50

Test	AC	Time
01 Diagnostic	OK	11:51
02 Room Air	0.000	11:51
03 *Subject Test	RFI*	11:52
04 Room Air	0.000	11:52

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = RFI CHECK  
Dr. Lic. = ND/DIS821456  
Lot No = 13314080A2  
Cyl No = 58  
Expiration Date = 06/05/2016  
County = 07                            Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

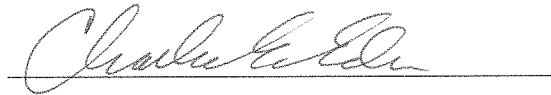

Operator Signature  
CHARLES EDER

Remarks:

RFI CHECK