



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 SFN 59281 (03-2014)

Serial Number <i>80-004942</i>	Instrument Location <i>BURKE Co.S.O. (BURK)</i>
Reason for Install/Repair <input type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input checked="" type="checkbox"/> Other (Specify) <u><i>INSTALL DURING ANNUAL INSPECTION</i></u>	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- ~~2.~~ Telephone Line Connected to Intoxilyzer® 8000. *SHARED W/FAX*
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P). ✓
 - B. ACA Test (Level 1, Function C). ✓
 - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test). ✓
- 7. Repair and/or Maintenance Performed (if Any): *INSTALL DURING ANNUAL INSPECTION*
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Charles E. Edin</i>	Date <i>5/27/15 ✓</i>
Reviewed By (Crime Laboratory Use Only) <i>Deb Kashner</i>	Date <i>29 May 15</i>

This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Certified By: *Charles E. Edin* Certified Date: *23 Nov. 2016*


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004942
Location = BURK 8164.13.00 06/09
05/27/2015 11:45

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:46
02 Std. Gas	0.080	11:46
03 Room Air	0.000	11:47
04 Std. Gas	0.079	11:47
05 Room Air	0.000	11:48
06 Std. Gas	0.079	11:48
07 Room Air	0.000	11:49

Lot No = 13314080A2
Cyl No = 58
Exp Date = 06/05/2016
County = 07 Oper No. = 666666

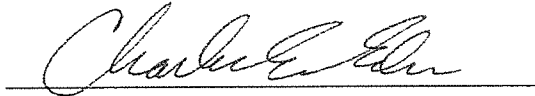

Operator Signature
CHARLES EDER

Remarks: *ACA TEST*
0.080 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004942
Location = BURK 8164.13.00 06/09
05/27/2015 11:50

Test	AC	Time
01 Diagnostic	OK	11:51
02 Room Air	0.000	11:51
03 *Subject Test	RFI*	11:52
04 Room Air	0.000	11:52

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 13314080A2
Cyl No = 58
Expiration Date = 06/05/2016
County = 07 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

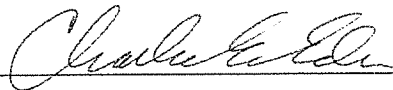

Operator Signature
CHARLES EDER

Remarks:

RFI CHECK