



**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**  
 OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION  
 Toxicology Section/Breath Alcohol Program  
 SFN 59281 (03-2014)

Serial Number <i>80-005957</i>	Instrument Location <i>DIVIDE COUNTY S.O. (CROS)</i>
Reason for Install/Repair <input type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input checked="" type="checkbox"/> Other (Specify) <u><i>INSTALL DURING ANNUAL INSPECTION</i></u>	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P). ✓
  - B. ACA Test (Level 1, Function C). ✓
  - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test). ✓
- 7. Repair and/or Maintenance Performed (if Any): *INSTALL DURING ANNUAL INSPECTION*
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Charles E. Edin</i>	Date <i>5/27/15 ✓</i>
Reviewed By (Crime Laboratory Use Only) <i>Deb Kashner</i>	Date <i>29 May 15</i>

This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Certified By: *Charles E. Edin* Certified Date: *21 Nov. 2016*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005957  
Location = CROS      8164.13.00 06/09  
05/27/2015      16:29

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890- = |  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890- = |  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^\*()\_+?

Current Instrument Setup


Data Entry Mode:      Enabled  
Start Test Sequence:      DABACABA  
Display Prelim Rslt?      Yes  
Display Third Digit?      Yes  
Inhib Printer(Y/N)?      No  
Display Volume?      No  
Disable On Memfull?      Yes  
# of Print Copies?      1  
Select Std (D/W/I)?      Dry  
Standard Value?      0.080  
Standard Lot #?      13314080A2  
Standard Cyl #?      57  
Standard Expiration?      06/05/2016  
Oper No?      666666

Flow Cal. Date:      06/21/2013  
Slope      647  
Intercept      -613317

IR Calibration Date:      05/11/2015  
  3um      9um

0th Coef(*100):	-16666	-18045
1st Coef(*100):	278478	132949
2nd Coef(*100):	3303	1397
H2O adj(mg/l*10k):	379	351

\*\*\*\*\* Printer Test End \*\*\*\*\*

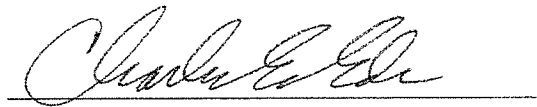

Operator Signature  
CHARLES EDER

Remarks:      PRINT TEST


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005957  
Location = CROS      8164.13.00 06/09  
05/27/2015      16:33

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:34
02 Std. Gas	0.078	16:34
03 Room Air	0.000	16:35
04 Std. Gas	0.078	16:35
05 Room Air	0.000	16:36
06 Std. Gas	0.077	16:36
07 Room Air	0.000	16:37

Lot No = 13314080A2  
Cyl No = 57  
Exp Date = 06/05/2016  
County = 12      Oper No. = 666666

  
Operator Signature  
CHARLES EDER

Remarks: *ACA TEST*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005957  
Location = CROS      8164.13.00 06/09  
05/27/2015      16:37

Test	AC	Time
01 Diagnostic	OK	16:38
02 Room Air	0.000	16:38
03 *Subject Test	RFI*	16:39
04 Room Air	0.000	16:39

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = RFI CHECK  
Dr. Lic. = ND/DIS821456  
Lot No = 13314080A2  
Cyl No = 57  
Expiration Date = 06/05/2016  
County = 12      Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
CHARLES EDER

Remarks:

*RFI CHECK*