



**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**  
 OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION  
 Toxicology Section/Breath Alcohol Program  
 SFN 59281 (03-2014)

Serial Number <i>80-005942</i>	Instrument Location <i>PLPD</i>
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): *W/A*
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Paul Malcolm</i>	Date <i>MAY 29, 2015 ✓</i>
Reviewed By (Crime Laboratory Use Only) <i>Deb Kashner</i>	Date <i>29 May 15</i>



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005942  
Location = PLPD      8164.13.00 06/09  
05/27/2015      14:59

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:00
02 Std. Gas	0.078	15:00
03 Room Air	0.000	15:01
04 Std. Gas	0.078	15:01
05 Room Air	0.000	15:02
06 Std. Gas	0.077	15:02
07 Room Air	0.000	15:03

Lot No = 14214080A6  
Cyl No = 57  
Exp Date = 07/05/2016  
County = 07      Oper No. = 131023



Operator Signature  
YACOB MAKONNEN

Remarks: ACA TEST

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005942  
Location = PLPD      8164.13.00 06/09  
05/27/2015      15:04

Test	AC	Time
01 Diagnostic	OK	15:06
02 Room Air	0.000	15:06
03 *Subject Test	RFI*	15:06
04 Room Air	0.000	15:07

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = RFI CHECK  
Dr. Lic. = ND/DIS821456  
Lot No = 14214080A6  
Cyl No = 57  
Expiration Date = 07/05/2016  
County = 07      Oper No. = 131023

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
YACOB MAKONNEN

Remarks: RFI CHECK