

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-005950

Location: SHER (SHERIDAN COUNTY S.O.)
47

- Check When Done
- A. Is the breath tube heated? ✓
 - B. Is the diagnostic check complete? ✓
 - C. Is the time, date, and year correct (re-set if necessary)? ✓
 - D. Print test. (Attach test record.) ✓
 - E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN DR5191 Lot No. 15160 Rep AC 0.020 ✓
 - F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN DR3847 Lot No. 16060 Rep AC 0.300 ✓
 - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN DR5190 Lot No. ICS4 Solution 0.10 AC + 0.05% ACETONE ✓
 - H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) ✓
 - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) 0.080 AC

Lot No. 14214080A6 Cyl No. 60 Exp. Date 07/05/2016

Test 1 <u>0.081</u>	Test 4 <u>0.082</u>	Test 7 <u>0.080</u>
Test 2 <u>0.080</u>	Test 5 <u>0.080</u>	Test 8 <u>0.081</u>
Test 3 <u>0.080</u>	Test 6 <u>0.081</u>	Test 9 <u>0.081</u>

Average 0.081

J. Tank pressure, Level 3 Func D Sub G P: 775 psi Reg 800 psi

K. Remarks/Maintenance record: ANNUAL INSPECTION

5/26/2016
Date

Charles E. Ely
Field Inspector's Signature

27 May 16 ✓
Date

Deb Shanaver
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005950
Location = SHER 8164.13.00 06/09
05/26/2016 12:59

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-|=|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-|=|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 14214080A6
Standard Cyl #? 60
Standard Expiration? 07/05/2016
Oper No? 666666

Flow Cal. Date: 06/20/2013
Slope 713
Intercept -723515

IR Calibration Date: 06/20/2013
 3um 9um

0th Coef(*100): -37081 -23816
1st Coef(*100): 290549 133411
2nd Coef(*100): 552 1090
H2O adj(mg/l*10k): 683 502

***** Printer Test End *****



Operator Signature
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005950
Location = SHER 8164.13.00 06/09
05/26/2016 13:19

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:20
02 Std. Sol.	0.018	13:21
03 Room Air	0.000	13:21
04 Std. Sol.	0.018	13:22
05 Room Air	0.000	13:22
06 Std. Sol.	0.018	13:23
07 Room Air	0.000	13:24


08 Sim Temp = 34.0°C

Simul Ser No = DR5191

Std Sol No = 15160

County = 42

Oper No. = 666666

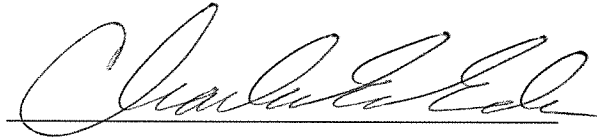

Operator Signature

CHARLES EDER

Remarks:

Low AC
0.018 AC 0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005950
Location = SHER 8164.13.00 06/09
05/26/2016 13:14

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:15
02 Std. Sol.	0.305	13:16
03 Room Air	0.000	13:16
04 Std. Sol.	0.306	13:17
05 Room Air	0.000	13:17
06 Std. Sol.	0.307	13:18
07 Room Air	0.000	13:19


08 Sim Temp = 34.0°C

Simul Ser No = DR3847

Std Sol No = 16060

County = 42

Oper No. = 666666

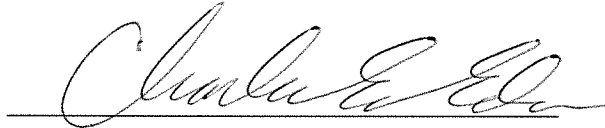

Operator Signature
CHARLES EDER

Remarks:

HIGH AC
0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005950
Location = SHER 8164.13.00 06/09
05/26/2016 13:06

Test	AC	Time
01 Room Air	0.000	13:07
02 *Subject Test	INT*	13:07
03 Room Air	0.000	13:08

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 14214080A6
Cyl No = 60
Expiration Date = 07/05/2016
County = 42 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Interferent Check
0.16% AC + 0.05% ACETONE
Eder

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005950
Location = SHER 8164.13.00 06/09
05/26/2016 12:59

Test	AC	Time
01 Diagnostic	OK	13:00
02 Room Air	0.000	13:01
03 *Subject Test	RFI*	13:01
04 Room Air	0.000	13:02

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 14214080A6
Cyl No = 60
Expiration Date = 07/05/2016
County = 42 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks:

RFI CHECK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005950
Location = SHER 8164.13.00 06/09
05/26/2016 13:25

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:26
02 Std. Gas	0.081	13:26
03 Room Air	0.000	13:27
04 Std. Gas	0.080	13:27
05 Room Air	0.000	13:28
06 Std. Gas	0.080	13:28
07 Room Air	0.000	13:29

Lot No = 14214080A6
Cyl No = 60
Exp Date = 07/05/2016
County = 42 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005950
Location = SHER 8164.13.00 06/09
05/26/2016 13:29

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:29
02 Std. Gas	0.082	13:30
03 Room Air	0.000	13:30
04 Std. Gas	0.080	13:31
05 Room Air	0.000	13:31
06 Std. Gas	0.081	13:32
07 Room Air	0.000	13:32

Lot No = 14214080A6
Cyl No = 60
Exp Date = 07/05/2016
County = 42 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005950
Location = SHER 8164.13.00 06/09
05/26/2016 13:33

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:34
02 Std. Gas	0.080	13:34
03 Room Air	0.000	13:34
04 Std. Gas	0.081	13:35
05 Room Air	0.000	13:35
06 Std. Gas	0.081	13:36
07 Room Air	0.000	13:36


Lot No = 14214080A6

Cyl No = 60

Exp Date = 07/05/2016

County = 42

Oper No. = 666666

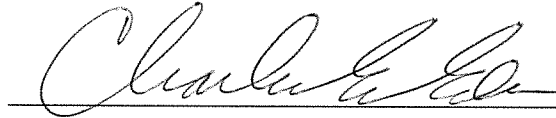

Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000