



**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**  
 OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION  
 Toxicology Section/Breath Alcohol Program  
 SFN 59281 (03-2014)

Serial Number <i>80-05944</i>	Instrument Location <i>Richland County Jail</i>
Reason for Install/Repair <input type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input checked="" type="checkbox"/> Other (Specify) <i>exchanged old intoxilyzer for new one</i>	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): *None*
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>[Signature]</i>	Date <i>3-26-14</i> ✓
Reviewed By (Crime Laboratory Use Only) <i>Deb Kasher</i>	Date <i>28 May 14</i>

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer           Alcohol Analyzer  
North Dakota Model 8000           SN 80-005944  
Location = WAHP                   8164.13.00 06/09  
03/26/2014                           21:20

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^\*()\_+?

Current Instrument Setup

Data Entry Mode:           Enabled  
Start Test Sequence:       DABACABA  
Display Prelim Rslt?       Yes  
Display Third Digit?       Yes  
Inhib Printer(Y/N)?       No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?         1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            16512080A2  
Standard Cyl #?            58  
Standard Expiration?       08/01/2014  
Oper No?                    020401

Flow Cal. Date:            06/20/2013  
Slope                      677  
Intercept                  -702620

IR Calibration Date:       06/20/2013  
                              3um                    9um

-----  
0th Coef(\*100):           -20012                   -21455  
1st Coef(\*100):           275397                   137470  
2nd Coef(\*100):           2699                     1252  
H2O adj(mg/l\*10k):       704                     563

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
MATTHEW ANDERSON

Remarks:

*Good Print Test*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-005944  
Location = WAHP                8164.13.00 06/09  
03/26/2014                            21:14

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	21:15
02 Std. Gas	0.082	21:15
03 Room Air	0.000	21:16
04 Std. Gas	0.083	21:16
05 Room Air	0.000	21:17
06 Std. Gas	0.083	21:17
07 Room Air	0.000	21:18

Lot No = 16512080A2  
Cyl No = 58  
Exp Date = 08/01/2014  
County = 39                            Oper No. = 020401

  
Operator Signature  
MATTHEW ANDERSON

Remarks:

*1<sup>st</sup> ACA test Good test*


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005944  
Location = WAHP      8164.13.00 06/09  
03/26/2014      21:20

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	21:21
02 Std. Gas	0.083	21:21
03 Room Air	0.000	21:22
04 Std. Gas	0.082	21:22
05 Room Air	0.000	21:22
06 Std. Gas	0.083	21:23
07 Room Air	0.000	21:23

Lot No = 16512080A2  
Cyl No = 58  
Exp Date = 08/01/2014  
County = 39      Oper No. = 020401

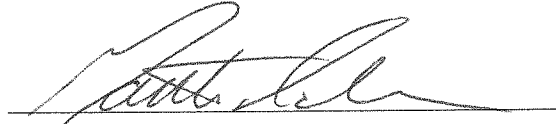

Operator Signature  
MATTHEW ANDERSON

Remarks:

*2nd AAA test*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-005944  
Location = WAHP                    8164.13.00 06/09  
03/26/2014                            21:24

Test	AC	Time
01 Diagnostic	OK	21:25
02 Room Air	RFI*	21:26
03 Room Air	RFI*	21:26

\*Invalid Test  
Inhibited - RFI

Sub Name = RFI, TEST NA  
Sub DOB = 01/01/1990  
Sub Sex = Male                            Weight = 150  
Test = DUI                                Cit = RFI TEST  
Dr. Lic. = ND/RFI TEST  
Lot No = 16512080A2  
Cyl No = 58  
Expiration Date = 08/01/2014  
County = 39                                Oper No. = 020401

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
MATTHEW ANDERSON

Remarks:

