



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 SFN 59281 (03-2014)

Serial Number <i>80-05944</i>	Instrument Location <i>Richland County Jail</i>
Reason for Install/Repair <input type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input checked="" type="checkbox"/> Other (Specify) <i>exchanged old intoxilyzer for new one</i>	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): *None*
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>[Signature]</i>	Date <i>3-26-14</i> ✓
Reviewed By (Crime Laboratory Use Only) <i>Bob Kasher</i>	Date <i>28 May 14</i>

This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Certified By: *Charles E. Edr* Certified Date: *19 Oct. 16*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005944
Location = WAHP 8164.13.00 06/09
03/26/2014 21:20

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-=
ABCDEFGHIJKLMNopQRSTUVWXYZ!@#%&^&*()_+?

abcdefghijklmnopqrstuvwxy1234567890-=
ABCDEFGHIJKLMNopQRSTUVWXYZ!@#%&^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 16512080A2
Standard Cyl #? 58
Standard Expiration? 08/01/2014
Oper No? 020401

Flow Cal. Date: 06/20/2013
Slope 677
Intercept -702620

IR Calibration Date: 06/20/2013
 3um 9um

0th Coef(*100): -20012 -21455
1st Coef(*100): 275397 137470
2nd Coef(*100): 2699 1252
H2O adj(mg/l*10k): 704 563

***** Printer Test End *****

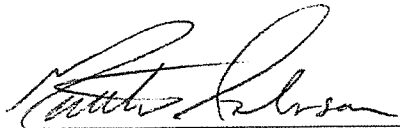

Operator Signature
MATTHEW ANDERSON

Remarks:

Good Print Test


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005944
Location = WAHP 8164.13.00 06/09
03/26/2014 21:14

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	21:15
02 Std. Gas	0.082	21:15
03 Room Air	0.000	21:16
04 Std. Gas	0.083	21:16
05 Room Air	0.000	21:17
06 Std. Gas	0.083	21:17
07 Room Air	0.000	21:18

Lot No = 16512080A2
Cyl No = 58
Exp Date = 08/01/2014
County = 39 Oper No. = 020401

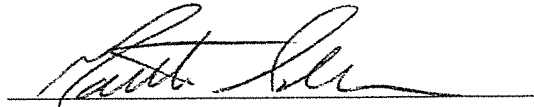

Operator Signature
MATTHEW ANDERSON

Remarks:

1st ACA test 6th test

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005944
Location = WAHP 8164.13.00 06/09
03/26/2014 21:20

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	21:21
02 Std. Gas	0.083	21:21
03 Room Air	0.000	21:22
04 Std. Gas	0.082	21:22
05 Room Air	0.000	21:22
06 Std. Gas	0.083	21:23
07 Room Air	0.000	21:23

Lot No = 16512080A2
Cyl No = 58
Exp Date = 08/01/2014
County = 39 Oper No. = 020401

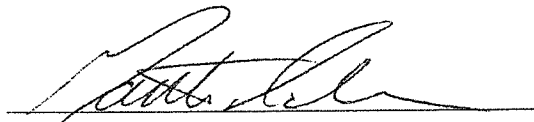

Operator Signature
MATTHEW ANDERSON

Remarks:

2nd ACA test

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005944
Location = WAHP 8164.13.00 06/09
03/26/2014 21:24

Test	AC	Time
01 Diagnostic	OK	21:25
02 Room Air	RFI*	21:26
03 Room Air	RFI*	21:26

*Invalid Test
Inhibited - RFI

Sub Name = RFI, TEST NA
Sub DOB = 01/01/1990
Sub Sex = Male Weight = 150
Test = DUI Cit = RFI TEST
Dr. Lic. = ND/RFI TEST
Lot No = 16512080A2
Cyl No = 58
Expiration Date = 08/01/2014
County = 39 Oper No. = 020401

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
MATTHEW ANDERSON

Remarks:

Good RFI test