



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 SFN 59281 (03-2014)

Serial Number 80-006500	Instrument Location Harvey, ND PD
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): Ø
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>[Signature]</i>	Date 5/28/2016 ✓
Reviewed By (Crime Laboratory Use Only) <i>Deb Shanaver</i>	Date 02 June 16

This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Certified By: *Charles E. Edm* Certified Date: **10 Nov. 16**


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006500
Location = HARV 8164.13.00 06/09
05/28/2016 18:42

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	18:43
02 Std. Gas	0.079	18:43
03 Room Air	0.000	18:44
04 Std. Gas	0.079	18:44
05 Room Air	0.000	18:45
06 Std. Gas	0.079	18:45
07 Room Air	0.000	18:46

Lot No = 19615080A5
Cyl No = 36
Exp Date = 09/05/2017
County = 52 Oper No. = 131106

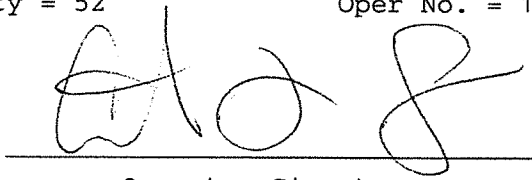

Operator Signature
COURTNEY LEMER

Remarks:

set up

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006500
Location = HARV 8164.13.00 06/09
05/28/2016 18:47

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	18:47
02 Std. Gas	0.079	18:48
03 Room Air	0.000	18:48
04 Std. Gas	0.079	18:49
05 Room Air	0.000	18:49
06 Std. Gas	0.079	18:49
07 Room Air	0.000	18:50

Lot No = 19615080A5
Cyl No = 36
Exp Date = 09/05/2017
County = 52 Oper No. = 131106

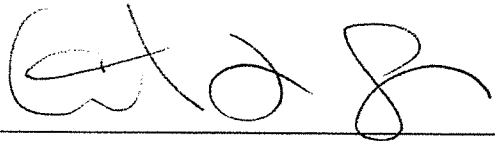

Operator Signature
COURTNEY LEMER

Remarks:

RFI Test #1, unable to get it
to detect radio

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006500
Location = HARV 8164.13.00 06/09
05/28/2016 18:52

Test	AC	Time
01 Room Air	RFI*	18:54
02 Room Air	0.000	18:54

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/08/1987
Sub Sex = Male Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS871456
Lot No = 19615080A5
Cyl No = 36
Expiration Date = 09/05/2017
County = 52 Oper No. = 131106

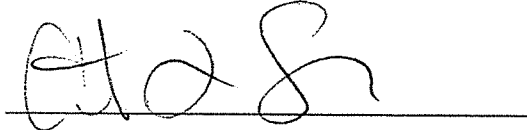

Operator Signature
COURTNEY LEMER

Remarks:

RFI test

Form 106-I8000