

INTOXILYZER 8000 INSPECTION

Serial No.: 80-005360

Location: ASTHL

- Check When Done
- A. Is the warm-up time less than 20 minutes? ✓
 - B. Is a three-pronged grounded outlet used? ✓
 - C. Is the breath tube heated? ✓
 - D. Is the diagnostic check complete? ✓
 - E. Is the time, date, and year correct (re-set if necessary)? ✓
 - F. Print test. (Attach test record.) ✓
 - G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)
Sim SN DR 3845 Lot No. 13070 Rep AC 0.020 ✓
 - H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN DR 3841 Lot No. 13020 Rep AC 0.306 ✓
 - I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.
(Attach test record.)
Sim SN DR 5191 Lot No. ICS#1 Rep AC 0.100 AC + acetone ✓
 - J. RFI check. Run CMS mode. Key radio on first room air.
(Attach test record.) ✓
 - K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.)

Lot No. <u>23411080A1</u>	Cyl No. <u>48</u>	Exp. Date <u>10/01/2013</u>
Test 1 <u>0.081</u>	Test 4 <u>0.081</u>	Test 7 <u>0.081</u>
Test 2 <u>0.082</u>	Test 5 <u>0.080</u>	Test 8 <u>0.081</u>
Test 3 <u>0.081</u>	Test 6 <u>0.081</u>	Test 9 <u>0.081</u>
Average <u>0.081</u>		

L. Tank pressure, Level 3 Func D Sub G P: 964 psi Reg 990 psi

M. Remarks/Maintenance record: Annual inspection

<u>30 May '13</u> Date	<u>Deb Kashner</u> Field Inspector's Signature
<u>31 MAY 13</u> Date	<u>Charles E. Edm</u> Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005360
Location = ASHL 8164.13.00 06/09
05/30/2013 11:41

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:41
02 Std. Sol.	0.017	11:42
03 Room Air	0.000	11:43
04 Std. Sol.	0.016	11:43
05 Room Air	0.000	11:44
06 Std. Sol.	0.018	11:45
07 Room Air	0.000	11:45

08 Sim Temp = 34.0°C

Simul Ser No = DR3845
Std Sol No = 13070
County = 26 Oper No. = 777777

Deb Kasher
Operator Signature
DEB KASHUR

Remarks:

Low AC 0.020 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005360
Location = ASHL 8164.13.00 06/09
05/30/2013 11:57

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:58
02 Std. Sol.	0.299	11:59
03 Room Air	0.000	12:00
04 Std. Sol.	0.299	12:00
05 Room Air	0.000	12:01
06 Std. Sol.	0.300	12:02
07 Room Air	0.000	12:02

08 Sim Temp = 34.0°C

Simul Ser No = DR3841

Std Sol No = 13020

County = 26

Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Linearity test 0.300 AC

Form 106-18000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005360
Location = ASHL 8164.13.00 06/09
05/30/2013 12:18

Test	AC	Time
01 Room Air	0.000	12:19
02 *Subject Test	INT*	12:19
03 Room Air	0.000	12:20

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 23411080A1
Cyl No = 48
Expiration Date = 10/01/2013
County = 26 Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

*Interferent check 0.100 AC
+ acetone*

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005360
Location = ASHL 8164.13.00 06/09
05/30/2013 12:04

Test	AC	Time
01 Diagnostic	OK	12:06
02 Room Air	RFI*	12:06
03 Room Air	0.000	12:06

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 23411080A1
Cyl No = 48
Expiration Date = 10/01/2013
County = 26 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

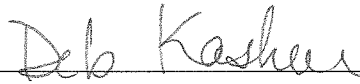

Operator Signature
DEB KASHUR

Remarks:

RFI test


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005360
Location = ASHL 8164.13.00 06/09
05/30/2013 11:27

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:27
02 Std. Gas	0.081	11:28
03 Room Air	0.000	11:28
04 Std. Gas	0.082	11:29
05 Room Air	0.000	11:29
06 Std. Gas	0.081	11:29
07 Room Air	0.000	11:30

Lot No = 23411080A1
Cyl No = 48
Exp Date = 10/01/2013
County = 26 Oper No. = 777777



Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005360
Location = ASHL 8164.13.00 06/09
05/30/2013 11:30

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:31
02 Std. Gas	0.081	11:31
03 Room Air	0.000	11:32
04 Std. Gas	0.080	11:32
05 Room Air	0.000	11:33
06 Std. Gas	0.081	11:33
07 Room Air	0.000	11:33

Lot No = 23411080A1
Cyl No = 48
Exp Date = 10/01/2013
County = 26 Oper No. = 777777

Deb Kashur
Operator Signature
DEB KASHUR

Remarks:
Calibration check 0.080 AC


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005360
Location = ASHL 8164.13.00 06/09
05/30/2013 11:34

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:35
02 Std. Gas	0.081	11:35
03 Room Air	0.000	11:35
04 Std. Gas	0.081	11:36
05 Room Air	0.000	11:36
06 Std. Gas	0.081	11:37
07 Room Air	0.000	11:37

Lot No = 23411080A1
Cyl No = 48
Exp Date = 10/01/2013
County = 26 Oper No. = 777777



Operator Signature
DEB KASHUR

Remarks:
Calibration check 0.080 AC