

INTOXILYZER 8000 INSPECTION

Serial No.: 80-004956 Location: Emmous Co SO
Linton

- Check When Done
- A. Is the warm-up time less than 20 minutes? ✓
 - B. Is a three-pronged grounded outlet used? ✓
 - C. Is the breath tube heated? ✓
 - D. Is the diagnostic check complete? ✓
 - E. Is the time, date, and year correct (re-set if necessary)? ✓
 - F. Print test. (Attach test record.) ✓
 - G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)
Sim SN DR5132 Lot No. — Rep AC 0.021 AC ✓
 - H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN DR3375 Lot No. — Rep AC 0.292 AC ✓
 - I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.
(Attach test record.)
Sim SN DR3454 Lot No. — Rep AC 0.10 AC + Acetone ✓
 - J. RFI check. Run CMS mode. Key radio on first room air.
(Attach test record.) ✓
 - K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.)

Lot No. 709352 Cyl No. 16 Exp. Date 12/23/11
Test 1 0.077 Test 4 0.078 Test 7 0.077
Test 2 0.077 Test 5 0.077 Test 8 0.077
Test 3 0.077 Test 6 0.077 Test 9 0.077
Average 0.077 AC

L. Tank pressure, Level 3 Func D Sub G P: 770 psi Reg 780 psi

M. Remarks/Maintenance record: Annual Inspection MODEM/
Forms Load FAX 284 5311
Date 2 June 2010 map Margy Pearson
Field Inspector's Signature
Date 14 June 2010 Deb Keshner
Reviewed by

80-004956
p. 410 of 48
AK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004956
Location = LINT 8164.13.00 06/09
06/02/2010 10:39

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:40
02 Std. Sol.	0.020	10:41
03 Room Air	0.000	10:41
04 Std. Sol.	0.020	10:42
05 Room Air	0.000	10:42
06 Std. Sol.	0.021	10:43
07 Room Air	0.000	10:44


08 Sim Temp = 34.0°C

Simul Ser No = 0000000

Std Sol No = 000000

County = 15

Oper No. = 107501

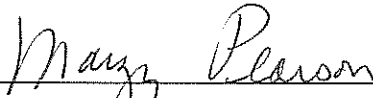

Operator Signature
MARGY PEARSON

Remarks:

Low AC 0.021 AC

Form 106-I8000

80-004956
p. 42 of 48
PW

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004956
Location = LINT 8164.13.00 06/09
06/02/2010 10:33

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:34
02 Std. Sol.	0.283	10:35
03 Room Air	0.000	10:35
04 Std. Sol.	0.285	10:36
05 Room Air	0.000	10:37
06 Std. Sol.	0.285	10:37
07 Room Air	0.000	10:38


08 Sim Temp = 34.0°C

Simul Ser No = 0000000

Std Sol No = 000000

County = 15

Oper No. = 107501

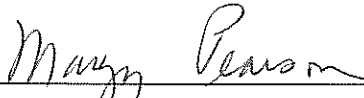

Operator Signature
MARGY PEARSON

Remarks:

Linearity 0.292 AC

Form 106-I8000

80-004956
p. 43 of 48
AM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004956
Location = LINT 8164.13.00 06/09
06/02/2010 10:47

Test	AC	Time
01 Room Air	0.000	10:48
02 *Subject Test	INT*	10:48
03 Room Air	0.000	10:49

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 11/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Simul Ser No = 0000000
Std Sol No = 000000
County = 15 Oper No. = 107501



Operator Signature
MARGY PEARSON

Remarks:

Interferent - Alc + Acetone (0.05%)

Form 106-I8000

80-004956
P. 44 of 48
KEM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004956
Location = LINT 8164.13.00 06/09
06/02/2010 10:49

Test	AC	Time
01 Diagnostic	OK	10:50
02 Room Air	RFI*	10:51
03 Room Air	0.000	10:51

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 11/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Simul Ser No = 0000000
Std Sol No = 000000
County = 15 Oper No. = 107501

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

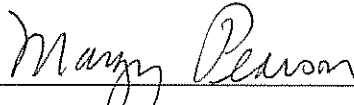

Operator Signature
MARGY PEARSON

Remarks:

RFI

Form 106-I8000

80-004956
P. 45 of 48
RPM


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004956
Location = LINT 8164.13.00 06/09
06/02/2010 10:52

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:53
02 Std. Gas	0.077	10:53
03 Room Air	0.000	10:54
04 Std. Gas	0.077	10:54
05 Room Air	0.000	10:55
06 Std. Gas	0.077	10:55
07 Room Air	0.000	10:56

Lot No = 709352
Cyl No = 16
Exp Date = 12/23/2011
County = 15 Oper No. = 107501



Operator Signature
MARGY PEARSON

Remarks:

Cal Check 0.080 AC

Form 106-I8000

*80-004956
P. 46 of 48
RM*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004956
Location = LINT 8164.13.00 06/09
06/02/2010 10:56

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:57
02 Std. Gas	0.078	10:57
03 Room Air	0.000	10:58
04 Std. Gas	0.077	10:58
05 Room Air	0.000	10:59
06 Std. Gas	0.077	10:59
07 Room Air	0.000	10:59

Lot No = 709352
Cyl No = 16
Exp Date = 12/23/2011
County = 15 Oper No. = 107501

Margy Pearson

Operator Signature
MARGY PEARSON

Remarks:

Cal Check 0.080 AC

Form 106-I8000

80-004956
P. 47 of 48
jtm

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004956
Location = LINT 8164.13.00 06/09
06/02/2010 11:00

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:01
02 Std. Gas	0.077	11:01
03 Room Air	0.000	11:01
04 Std. Gas	0.077	11:02
05 Room Air	0.000	11:02
06 Std. Gas	0.077	11:03
07 Room Air	0.000	11:03

Lot No = 709352
Cyl No = 16
Exp Date = 12/23/2011
County = 15 Oper No. = 107501

Margy Pearson

Operator Signature
MARGY PEARSON

Remarks:

Cal Check 0.080AC

Form 106-I8000

*80-004956
P. 48 of 48
JAN*