



**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**  
 OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION  
 Toxicology Section/Breath Alcohol Program  
 SFN 59281 (5-2011)

Serial Number <b>80-003069</b>	Instrument Location <b>Rolla</b>
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. RFI Test (Use CMS Mode; Key Radio During First Room Air).
- 7. Repair and/or Maintenance Performed (if Any): N/A
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <b>[Signature]</b>	Date <b>6/4/2011</b>
Reviewed By (Crime Laboratory Use Only) <b>[Signature]</b>	Date <b>09 June 2011</b>

80-003069  
 P. 118 of 122  
 KM

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer  
North Dakota Model 8000 SN 80-003069  
Location = ~~FOX~~ Roll 8164.13.00 06/09  
06/04/2011 DK 09 Junell 18:48

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890-=\_|  
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$\$%^&\*()\_+?

abcdefghijklmnopqrstuvwxy1234567890-=\_|  
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$\$%^&\*()\_+?

Current Instrument Setup


Data Entry Mode: Enabled  
Start Test Sequence: DABACABA  
Display Prelim Rslt? Yes  
Display Third Digit? Yes  
Inhib Printer(Y/N)? No  
Display Volume? No  
Disable On Memfull? Yes  
# of Print Copies? 1  
Select Std (D/W/I)? Dry  
Standard Value? 0.080  
Standard Lot #? 05911080A1  
Standard Cyl #? 28  
Standard Expiration? 02/01/2013  
Oper No? 069108

Flow Cal. Date: 04/08/2008  
Slope 691  
Intercept -599264

IR Calibration Date: 06/04/2008  
3um 9um

-----  
0th Coef(\*100): -14879 -13696  
1st Coef(\*100): 280678 136270  
2nd Coef(\*100): 177 615  
H2O adj(mg/l\*10k): 391 185

\*\*\*\*\* Printer Test End \*\*\*\*\*

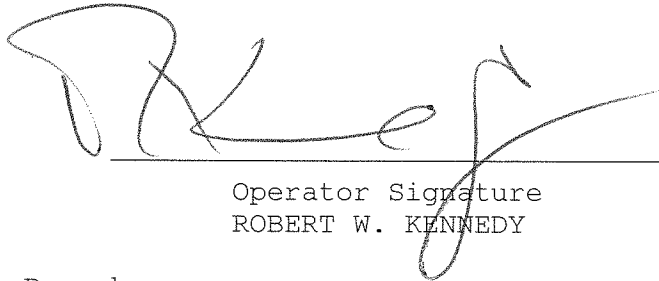

Operator Signature  
ROBERT W. KENNEDY

Remarks:


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003069  
Location = ~~TOXE ROLL~~      8164.13.00 06/09  
06/04/2011      09 Jewell DK      18:48

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	18:49
02 Std. Gas	0.077	18:50
03 Room Air	0.000	18:50
04 Std. Gas	0.077	18:50
05 Room Air	0.000	18:51
06 Std. Gas	0.078	18:51
07 Room Air	0.000	18:52

Lot No = 05911080A1  
Cyl No = 28  
Exp Date = 02/01/2013  
County = 40      Oper No. = 069108

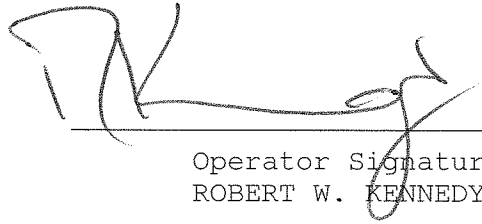  
Operator Signature  
ROBERT W. KENNEDY

Remarks:

Form 106-I8000

80-003069  
P.120 of 122  
JK


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer                      Alcohol Analyzer  
North Dakota Model 8000                      SN 80-003069  
Location = ~~TOXL~~ ROLL                      8164.13.00 06/09  
06/04/2011                      09 June 11 ~~PK~~                      18:53

Test	AC	Time
01 Room Air	RFI*	18:55
02 Room Air	RFI*	18:55

\*Invalid Test  
Inhibited - RFI

Sub Name = TEST, TEST T  
Sub DOB = 07/09/1965  
Sub Sex = Male                      Weight = 222  
Test = DUI                      Cit = 987678  
Dr. Lic. = ND/TES657890  
Lot No = 05911080A1  
Cyl No = 28  
Expiration Date = 02/01/2013  
County = 40                      Oper No. = 069108

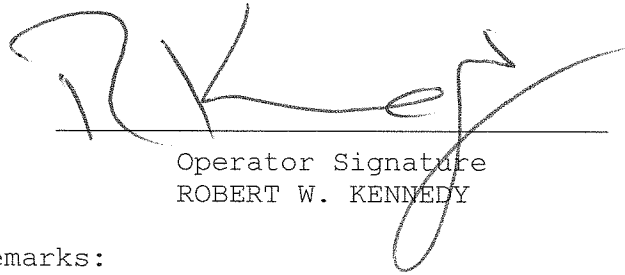

Operator Signature  
ROBERT W. KENNEDY

Remarks:

Form 106-I8000

80-003069  
P121 of 122  
EM

June 9, 2011

I emailed Robert Kennedy regarding 8000 install paperwork for 80-003069. He forgot to change the location code from TOXL to ROLL. He emailed me back stating that he saw it after he completed the install and changed it then.

Deb Kashur

80-003069  
P. 122 of 122  
KTM