

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-004204

Location: CAVA (PEMBINA COUNTY S.O.)

- Check When Done
- A. Is the breath tube heated? ✓
 - B. Is the diagnostic check complete? ✓
 - C. Is the time, date, and year correct (re-set if necessary)? ✓
 - D. Print test. (Attach test record.) ✓
 - E. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)
Sim SN DR5115 Lot No. 12111 Rep AC 0.020 ✓
 - F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN DR5143 Lot No. 13080 Rep AC 0.300 ✓
 - G. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.
(Attach test record.)
Sim SN DR3375 Lot No. JCS#2 Rep AC 0.100 AC + 0.05% ACETONE ✓
 - H. RFI check. Run CMS mode. Key radio on first room air.
(Attach test record.) ✓
 - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.)

Lot No. 16512080A2 Cyl No. 067 Exp. Date 08/01/2014
Test 1 0.080 Test 4 0.080 Test 7 0.081
Test 2 0.079 Test 5 0.081 Test 8 0.080
Test 3 0.080 Test 6 0.080 Test 9 0.080
Average 0.080

J. Tank pressure, Level 3 Func D Sub G P: 268 psi Reg 260 psi

K. Remarks/Maintenance record: REPLACED BREATH HOSE O-RING,

ANNUAL INSPECTION

02 JUN 14
Date

04 June 14 ✓
Date

Charles E. Edwards
Field Inspector's Signature

Deb Keshner
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004204
Location = CAVA 8164.13.00 06/09
06/02/2014 12:59

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:00
02 Std. Sol.	0.020	13:00
03 Room Air	0.000	13:01
04 Std. Sol.	0.021	13:02
05 Room Air	0.000	13:02
06 Std. Sol.	0.022	13:03
07 Room Air	0.000	13:03

08 Sim Temp = 34.0°C

Simul Ser No = DR5115

Std Sol No = 12111

County = 34

Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Low AC
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004204
Location = CAVA 8164.13.00 06/09
06/02/2014 13:04

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:05
02 Std. Sol.	0.308	13:06
03 Room Air	0.000	13:06
04 Std. Sol.	0.309	13:07
05 Room Air	0.000	13:07
06 Std. Sol.	0.308	13:08
07 Room Air	0.000	13:09


08 Sim Temp = 34.0°C

Simul Ser No = DR5143

Std Sol No = 13080

County = 34

Oper No. = 666666

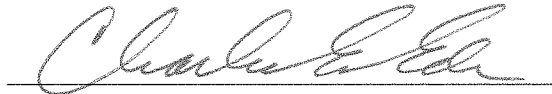

Operator Signature
CHARLES EDER

Remarks:

HIGH AC
0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004204
Location = CAVA 8164.13.00 06/09
06/02/2014 12:52

Test	AC	Time
01 Room Air	0.000	12:53
02 *Subject Test	INT*	12:54
03 Room Air	0.000	12:54

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 16512080A2
Cyl No = 67
Expiration Date = 08/01/2014
County = 34 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *INTERFERENT CHECK*
0.100 AC + 0.05% ACETONE

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004204
Location = CAVA 8164.13.00 06/09
06/02/2014 12:49

Test	AC	Time
01 Diagnostic	OK	12:50
02 Room Air	RFI*	12:50
03 Room Air	0.000	12:51

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 16512080A2
Cyl No = 67
Expiration Date = 08/01/2014
County = 34 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

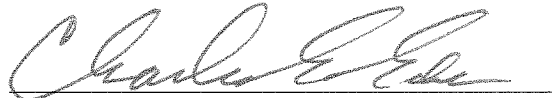

Operator Signature
CHARLES EDER

Remarks: *RFI CHECK*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004204
Location = CAVA 8164.13.00 06/09
06/02/2014 12:55

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:55
02 Std. Gas	0.080	12:55
03 Room Air	0.000	12:56
04 Std. Gas	0.079	12:56
05 Room Air	0.000	12:57
06 Std. Gas	0.080	12:57
07 Room Air	0.000	12:58

Lot No = 16512080A2
Cyl No = 67
Exp Date = 08/01/2014
County = 34 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION
CHECK
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004204
Location = CAVA 8164.13.00 06/09
06/02/2014 13:11

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:11
02 Std. Gas	0.080	13:11
03 Room Air	0.000	13:12
04 Std. Gas	0.081	13:12
05 Room Air	0.000	13:13
06 Std. Gas	0.080	13:13
07 Room Air	0.000	13:14

Lot No = 16512080A2
Cyl No = 67
Exp Date = 08/01/2014
County = 34 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

CALIBRATION
CHECK
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004204
Location = CAVA 8164.13.00 06/09
06/02/2014 13:14

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:15
02 Std. Gas	0.081	13:15
03 Room Air	0.000	13:16
04 Std. Gas	0.080	13:16
05 Room Air	0.000	13:16
06 Std. Gas	0.080	13:17
07 Room Air	0.000	13:17


Lot No = 16512080A2

Cyl No = 67

Exp Date = 08/01/2014

County = 34

Oper No. = 666666

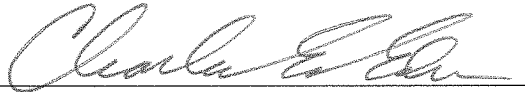

Operator Signature
CHARLES EDER

Remarks:

*CALIBRATION
CHECK
0.080 AC*

Form 106-I8000