

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-004937

Location: TOXL

- Check When Done
- A. Is the breath tube heated? ✓
- B. Is the diagnostic check complete? ✓
- C. Is the time, date, and year correct (re-set if necessary)? ✓
- D. Print test. (Attach test record.) ✓
- E. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)  
Sim SN DR3377 Lot No. 13270 Rep AC 0.020 ✓
- F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)  
Sim SN DR3453 Lot No. 13113 Rep AC 0.400 ✓
- G. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.  
(Attach test record.)  
Sim SN DR5153 Lot No. ICS#2 Rep AC 0.100AC + 0.05% ACETONE ✓  
*SEE DR5135*
- H. RFI check. Run CMS mode. Key radio on first room air.  
(Attach test record.) ✓
- I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.  
(Attach test record.)

Lot No. 16512080A2 Cyl No. 037 Exp. Date 08/01/2014

Test 1 0.081 Test 4 0.082 Test 7 0.081

Test 2 0.081 Test 5 0.082 Test 8 0.082

Test 3 0.082 Test 6 0.082 Test 9 0.082

Average 0.082

J. Tank pressure, Level 3 Func D Sub G P: 984 psi Reg 1000 psi

K. Remarks/Maintenance record: REPLACED BREATH HOSE O-RING, ANNUAL

INSPECTOR

04 JUN 14  
Date

Charles E. Egan  
Field Inspector's Signature

04 June 14  
Date

Deb Kashner  
Reviewed by

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004937  
Location = TOXL                8164.13.00 06/09  
06/04/2014                      15:38

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890-=  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%^&\*()\_+?

abcdefghijklmnopqrstuvwxy1234567890-=  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%^&\*()\_+?

Current Instrument Setup

Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?          1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            16512080A2  
Standard Cyl #?            37  
Standard Expiration?        08/01/2014  
Oper No?                    666666

Flow Cal. Date:            01/18/2010  
Slope                        730  
Intercept                   -779991

IR Calibration Date:        05/24/2011  
                                  3um                            9um

-----  
0th Coef(\*100):            -12537                        -15884  
1st Coef(\*100):            262613                        132554  
2nd Coef(\*100):            1600                            951  
H2O adj(mg/l\*10k):        529                            537

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004937  
Location = TOXL      8164.13.00 06/09  
06/04/2014      15:41

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:41
02 Std. Sol.	0.017	15:42
03 Room Air	0.000	15:43
04 Std. Sol.	0.018	15:43
05 Room Air	0.000	15:44
06 Std. Sol.	0.018	15:45
07 Room Air	0.000	15:45

08 Sim Temp = 34.0°C

Simul Ser No = DR3377  
Std Sol No = 13270  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:      *Low AC*  
                  *0.020 AC*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004937  
Location = TOXL      8164.13.00 06/09  
06/04/2014      15:47

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:48
02 Std. Sol.	0.390	15:49
03 Room Air	0.000	15:49
04 Std. Sol.	0.391	15:50
05 Room Air	0.000	15:50
06 Std. Sol.	0.393	15:51
07 Room Air	0.000	15:52

08 Sim Temp = 34.0°C

Simul Ser No = DR3453  
Std Sol No = 13113  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

HIGH AC  
0.400 AC

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004937  
Location = TOXL                    8164.13.00 06/09  
06/04/2014                            15:54

Test	AC	Time
01 Room Air	0.000	15:56
02 *Subject Test	INT*	15:56
03 Room Air	0.000	15:57

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = INTERFERENT  
Dr. Lic. = ND/DIS821456  
Lot No = 16512080A2  
Cyl No = 37  
Expiration Date = 08/01/2014  
County = 08                            Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: *INTERFERENT  
CHECK  
0.100 AC + 0.05% ACETONE*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004937  
Location = TOXL                 8164.13.00 06/09  
06/04/2014                        15:57

Test	AC	Time
01 Diagnostic	OK	15:58
02 Room Air	RFI*	15:59
03 Room Air	0.000	15:59

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                 Weight = 150  
Test = DUI                        Cit = RFI CHECK  
Dr. Lic. = ND/DIS821456  
Lot No = 16512080A2  
Cyl No = 37  
Expiration Date = 08/01/2014  
County = 08                        Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
CHARLES EDER

Remarks:

*RFI CHECK*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004937  
Location = TOXL      8164.13.00 06/09  
06/04/2014      15:59

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:00
02 Std. Gas	0.081	16:00
03 Room Air	0.000	16:01
04 Std. Gas	0.081	16:01
05 Room Air	0.000	16:02
06 Std. Gas	0.082	16:02
07 Room Air	0.000	16:03

Lot No = 16512080A2  
Cyl No = 37  
Exp Date = 08/01/2014  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

CALIBRATION  
CHECK  
0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004937  
Location = TOXL      8164.13.00 06/09  
06/04/2014      16:03

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:04
02 Std. Gas	0.082	16:04
03 Room Air	0.000	16:04
04 Std. Gas	0.082	16:05
05 Room Air	0.000	16:05
06 Std. Gas	0.082	16:06
07 Room Air	0.000	16:06

Lot No = 16512080A2  
Cyl No = 37  
Exp Date = 08/01/2014  
County = 08      Oper No. = 666666

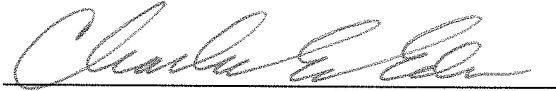

Operator Signature  
CHARLES EDER

Remarks: *CALIBRATION  
CHECK  
0.080 AC*



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004937  
Location = TOXL                    8164.13.00 06/09  
06/04/2014                            16:07

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:07
02 Std. Gas	0.081	16:07
03 Room Air	0.000	16:08
04 Std. Gas	0.082	16:08
05 Room Air	0.000	16:09
06 Std. Gas	0.082	16:09
07 Room Air	0.000	16:10

Lot No = 16512080A2  
Cyl No = 37  
Exp Date = 08/01/2014  
County = 08                            Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION  
CHECK  
0.080AC