



**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**  
 OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION  
 Toxicology Section/Breath Alcohol Program  
 SFN 59281 (5-2011)

Serial Number: <u>80-003058</u>	Instrument Location: <u>Walsh Co.</u>
Reason for Install/Repair:	
<input checked="" type="checkbox"/> Install After Receiving from Crime Laboratory	<input type="checkbox"/> Install After Location Change
<input type="checkbox"/> Other (Specify)	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location). (Level 2, Function E)
- 5. Scan/Enter Gas Cylinder Information. (Level 1, Function S)
- 6. Run Tests:
  - A. Print Test (Level 1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. RFI Test (Use CMS Mode – Key Radio During First Room Air).
- 7. Repair and/or Maintenance Performed (if Any): N/A
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN 50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN 50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN 59281, Form 104-G), Print Test, ACA Test, and RFI Test.

[Signature]  
 Field Inspector Signature

6/5/11  
 Date

Deb Kashur  
 Reviewed By

08 June 2011  
 Date




Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003058  
Location = GRAF      8164.13.00 06/09  
06/05/2011      00:55

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	00:56
02 Std. Gas	0.078	00:56
03 Room Air	0.000	00:57
04 Std. Gas	0.079	00:57
05 Room Air	0.000	00:58
06 Std. Gas	0.079	00:58
07 Room Air	0.000	00:59

Lot No = 05911080A1  
Cyl No = 8  
Exp Date = 02/01/2013  
County = 50      Oper No. = 049907

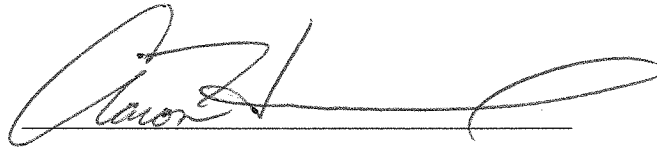

Operator Signature  
AARON HUMMEL

Remarks: *ACA Test - New Install*

Form 106-I8000

*80-003058  
p. 105 of 106  
vfm*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003058  
Location = GRAF      8164.13.00 06/09  
06/05/2011      00:59

Test	AC	Time
01 Diagnostic	OK	01:00
02 Room Air	RFI*	01:00
03 Room Air	0.000	01:01

\*Invalid Test  
Inhibited - RFI

Sub Name = DOE, JOHN NA  
Sub DOB = 01/01/1999  
Sub Sex = Male      Weight = NA  
Test = DUI      Cit = 000000  
Dr. Lic. = ND/0000000  
Lot No = 05911080A1  
Cyl No = 8  
Expiration Date = 02/01/2013  
County = 50      Oper No. = 049907

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

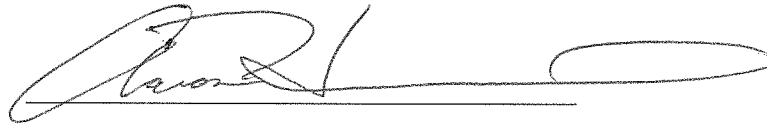

Operator Signature  
AARON HUMMEL

Remarks:

*RFI test on Install*

Form 106-I8000

80-003058  
p. 106 of 106  
JEM