



ETHANOL BREATH STANDARD CYLINDER REPORT

OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
Toxicology Section/Breath Alcohol Program
SFN 59282 (5-2011)

Chemical Test Operator Name (Print) Chad Thompson #3524		
Location Cass Jail	Intoxilyzer® Serial Number 80-005939	
Gas Lot Number 33913080 A4	Gas Cylinder Number 45	Gas Expiration Date 1-1-16

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 - A. 0. 079 AC
 - B. 0. 080 AC
 - C. 0. 081 AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Breath Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature [Signature] 3524	Date 6-6-14 ✓
Reviewed By (Crime Laboratory Use Only) Deb Kashur	Date 11 June 14

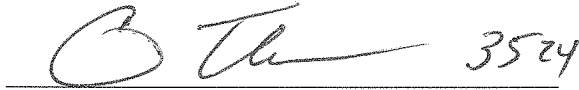
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005939
Location = CASS 8164.13.00 06/09
06/06/2014 02:36

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	02:37
02 Std. Gas	0.079	02:37
03 Room Air	0.000	02:38
04 Std. Gas	0.080	02:38
05 Room Air	0.000	02:38
06 Std. Gas	0.081	02:39
07 Room Air	0.000	02:39

Lot No = 33913080A4
Cyl No = 45
Exp Date = 01/01/2016
County = 09 Oper No. = 050910

 3524

Operator Signature
CHAD THOMPSON

Remarks: Calibration Check after new
cylinder install