

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-004943

Location: Minot Air Force Base

- Check When Done
- A. Is the breath tube heated? ✓
 - B. Is the diagnostic check complete? ✓
 - C. Is the time, date, and year correct (re-set if necessary)? ✓
 - D. Print test. (Attach test record.) ✓
 - E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN DR513 Lot No. 0.02^{DS} Rep AC 0.020
201507F ✓
 - F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN DR3378 Lot No. 16060 Rep AC 0.300 ✓
 - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN DR3803 Lot No. 1C55 Solution 0.10AC + 0.05%²⁰
acetone ✓
 - H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) ✓
 - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) 0.080 AC

Lot No. 01316080A2 Cyl No. 30 Exp. Date 02/05/2018

Test 1 <u>0.078</u>	Test 4 <u>0.079</u>	Test 7 <u>0.078</u>
Test 2 <u>0.079</u>	Test 5 <u>0.078</u>	Test 8 <u>0.079</u>
Test 3 <u>0.078</u>	Test 6 <u>0.078</u>	Test 9 <u>0.079</u>

Average 0.078

J. Tank pressure, Level 3 Func D Sub G P: 993 psi Reg 1000 psi

K. Remarks/Maintenance record: Annual inspection, replaced breath
hose O-ring

06 June 17
Date

Dib Shanaver
Field Inspector's Signature

06 June 17
Date

Roberta Sueger-Hummo
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004943
Location = MNAF 8164.14.00 09/16
06/06/2017 11:18

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-=
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxy1234567890-=
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup
Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? No
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 01316080A2
Standard Cyl #? 30
Standard Expiration? 02/05/2018
Oper No? 777777

Flow Cal. Date: 01/18/2010
Slope 693
Intercept -665387

IR Calibration Date: 05/21/2015

	3um	9um
0th Coef(*100):	-28195	-26668
1st Coef(*100):	268915	137995
2nd Coef(*100):	103	992
H2O adj(mg/l*10k):	222	308

***** Printer Test End *****

Deb Shanaver
Operator Signature
DEB SHANAVER

Remarks:
Print test

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004943
Location = MNAF 8164.14.00 09/16
06/06/2017 11:32

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:33
02 Std. Sol.	0.017	11:34
03 Room Air	0.000	11:34
04 Std. Sol.	0.017	11:35
05 Room Air	0.000	11:35
06 Std. Sol.	0.017	11:36
07 Room Air	0.000	11:37

08 Sim Temp = 34.0°C

Simul Ser No = DR5113

Std Sol No = 201507F

County = 64

Oper No. = 77777

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

low AC 0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004943
Location = MNAF 8164.14.00 09/16
06/06/2017 11:38

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:39
02 Std. Sol.	0.299	11:40
03 Room Air	0.000	11:40
04 Std. Sol.	0.301	11:41
05 Room Air	0.000	11:41
06 Std. Sol.	0.302	11:42
07 Room Air	0.000	11:43

08 Sim Temp = 34.0°C

Simul Ser No = DR3378

Std Sol No = 16060

County = 64

Oper No. = 777777

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

High AC 0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004943
Location = MNAF 8164.14.00 09/16
06/06/2017 11:45

Test	AC	Time
01 Room Air	0.000	11:46
02 *Subject Test	INT*	11:47
03 Room Air	0.000	11:47

*Invalid Test
Interferent Detected

Sub Name = TEST, NA NA
Sub DOB = 08/26/1978
Sub Sex = Unknown Weight = NA
Test = DUI Cit = NA
Dr. Lic. = ND/NA
Lot No = 01316080A2
Cyl No = 30
Expiration Date = 02/05/2018
County = 64 Oper No. = 777777

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

Interferent detected 0.10AC + 0.05% acetone

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004943
Location = MNAF 8164.14.00 09/16
06/06/2017 11:48

Test	AC	Time
01 Diagnostic	OK	11:49
02 Room Air	0.000	11:49
03 *Subject Test	RFI*	11:49
04 Room Air	0.000	11:50

*Invalid Test
 Inhibited - RFI

Sub Name = RFI, NA NA
Sub DOB = 09/21/1945
Sub Sex = Unknown Weight = NA
Test = DUI Cit = NA
Dr. Lic. = ND/NA
Lot No = 01316080A2
Cyl No = 30
Expiration Date = 02/05/2018
County = 64 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
DEB SHANAVER

Remarks:

RFI test

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004943
Location = MNAF 8164.14.00 09/16
06/06/2017 11:20

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:21
02 Std. Gas	0.078	11:21
03 Room Air	0.000	11:21
04 Std. Gas	0.079	11:22
05 Room Air	0.000	11:22
06 Std. Gas	0.078	11:23
07 Room Air	0.000	11:23

Lot No = 01316080A2

Cyl No = 30

Exp Date = 02/05/2018

County = 64

Oper No. = 777777

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

Calibration check 0.080AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004943
Location = MNAF 8164.14.00 09/16
06/06/2017 11:24

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:24
02 Std. Gas	0.079	11:25
03 Room Air	0.000	11:25
04 Std. Gas	0.078	11:25
05 Room Air	0.000	11:26
06 Std. Gas	0.078	11:26
07 Room Air	0.000	11:27

Lot No = 01316080A2
Cyl No = 30
Exp Date = 02/05/2018
County = 64 Oper No. = 777777

Deb Shanaver
Operator Signature
DEB SHANAVER

Remarks:

Calibration check 0.080AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004943
Location = MNAF 8164.14.00 09/16
06/06/2017 11:28

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:28
02 Std. Gas	0.078	11:29
03 Room Air	0.000	11:29
04 Std. Gas	0.079	11:29
05 Room Air	0.000	11:30
06 Std. Gas	0.079	11:30
07 Room Air	0.000	11:31

Lot No = 01316080A2
Cyl No = 30
Exp Date = 02/05/2018
County = 64 Oper No. = 777777

Deb Shanaver
Operator Signature
DEB SHANAVER

Remarks:
Calibration check 0.080 AC