

INTOXILYZER 8000 INSPECTION

Serial No.: 80-004206

Location: TOXL

- | | Check When Done |
|--|-----------------|
| A. Is the warm-up time less than 20 minutes? | <u>✓</u> |
| B. Is a three-pronged grounded outlet used? | <u>✓</u> |
| C. Is the breath tube heated? | <u>✓</u> |
| D. Is the diagnostic check complete? | <u>✓</u> |
| E. Is the time, date, and year correct (re-set if necessary)? | <u>✓</u> |
| F. Print test. (Attach test record.) | <u>✓</u> |
| G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)
Sim SN <u>DR5144</u> Lot No. <u> </u> Rep AC <u>0.021</u> | <u>✓</u> |
| H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN <u>DR5114</u> Lot No. <u> </u> Rep AC <u>0.298</u> | <u>✓</u> |
| I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.
(Attach test record.)
Sim SN <u>DR5186</u> Lot No. <u> </u> Rep AC <u>0.100</u> | <u>✓</u> |
| J. RFI check. Run CMS mode. Key radio on first room air.
(Attach test record.) | <u>✓</u> |
| K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) | |

Lot No. <u>709352</u>	Cyl No. <u>38</u>	Exp. Date <u>23 Dec 2011</u>
Test 1 <u>0.079</u>	Test 4 <u>0.079</u>	Test 7 <u>0.080</u>
Test 2 <u>0.080</u>	Test 5 <u>0.079</u>	Test 8 <u>0.079</u>
Test 3 <u>0.079</u>	Test 6 <u>0.079</u>	Test 9 <u>0.079</u>
Average <u>0.079</u>		

L. Tank pressure, Level 3 Func D Sub G P: 627 psi Reg 630 psi

M. Remarks/Maintenance record: Recalibration, Annual inspection ^{OK}

07 June 2010
Date

Deb Keshur
Field Inspector's Signature

14 June 2010
Date

Mary Plann
Reviewed by

80-004206
p. 86 of 94
JAN

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004206
Location = TOXL 8164.13.00 06/09
06/07/2010 14:36

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 709352
Standard Cyl #? 38
Standard Expiration? 12/23/2011
Oper No? 777777

Flow Cal. Date: 06/10/2009
Slope 690
Intercept -645883

IR Calibration Date: 06/07/2010

	3um	9um
0th Coef(*100):	-17579	-21810
1st Coef(*100):	254057	127149
2nd Coef(*100):	4591	1650
H2O adj(mg/l*10k):	756	795

***** Printer Test End *****

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Print test

Form 106-I8000

80-004206
P. 87 of 94
JEM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004206
Location = TOXL 8164.13.00 06/09
06/07/2010 14:38

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:38
02 Std. Sol.	0.019	14:40
03 Room Air	0.000	14:40
04 Std. Sol.	0.020	14:41
05 Room Air	0.000	14:41
06 Std. Sol.	0.019	14:42
07 Room Air	0.000	14:43

08 Sim Temp = 34.0°C

Simul Ser No = 00000

Std Sol No = 000

County = 08

Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

low AC 0.021 AC

Form 106-18000

80-004206
P. 88 of 94
KM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004206
Location = TOXL 8164.13.00 06/09
06/07/2010 14:44

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:44
02 Std. Sol.	0.298	14:45
03 Room Air	0.000	14:45
04 Std. Sol.	0.297	14:46
05 Room Air	0.000	14:47
06 Std. Sol.	0.298	14:47
07 Room Air	0.000	14:48

08 Sim Temp = 34.0°C

Simul Ser No = 00000

Std Sol No = 000

County = 08

Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Linearity test 0.298 AC

Form 106-I8000

80-004206
P. 87 of 94
JK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004206
Location = TOXL 8164.13.00 06/09
06/07/2010 14:50

Test	AC	Time
01 Room Air	0.000	14:51
02 *Subject Test	INT*	14:51
03 Room Air	0.000	14:52

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 709352
Cyl No = 38
Expiration Date = 12/23/2011
County = 08 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Interferent detected 0.100 AC + acetone

Form 106-I8000

80-004206
P. 90 of 94
EM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004206
Location = TOXL 8164.13.00 06/09
06/07/2010 14:54

Test	AC	Time
01 Diagnostic	OK	14:55
02 Room Air	RFI*	14:55
03 Room Air	0.000	14:55

*Invalid Test
 Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 709352
Cyl No = 38
Expiration Date = 12/23/2011
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

RFI check

Form 106-I8000

80-004206
P. 7/14/11
YH

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004206
Location = TOXL 8164.13.00 06/09
06/07/2010 14:58

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:58
02 Std. Gas	0.079	14:59
03 Room Air	0.000	14:59
04 Std. Gas	0.080	14:59
05 Room Air	0.000	15:00
06 Std. Gas	0.079	15:00
07 Room Air	0.000	15:01

Lot No = 709352
Cyl No = 38
Exp Date = 12/23/2011
County = 08 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000

80-004206
P. 92-194
KSH

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004206
Location = TOXL 8164.13.00 06/09
06/07/2010 15:02

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:02
02 Std. Gas	0.079	15:03
03 Room Air	0.000	15:03
04 Std. Gas	0.079	15:04
05 Room Air	0.000	15:04
06 Std. Gas	0.079	15:04
07 Room Air	0.000	15:05

Lot No = 709352

Cyl No = 38

Exp Date = 12/23/2011

County = 08

Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000

80-004206
P. 93 of 94
JKM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004206
Location = TOXL 8164.13.00 06/09
06/07/2010 16:27

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:28
02 Std. Gas	0.080	16:28
03 Room Air	0.000	16:29
04 Std. Gas	0.079	16:29
05 Room Air	0.000	16:30
06 Std. Gas	0.079	16:30
07 Room Air	0.000	16:31

Lot No = 709352
Cyl No = 38
Exp Date = 12/23/2011
County = 08 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000

80-004206
P. 94/94
JAN