



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
Toxicology Section/Breath Alcohol Program
SFN 59281 (03-2014)

Serial Number 80-005952	Instrument Location Burleigh County Sheriff's Department
Reason for Install/Repair <input type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input checked="" type="checkbox"/> Other (Specify) <u>Switch of Field Inspectors</u>	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): N/A
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Rebecca Kopp</i>	Date <i>6-8-15</i> ✓
Reviewed By (Crime Laboratory Use Only) <i>Deb Kasher</i>	Date <i>15 June 15</i>

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005952
Location = BCSO 8164.13.00 06/09
06/08/2015 11:00

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-==|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-==|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 13314080A3
Standard Cyl #? 25
Standard Expiration? 06/05/2016
Oper No? 041005

Flow Cal. Date: 06/21/2013
Slope 734
Intercept -865305

IR Calibration Date: 06/21/2013
 3um 9um

0th Coef(*100): -21281 -23057
1st Coef(*100): 292132 137092
2nd Coef(*100): 2334 1231
H2O adj(mg/l*10k): 497 482

***** Printer Test End *****

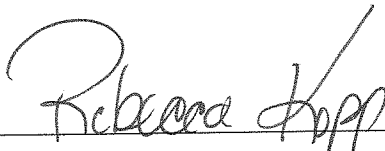

Operator Signature
REBECCA KOPP

Remarks:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005952
Location = BCSO 8164.13.00 06/09
06/08/2015 10:55

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:55
02 Std. Gas	0.081	10:56
03 Room Air	0.000	10:56
04 Std. Gas	0.081	10:57
05 Room Air	0.000	10:57
06 Std. Gas	0.080	10:57
07 Room Air	0.000	10:58

Lot No = 13314080A3
Cyl No = 25
Exp Date = 06/05/2016
County = 08 Oper No. = 041005


Operator Signature
REBECCA KOPP

Remarks:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005952
Location = BCSO 8164.13.00 06/09
06/08/2015 11:01

DRY CAL CHECK

Test	AC	Time
01 Room Air	RFI*	11:01
02 Room Air	RFI*	11:02

*Invalid Test
Inhibited RFI

Lot No = 13314080A3
Cyl No = 25
Exp Date = 06/05/2016
County = 08 Oper No. = 041005

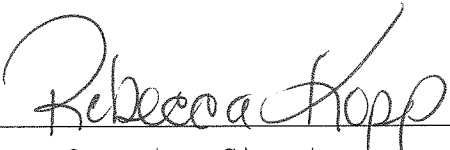

Operator Signature
REBECCA KOPP

Remarks: