



# INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

OFFICE OF ATTORNEY GENERAL  
CRIME LABORATORY DIVISION  
Toxicology Section/Breath Alcohol Program  
SFN 59281 (03-2014)

Serial Number 80-005952	Instrument Location Burleigh County Sheriff's Department
Reason for Install/Repair <input type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input checked="" type="checkbox"/> Other (Specify) <u>Switch of Field Inspectors</u>	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): N/A
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Robert Koop</i>	Date 6-8-15 ✓
Reviewed By (Crime Laboratory Use Only) <i>Deb Kasher</i>	Date 15 June 15

This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Certified By: *Charles E. Ede* Certified Date: 21 Nov. 16

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-005952  
Location = BCSO                    8164.13.00 06/09  
06/08/2015                            11:00

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-|=|  
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$\$%^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-|=|  
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$\$%^&\*()\_+?

Current Instrument Setup


Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?          1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            13314080A3  
Standard Cyl #?            25  
Standard Expiration?        06/05/2016  
Oper No?                    041005

Flow Cal. Date:            06/21/2013  
Slope                      734  
Intercept                  -865305

IR Calibration Date:        06/21/2013  
                                  3um                            9um

-----  
0th Coef(\*100):            -21281                    -23057  
1st Coef(\*100):            292132                    137092  
2nd Coef(\*100):            2334                      1231  
H2O adj(mg/l\*10k):        497                        482

\*\*\*\*\* Printer Test End \*\*\*\*\*

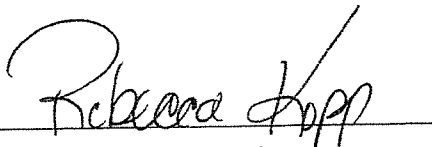

Operator Signature  
REBECCA KOPP

Remarks:


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005952  
Location = BCSO      8164.13.00 06/09  
06/08/2015      10:55

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:55
02 Std. Gas	0.081	10:56
03 Room Air	0.000	10:56
04 Std. Gas	0.081	10:57
05 Room Air	0.000	10:57
06 Std. Gas	0.080	10:57
07 Room Air	0.000	10:58

Lot No = 13314080A3  
Cyl No = 25  
Exp Date = 06/05/2016  
County = 08      Oper No. = 041005

  
Operator Signature  
REBECCA KOPP

Remarks:

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005952  
Location = BCSO      8164.13.00 06/09  
06/08/2015      11:01

DRY CAL CHECK

Test	AC	Time
01 Room Air	RFI*	11:01
02 Room Air	RFI*	11:02

\*Invalid Test  
Inhibited RFI

Lot No = 13314080A3  
Cyl No = 25  
Exp Date = 06/05/2016  
County = 08      Oper No. = 041005

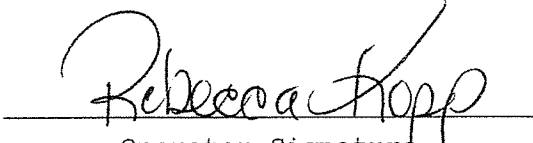  
Operator Signature  
REBECCA KOPP

Remarks: