



**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**  
 OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION  
 Toxicology Section/Breath Alcohol Program  
 SFN 59281 (5-2011)

Serial Number <i>80-004947</i>	Instrument Location <i>Walsh Co SD</i>
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A.  Print Test (Level 1, Function P).
  - B.  ACA Test (Level 1, Function C).
  - C.  RFI Test (Use CMS Mode; Key Radio During First Room Air).
- 7. Repair and/or Maintenance Performed (if Any): *install during annual inspection*
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- NA* 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Deb Kashner</i>	Date <i>10</i> <i>OK 26 June 2013</i>
Reviewed By (Crime Laboratory Use Only) <i>Charles E. Eder</i>	Date <i>17 JUN 13</i>

This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Certified By: *Charles E. Eder*      Certified Date: *23 Nov. 2016*



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004947  
Location = GRAF      8164.13.00 06/09  
06/10/2013      19:04

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	19:05
02 Std. Gas	0.078	19:05
03 Room Air	0.000	19:06
04 Std. Gas	0.078	19:06
05 Room Air	0.000	19:06
06 Std. Gas	0.078	19:07
07 Room Air	0.000	19:07

Lot No = 16512080A2  
Cyl No = 46  
Exp Date = 08/01/2014  
County = 50      Oper No. = 777777

Deb Kasher  
Operator Signature  
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004947  
Location = GRAF                8164.13.00 06/09  
06/10/2013                        19:08

Test	AC	Time
01 Diagnostic	OK	19:08
02 Room Air	RFI*	19:08
03 Room Air	0.000	19:09

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                Weight = 150  
Test = DUI                        Cit = NA  
Dr. Lic. = ND/DIS921456  
Lot No = 16512080A2  
Cyl No = 46  
Expiration Date = 08/01/2014  
County = 50                        Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
DEB KASHUR

Remarks:

*RFI check*

Form 106-I8000