



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
Toxicology Section/Breath Alcohol Program
SFN 59281 (1-2010)

Serial Number: <u>80-004954</u>	Instrument Location: <u>FYPD</u>
Reason for Install/Repair: <input type="checkbox"/> Install After Lab Repair/Inspection <input checked="" type="checkbox"/> Other (Specify) <u>Moved in Office / Corrections</u>	

Check When Done:

- A. Surge Protector Installed/Properly Grounded.
- B. Telephone Line Connected to Intoxilyzer® 8000.
- C. Breath Tube Heated.
- D. Review/Enter Preliminary Data Entry (Level 2, Function E).
- E. Review/Enter Gas Setup (Level 1, Function S).
- F. Print Test (Level 1, Function P). (Attach Test Record.)
- G. ACA Test (Level 1, Function C). (Attach Test Record.)
- H. RFI Test (Use CMS Mode-First Room Air). (Attach Test Record.)
- I. Repair and/or Maintenance Performed: NONE

[Signature]
Field Inspector Signature

06-11-10
Date

Deb Kashner
Reviewed By

17 June 2010
Date

80-004954
P. 53 of 56
[Signature]

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004954
Location = FYPD 8164.13.00 06/09
06/10/2010 23:52

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? No
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 709352
Standard Cyl #? 26
Standard Expiration? 12/23/2011
Oper No? 010606

Flow Cal. Date: 01/19/2010
Slope 679
Intercept -671831

IR Calibration Date: 01/19/2010

	3um	9um
0th Coef(*100):	-34134	-33955
1st Coef(*100):	257873	132608
2nd Coef(*100):	1059	850
H2O adj(mg/l*10k):	762	516

***** Printer Test End *****

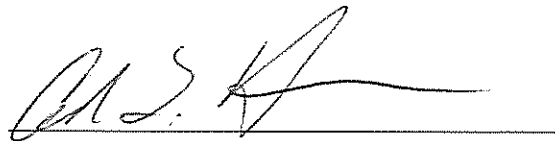

Operator Signature
CHAD L. HARMON

Remarks:


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004954
Location = FYPD 8164.13.00 06/09
06/10/2010 23:52

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	23:53
02 Std. Gas	0.078	23:53
03 Room Air	0.000	23:54
04 Std. Gas	0.079	23:54
05 Room Air	0.000	23:55
06 Std. Gas	0.079	23:55
07 Room Air	0.000	23:55

Lot No = 709352
Cyl No = 26
Exp Date = 12/23/2011
County = 61 Oper No. = 010606

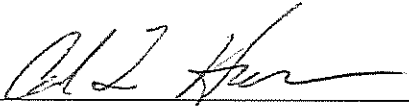

Operator Signature
CHAD L. HARMON

Remarks:

Form 106-I8000

80-004954
P. 55 of 56
JAM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004954
Location = FYPD 8164.13.00 06/09
06/11/2010 00:02

Test	AC	Time
01 Diagnostic	OK	00:04
02 Room Air	RFI*	00:04
03 Room Air	0.000	00:04

*Invalid Test
 Inhibited - RFI

Sub Name = HARMON, CHAD LEROY
Sub DOB = 02/10/1973
Sub Sex = Male Weight = 200
Test = OTH Cit = NA
Dr. Lic. = SD/01241218
Lot No = 709352
Cyl No = 26
Expiration Date = 12/23/2011
County = 61 Oper No. = 010606

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

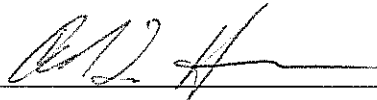

Operator Signature
CHAD L. HARMON

Remarks:

Form 106-I8000

80-004954
p. 56 of 56
JRM