



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59281 (06/2018)

Serial Number <i>80-004935</i>	Instrument Location <i>EMMONS County</i>
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Property Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if any): *None*
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>[Signature]</i>	Date <i>6/14/22</i>
---	------------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By <i>Janelle Pertscheller</i>	Certified Date <i>6/14/22</i>
--	----------------------------------

12 char/line
abcdefghijkl
ABCDEFGHIJKL

16 char per line
abcdefghijklmnop
ABCDEFGHIJKLMN

21 chars per line
abcdefghijklmnopqrstu
ABCDEFGHIJKLMN

24 characters per line
abcdefghijklmnopqrstuwx
ABCDEFGHIJKLMN

32 characters per line
abcdefghijklmnopqrstuwx123456
ABCDEFGHIJKLMN

42 characters per line
abcdefghijklmnopqrstuwx1234567890abcdef
ABCDEFGHIJKLMN

Intoxilyzer Test Record and Checklist
NDDAG Crime Lab. Div., Bismarck, ND 58501

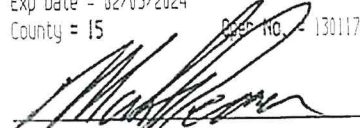
CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004935
Location = LINT 8164.14.00 09/16
06/14/2022 12:32

DRY CAL CHECK

Test	AC	Time
01 Room Air	RFI*	12:33
02 Room Air	0.000	12:33

*Invalid Test
Inhibited - RFI

Lot No = 34921080A1
Cyl No = 56
Exp Date = 02/05/2024
County = 15 Cert No. = 130117


Operator Signature
MARK PEARSON

Remarks:

Form 106-18000

Current Instrument Setup
 Data Entry Mode: Enabled
 Start Test Sequence: DABACABA
 Display Prelim Rslt? Yes
 Display Third Digit? Yes
 Inhib Printer(Y/N)? No
 Display Volume? No
 Disable On Menu!?: Yes
 # of Print Copies? 1
 Select Std (0/W/I)? Dry
 Standard Value? 0.080
 Standard Lot #: 34921080A1
 Standard Cyl #: 56
 Standard Expiration? 02/05/2024
 Oper No? 130117

Flow Cal. Date: 07/12/2012
Slope: 655
Intercept: -510289

IR Calibration Date: 05/20/2015

	Sum	Sum
0th Coef(*100):	-7871	-19684
1st Coef(*100):	264505	133108
2nd Coef(*100):	2865	1320
H2O adj(mg/l*10K): 200		298

***** Test End *****


Operator Signature
MARK PEARSON


Intoxilyzer Test Record and Checklist
NDODJ Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004935
Location = LINT 0154.14.00 09/16
05/14/2022 12:09

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.009	12:10
02 Std. Gas	0.077	12:10
03 Room Air	0.006	12:11
04 Std. Gas	0.075	12:11
05 Room Air	0.000	12:12
06 Std. Gas	0.076	12:12
07 Room Air	0.000	12:13

Lot No = 3492.000A1
Cyl No = 56
Exp Date = 02/05/2024
County = 15 Cert No. = 130117


Operator Signature
MARK PEARSON

Remarks: