



**ETHANOL GAS STANDARD CYLINDER REPORT**  
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM  
 SFN 59282 (08/2018)

|  |  |  |
|--|--|--|
| Chemical Test Operator Name (Print)<br><i>Tanner Smith</i> |  |  |
| Location<br><i>Walsh Co. LEC</i>                           | Intoxilyzer® Serial Number<br><i>80-004948</i> |  |
| Gas Lot Number<br><i>02621080A1</i>                        | Gas Cylinder Number<br><i>006</i>              | Gas Expiration Date<br><i>03/05/2023</i> |

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).  
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).  
 A. 0.080 AC  
 B. 0.081 AC  
 C. 0.081 AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
  - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
  - B. ACA Test Record.

|  |                          |
|--|--------------------------|
| Chemical Test Operator Signature<br><i>T. Smith 254</i>                | Date<br><i>6/17/2022</i> |
| Reviewed By (Crime Laboratory Use Only)<br><i>Janelle Pertschiller</i> | Date<br><i>6/22/22</i>   |

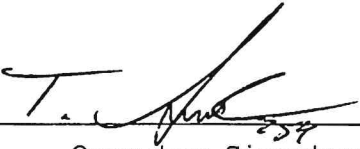
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004948  
Location = GRAF      8164.14.00 09/16  
06/17/2022      15:33

DRY CAL CHECK

| Test        | AC    | Time  |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 15:34 |
| 02 Std. Gas | 0.080 | 15:34 |
| 03 Room Air | 0.000 | 15:35 |
| 04 Std. Gas | 0.081 | 15:35 |
| 05 Room Air | 0.000 | 15:36 |
| 06 Std. Gas | 0.081 | 15:36 |
| 07 Room Air | 0.000 | 15:37 |

Lot No = 02621080A1  
Cyl No = 6  
Exp Date = 03/05/2023  
County = 50      Oper No. = 132660

  
\_\_\_\_\_  
Operator Signature  
TANNER SMITH

Remarks:

ACA Test

Form 106-I8000