



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
Toxicology Section/Breath Alcohol Program
SFN 59281 (5-2011)

Serial Number 80-004942	Instrument Location UNDPD
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (Use CMS Mode; Key Radio During First Room Air).
- 7. Repair and/or Maintenance Performed (if Any): None
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>H. Long H. Midonig #258</i>	Date 06-18-2013 ^{DK} 06-16-2013
Reviewed By (Crime Laboratory Use Only) <i>Deb Keshner</i>	Date 21 June 13

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004942
Location = UNDP 8164.13.00 06/09
06/11/2013 13:55

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890--|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890--|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 16512080A2
Standard Cyl #? 23
Standard Expiration? 08/01/2014
Oper No? 119409

Flow Cal. Date: 08/21/2012
Slope 638
Intercept -487461

IR Calibration Date: 05/30/2013
 3um 9um

0th Coef(*100):	-17966	-17324
1st Coef(*100):	263079	132742
2nd Coef(*100):	2170	1338
H2O adj(mg/l*10k):	723	660

***** Printer Test End *****


Operator Signature
TRACY MEIDINGER

Remarks:

Printer Test

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004942
Location = UNDP 8164.13.00 06/09
06/11/2013 13:56

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:57
02 Std. Gas	0.080	13:57
03 Room Air	0.000	13:58
04 Std. Gas	0.080	13:58
05 Room Air	0.000	13:59
06 Std. Gas	0.080	13:59
07 Room Air	0.000	14:00

Lot No = 16512080A2

Cyl No = 23

Exp Date = 08/01/2014

County = 18

Oper No. = 119409

 #258

Operator Signature
TRACY MEIDINGER

Remarks:

Form 106-I8000

ACA

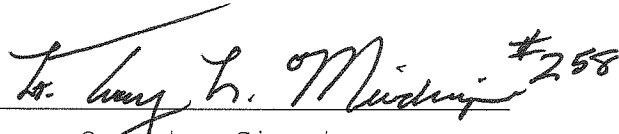
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004942
Location = UNDP 8164.13.00 06/09
06/11/2013 14:01

Test	AC	Time
01 Room Air	RFI*	14:03
02 Room Air	0.000	14:03

*Invalid Test
Inhibited - RFI

Sub Name = BIG, JIM LONLY
Sub DOB = 12/12/1970
Sub Sex = Male Weight = 200
Test = DUI Cit = 4444444444
Dr. Lic. = ND/444444444
Lot No = 16512080A2
Cyl No = 23
Expiration Date = 08/01/2014
County = 18 Oper No. = 119409


Operator Signature
TRACY MEIDINGER

Remarks:

Form 106-I8000

RFI Test

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004942
Location = UNDP 8164.13.00 06/09
06/18/2013 09:23

Test	AC	Time
01 Diagnostic	OK	09:24
02 Room Air	RFI*	09:25
03 Room Air	0.000	09:25

*Invalid Test
Inhibited - RFI

Sub Name = SALL, BIG JOHN
Sub DOB = 12/12/1912
Sub Sex = Male Weight = 255
Test = DUI Cit = 55555555
Dr. Lic. = ND/23232323
Lot No = 16512080A2
Cyl No = 23
Expiration Date = 08/01/2014
County = 18 Oper No. = 119409

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

 #258
Operator Signature
TRACY MEIDINGER

Remarks:

RFI-Test