



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
Toxicology Section/Breath Alcohol Program
SFN 59281 (5-2011)

Serial Number 80-003065	Instrument Location 6F50 (6F County Jail)
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (Use CMS Mode; Key Radio During First Room Air).
- 7. Repair and/or Maintenance Performed (if Any): N/A
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Jon Lauri</i>	Date 06-20-13
Reviewed By (Crime Laboratory Use Only) <i>Deb Rasker</i>	Date 24 June 13

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003065
Location = GFSO 8164.13.00 06/09
06/20/2013 23:40

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	23:41
02 Std. Gas	0.080	23:41
03 Room Air	0.000	23:41
04 Std. Gas	0.079	23:42
05 Room Air	0.000	23:42
06 Std. Gas	0.080	23:43
07 Room Air	0.000	23:43

Lot No = 16512080A2
Cyl No = 43
Exp Date = 08/01/2014
County = 18 Oper No. = 039947


Operator Signature
JONATHAN LAMPI

Remarks:

ACA test for install

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

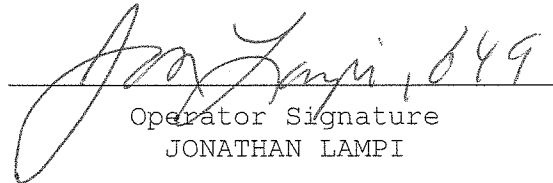
CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003065
Location = GFSO 8164.13.00 06/09
06/20/2013 23:45

Test	AC	Time
01 Diagnostic	OK	23:46
02 Room Air	RFI*	23:46
03 Room Air	0.000	23:47

*Invalid Test
Inhibited - RFI

Sub Name = INSTALL, INSTALL I
Sub DOB = 01/01/1990
Sub Sex = Male Weight = 220
Test = DUI Cit = 12345
Dr. Lic. = ND/12345
Lot No = 16512080A2
Cyl No = 43
Expiration Date = 08/01/2014
County = 18 Oper No. = 039947

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.


Operator Signature
JONATHAN LAMPI

Remarks:

RFI test for install

Form 106-I8000