



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
Toxicology Section/Breath Alcohol Program
SFN 59281 (5-2011)

Serial Number 80-004936	Instrument Location LaMoure County Courthouse
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (Use CMS Mode; Key Radio During First Room Air).
- 7. Repair and/or Maintenance Performed (if Any): _____
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>[Signature]</i>	Date 6-24-11
Reviewed By (Crime Laboratory Use Only) Deb Kashner	Date 29 June 2011

80-004936
P. 89 of 92
SM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004936
Location = LAMO 8164.13.00 06/09
06/24/2011 16:53

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-==|
ABCDEFGHIJKLMNopQRSTUVWXYZ!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxy1234567890-==|
ABCDEFGHIJKLMNopQRSTUVWXYZ!@#\$\$%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 05911080A1
Standard Cyl #? 35
Standard Expiration? 02/01/2013
Oper No? 020828

Flow Cal. Date: 01/18/2010
Slope 732
Intercept -767769

IR Calibration Date: 06/09/2011

	3um	9um
0th Coef(*100):	-2249	-2916
1st Coef(*100):	245643	121920
2nd Coef(*100):	3450	1789
H2O adj(mg/l*10k):	557	621

***** Printer Test End *****

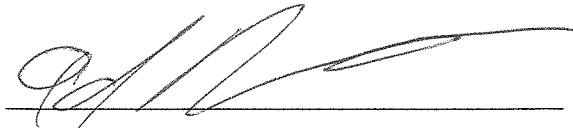

Operator Signature
ADAM R. KRABBENHOFT

Remarks: *Print test*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004936
Location = LAMO 8164.13.00 06/09
06/24/2011 16:56

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:56
02 Std. Gas	0.076	16:57
03 Room Air	0.000	16:57
04 Std. Gas	0.076	16:58
05 Room Air	0.000	16:58
06 Std. Gas	0.076	16:59
07 Room Air	0.000	16:59

Lot No = 05911080A1
Cyl No = 35
Exp Date = 02/01/2013
County = 23 Oper No. = 020828


Operator Signature
ADAM R. KRABBENHOFT

Remarks: *ACA TEST*

Form 106-I8000

80-004936
P. 91 + 92
VEM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004936
Location = LAMO 8164.13.00 06/09
06/24/2011 17:22

Test	AC	Time
01 Diagnostic	OK	17:24
02 Room Air	RFI*	17:25
03 Room Air	0.000	17:25

*Invalid Test
Inhibited - RFI

Sub Name = SUBJECT, TEST Q
Sub DOB = 01/01/1989
Sub Sex = Male Weight = 130
Test = DUI Cit = 01234567
Dr. Lic. = ND/SUB890110
Lot No = 05911080A1
Cyl No = 35
Expiration Date = 02/01/2013
County = 23 Oper No. = 020828

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

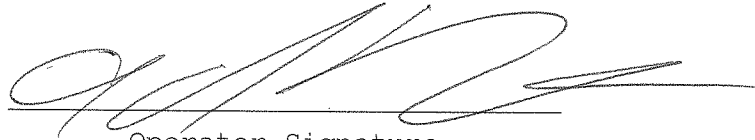

Operator Signature
ADAM R. KRABBENHOFT

Remarks:

RFI TEST

Form 106-I8000

80-004936
P.92 / 92
EM